

Parts Book

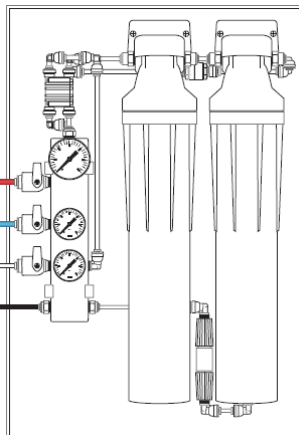


Table of Contents


Table of Contents	2-4
Statim 2000 Front View	5
Statim 2000 Rear View	6
Statim 2000 Bottom View	7
Statim 2000 Top View No Cover	8
Statim 2000 Revision 5XX & Higher Software Alex Parts	9
Statim 2000 Individual Parts Pictures	10-17
Statim 2000 G4 Front View	18
Statim 2000 G4 Rear View	19
Statim 2000 G4 Top View No Cover	20
Statim 2000 G4 View Inside Cover	21
Statim 5000 Front View	22
Statim 5000 Rear View	23
Statim 5000 Top View No Cover	24
Statim 5000 Revision 5XX & Higher Software Alex Parts	25
Statim 5000 Individual Parts Pictures	26-32
Statim 5000 G4 Front View	33
Statim 5000 G4 Rear View	34
Statim 5000 G4 Rear No Cover	35
Statim 5000 G4 Front Left No Cover	36
Statim 5000 G4 View Inside Cover	37
Statim 5000 G4 Cassette	38
Statim 7000 Front View	39
Statim 7000 Front Accessories & Rear View	40
Statim 7000 Front & Side View No Cover	41
Statim 7000 Rear View No Cover	42
Statim 7000 Cassette	43
Statim 7000 Waste, Condenser & Overflow Bottles	44
Statim 900 Front View	45
Statim 900 Rear View	46
Statim 900 Left Side No Cover	47
Statim 900 Right Side No Cover	48
Statim 900 Parts Top View No Cover	49
Statim 900 Individual Parts Pictures	50-53
Bravo Accessories	54
Bravo Parts Table 1	55
Bravo Parts List for Table 1	56
Bravo Parts Table 2	57
Bravo Parts List for Table 2	58
Bravo Parts Table 3	59
Bravo Parts List for Table 3 & Bravo Software	60
Bravo Parts Table 4	61
Bravo Parts List for Table 4	62
Bravo Parts Table 4A & 4B	63
Bravo Parts List for Tables 4A & 4B	64
Bravo Parts Table 5	65
Bravo Parts List for Table 5	66
Bravo Parts Table 6	67

Bravo Parts List for Table 6	68
Bravo Parts Table 7	69
Bravo Parts List for Table 7	70
Bravo Parts Fuses	71
Fittings Pressure Transducer & Safety Switch	72
Bravo Individual Parts Pictures	73-77
Bravo Water Intake Pull Kit #01-111774S	78
Bravo Direct to Drain Kit #01-111775S	79
Quantim Front Door Closed	80
Quantim Front Door Open	81
Quantim Rear	82
Quantim Right Side No Cover	83
Quantim Front Right Side Behind Bezel	84
Quantim Rear No Cover	85
Quantim Left Side Lower Front & Rear	86
Quantim Left Side Upper Front	87
Quantim Bottom View	88
Hydrim C51W w/Push Buttons Front View Door Closed	89
Hydrim C51W w/Push Buttons Front View Door Open	90
Hydrim C51W w/Push Buttons Right Side	91
Hydrim C51W w/Push Buttons Right Side & Inside Chamber with Dryer	92
Hydrim C51W w/Push Buttons Bottom View	93
Hydrim C51W w/Push Buttons Spare Parts Update 06-TSB-230	94
Hydrim C51W w/Push Buttons Bur Screen	95
Hydrim C51W w/Touchscreen Accessories	96
Hydrim C51W w/Touchscreen Front View	97
Hydrim C51W w/Touchscreen Front View Door Open w/Trolley	98
Hydrim C51W w/Touchscreen Front View Door Open No Trolley	99
Hydrim C51W w/Touchscreen Rear View	100
Hydrim C51W w/Touchscreen Right Side No Cover	101
Hydrim C51W w/Touchscreen Right Side Behind Chemical Bracket	102
Hydrim C51W w/Touchscreen Left Side No Cover	103
Hydrim C51W w/Touchscreen Top No Cover	104
Hydrim C51W w/Touchscreen Behind Controller Facia	105
Hydrim C51W w/Touchscreen Bottom No Cover	106
Hydrim C51W w/Touchscreen Individual Parts Pictures	107-109
Hydrim C61W G4 Accessories	110
Hydrim C61W G4 Front Right View	111
Hydrim C61W G4 Front View Door Open with Trolley & Cassette Rack	112
Hydrim C61W G4 Front View Door Open No Trolley	113
Hydrim C61W G4 Front View Kick Plate Open	114
Hydrim C61W G4 Front View No Door Cover	115
Hydrim C61W G4 Left Side No Cover	116
Hydrim C61W G4 Right Side No Cover	117
Hydrim C61W G4 Rear View	118
Hydrim C61W G4 Rear View No Cover	119
Hydrim C61W G4 Bottom View No Cover	120
Hydrim C61W G4 Top View No Cover	121
Hydrim L110W Solid Door Front Door Closed	122
Hydrim L110W Solid Door Front Door Open	123
Hydrim L110W Solid Door Left Side No Cover	124

Hydrim L110W Solid Door Right Side No Cover	125
Hydrim L110W Solid Door Inlet Solenoids & Controllers	126
Hydrim L110W Solid Door Preventative Maintenance Kit	127
Hydrim L110W w/Glass Door Front View	128
Hydrim L110W w/Glass Door Chamber w/Trolleys	129
Hydrim L110W w/Glass Door Chamber	130
Hydrim L110W w/Glass Door Under Mesh Filter & Inside Detergent Door	131
Hydrim L110W w/Glass Door Control Boards	132
Hydrim L110W w/Glass Door Behind Front Kickplate	133
Hydrim L110W w/Glass Door Upper Right Side	134
Hydrim L110W w/Glass Door Lower Right Side	135
Hydrim L110W w/Glass Door Rear View No Cover	136
Hydrim L110W w/Glass Door Top View No Cover	137
Hydrim L110W G4 Accessories	138
Hydrim L110W G4 Front View	139
Hydrim L110W G4 Covers (serial #400114I00013 & Below)	140
Hydrim L110W G4 Covers (serial #400114I00014 & Above)	141
Hydrim L110W G4 Chamber	142
Hydrim L110W G4 Front View No Kickplate	143
Hydrim L110W G4 Left Side View No Cover	144
Hydrim L110W G4 Right Side View Upper No Cover	145
Hydrim L110W G4 Right Side View Lower No Cover	146
Hydrim L110W G4 Rear View No Cover	147
Hydrim L110W G4 Top View No Cover	148
Statmatic Parts	149-153
Statmatic Couplers	154
Aquastat Boiler	155
Aquastat Chassis	156
Aquastat Reservoir	157
Aquastat Individual Parts Pictures	158-161
VistaBrite	162-164
VistaCool Single V7501	165
VistaCool Double V7502	166

Statim 2000

Front View

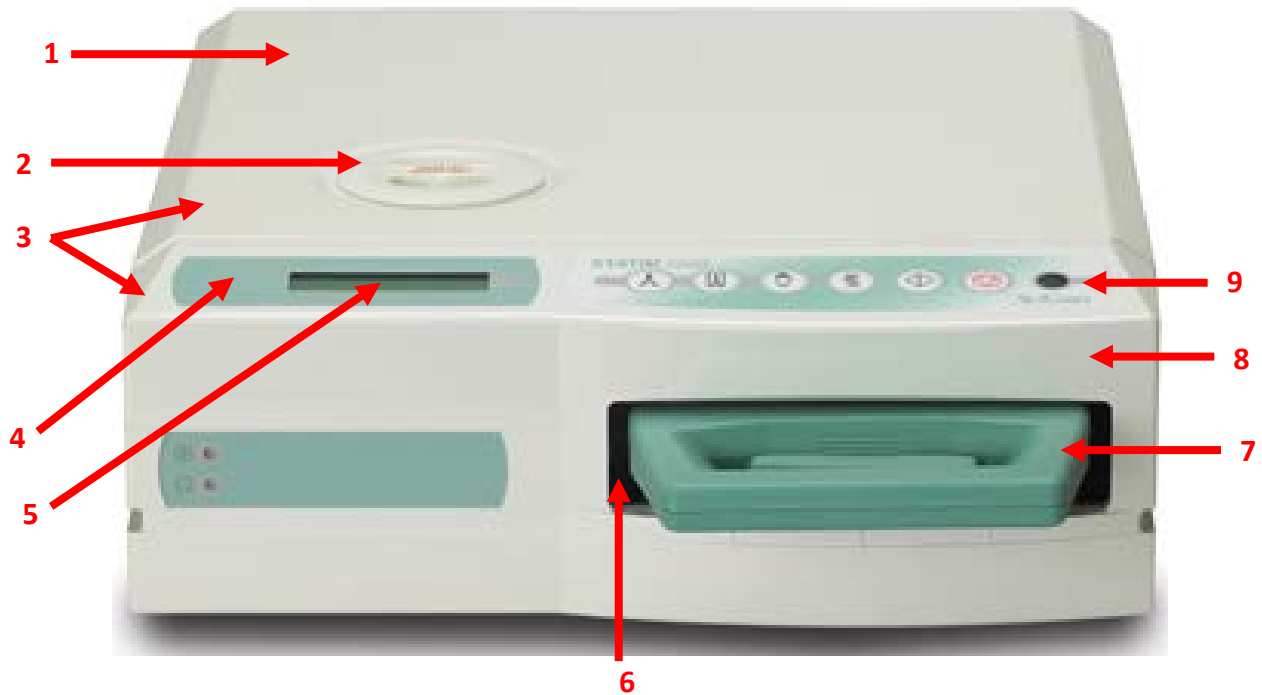

Item #	Part #	Description
1	01-101611S	Cover without Facia
2	01-101783S	Reservoir Cap & Filter
2	01-109300S	Filter Only for Reservoir (under Reservoir Cap)
3	01-100564S	Cover Complete for Model 101102
3	01-106121S	Cover Complete for Model 01-121101
4	01-106950S	LCD Decal Only
5	01-104381S	LCD includes Decal (located inside Facia)
6	01-103856S	Isoplate (not field replaceable requires alignment jig)
7	01-100008A	Cassette Complete
8	01-100563S	Facia Complete for Model 101102
8	01-106122S	Facia Complete for Model 01-121101
8	01-103518S	Facia without LCD for Model 101102
8	01-106124S	Facia without LCD for Model 01-121101
9	01-101707S	Keypad (old style not shown)
9	01-109680S	Keypad (pictured above)
Not Visible	01-100842S	Facia Gasket (located inside Facia)
Not Shown	01-112867S	Silicone Mat 2000

Statim 2000


Rear View



Item #	Part #	Description
1	01-100564S	Cover Complete (Model #101102)
1	01-106121S	Cover Complete (Model #01-121101)
2	01-104284S	Biological Filter Bracket (behind Biological Filter)
3	01-102119S	Biological Filter (not on all units)
4	01-100560S	Air Compressor
4	01-100207S	Air Filter (behind cover of Air Compressor)
5	01-107817S	Fitting $\frac{3}{8}$ " for Biological Filter Tubes (Pkg. of 2)
6	01-100782S	Push-In-Fitting
7	01-106087S	Line Filter
8	01-100573S	Power Switch
9	01-106675S	Label Kit (Pkg. of 10)
10	01-101783S	Reservoir Cap & Filter
10	01-109300S	Filter Only for Reservoir (under Reservoir Cap)

Statim 2000


Bottom View

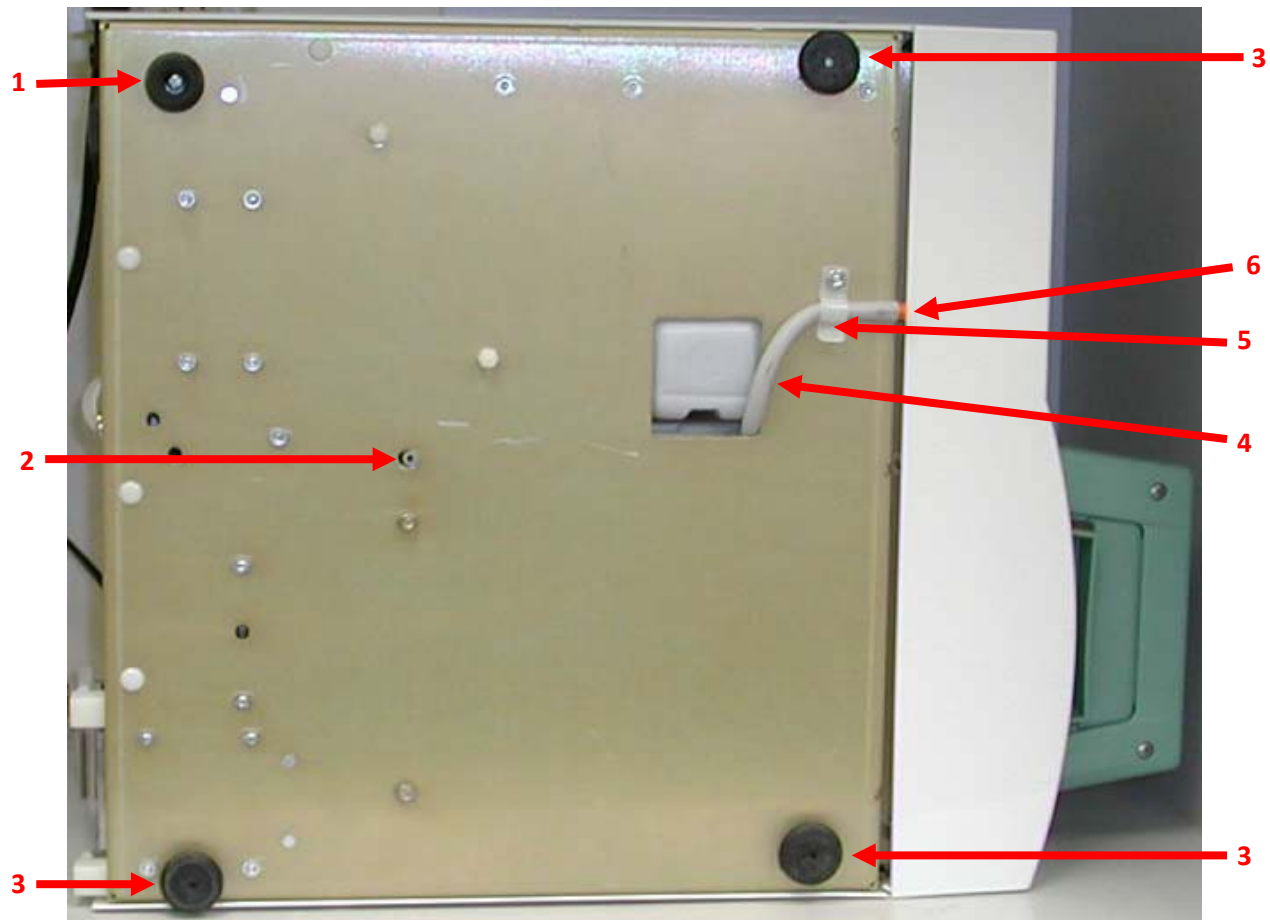

Item #	Part #	Description
1	01-100780S	Bumper
2	74-100335A	Drain Tube 1"
3	01-101553S	Leveler
3	01-104180S	Leveler Repair Kit (includes Leveler & Insert for Leveler)
4	01-108152S	Drain Tubing Kit
5	01-112091S	Drain Tube Clip
6	01-104343S	Plug for Drain Tubing

Statim 2000



Top View No Cover

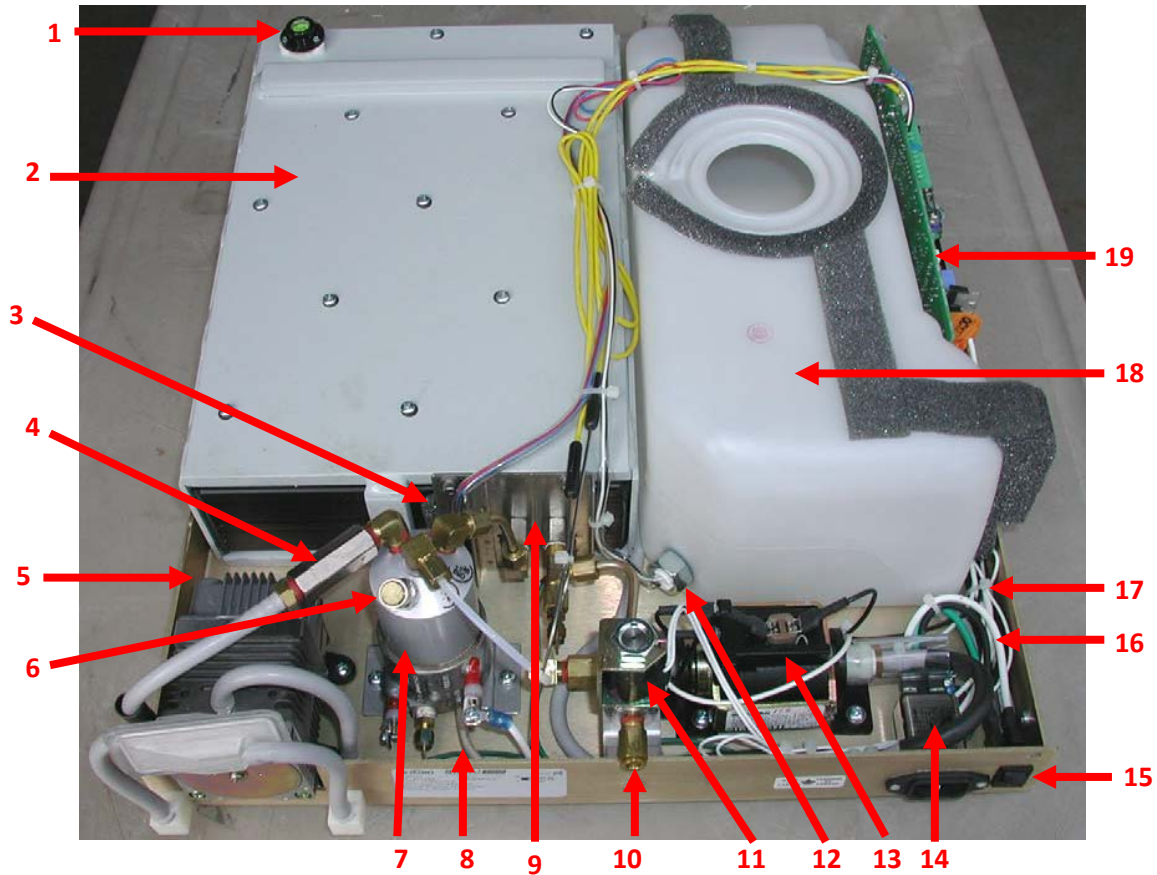

Item #	Part #	Description
1	01-100684S	Bubble Level
2	01-100561S	Armature (not field replaceable requires alignment jig)
3	01-100571S	Microswitch (on Probe Bracket behind Boiler)
4	01-100574S	Check Valve
5	01-100560S	Compressor
6	01-106787S	Pressure Relief Valve
7	01-108979S	Boiler (software must be revision 5XX or higher to replace in field)
8	01-104461S	Thermal Fuse (under Boiler)
9	01-100570S	Probe Bracket (not field replaceable requires complete calibration)
10	01-100782S	Push-In-Fitting
11	01-100557S	Solenoid
12	01-103571S	Water Quality Sensor
13	01-100568S	Pump
13	01-110445S	Pump Ulka (Revision 6XX or higher software)
14	01-106087S	Line Filter
15	01-100573S	Power Switch
16	01-108541S	Power Cable Filter/Switch
17	01-106349S	PCB Power Cable
18	01-110868S	Reservoir with Water Quality Sensor
18	01-100565S	Reservoir with Float Switch (not shown)
18	01-106459S	Reservoir with Water Quality Sensor & Float Switch
19	01-109365S	PCB Rev. 7 (not field replaceable requires complete calibration)


Statim 2000 Revision 5XX & Higher Software

Alex Parts

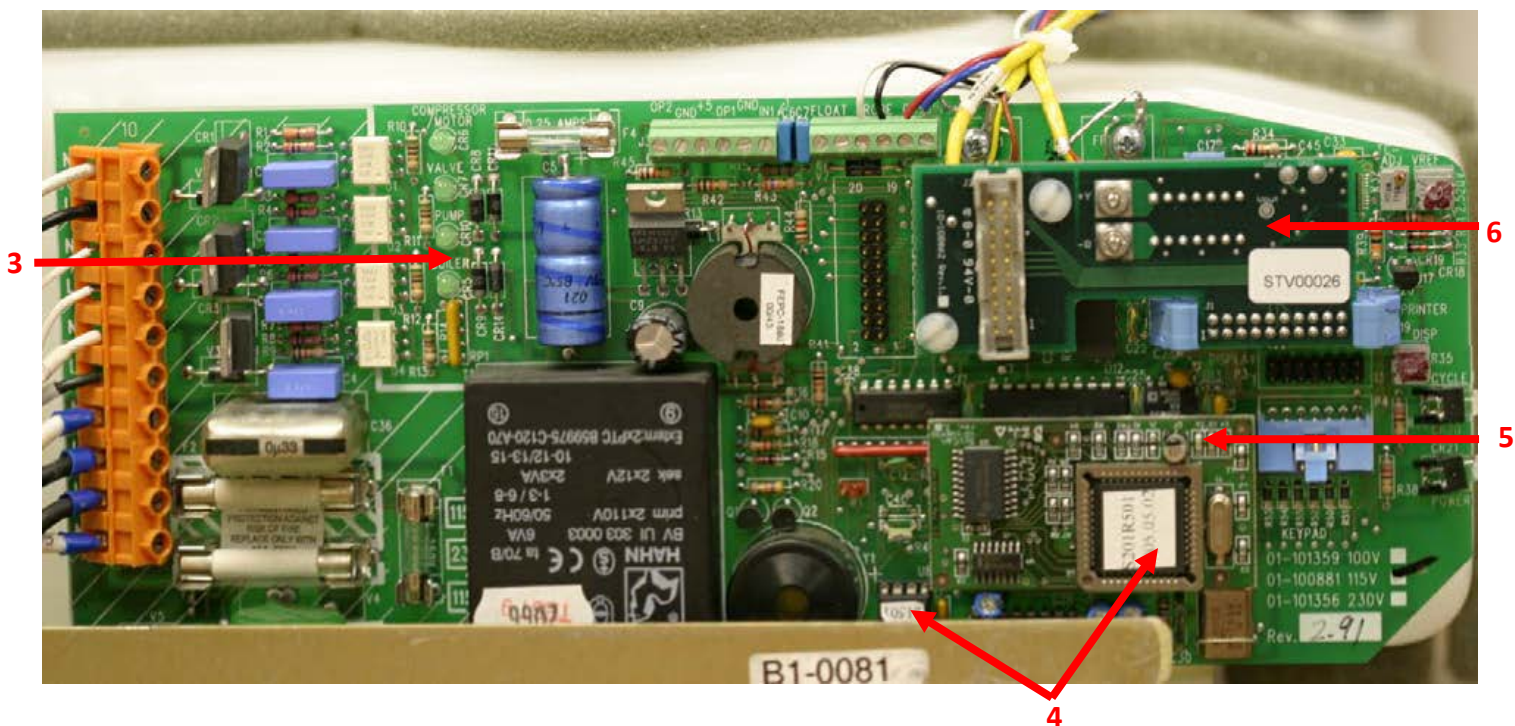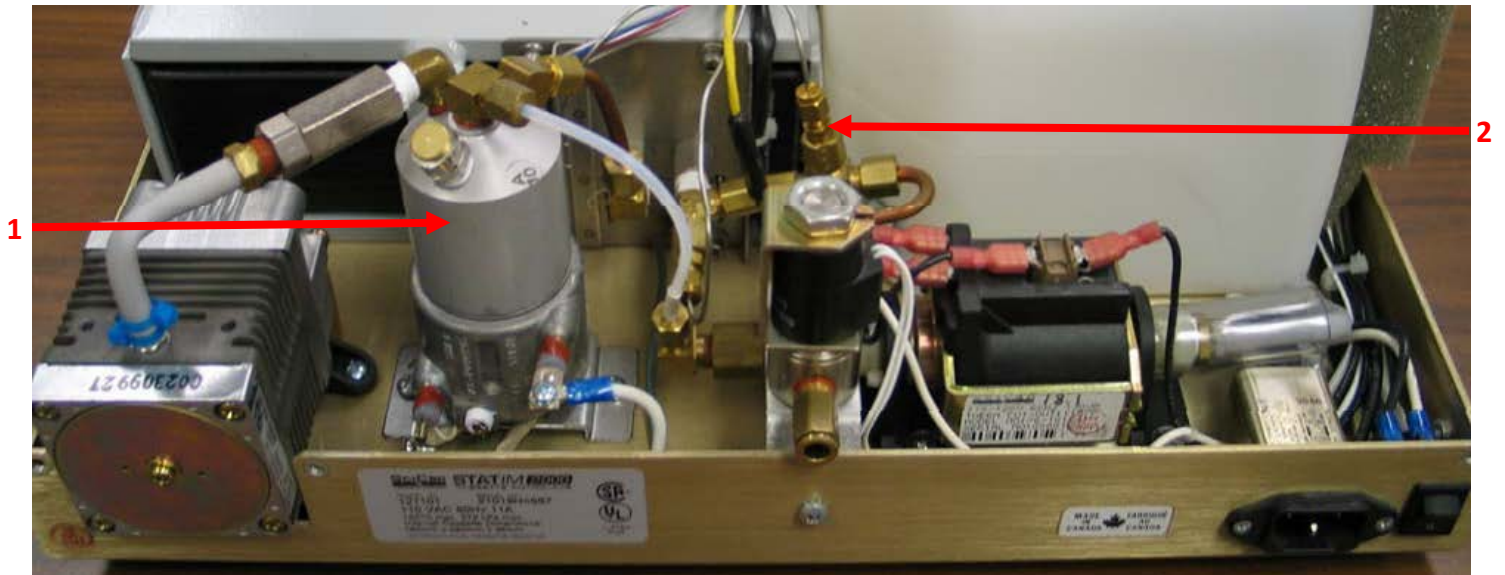

Item #	Part #	Description
1	01-108979S	Boiler (software must be revision 5XX or higher to replace in field)
2	01-108983S	Validation Thermocouple Statim 2000
3	01-109365S	PCB Rev. 7 (not field replaceable requires complete calibration)
4	01-108976S	Microprocessor Kit Revision 5XX Software
4	01-109385S	Microprocessor Kit Revision 6XX Software for Rev. 7 PCB
5	01-108985S	PCB Adapter Board Revision 5XX Software Statim 2000
6	01-108981S	Temperature Adapter Board for Alex with Validation Thermocouple

Statim 2000 Parts

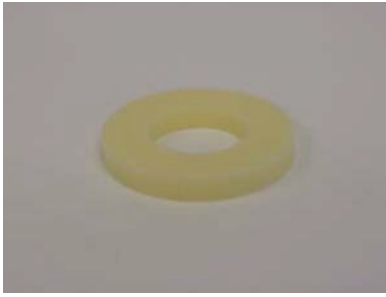


Air Filter
#01-100207S



Alex Kit 2XX Software
#01-108996S



Alex Kit 4XX Software
#01-108993S

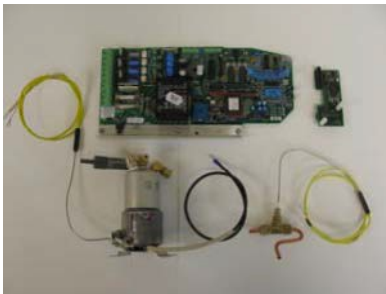

Alex Kit PCB Rev 3.0 & 4.2
#01-109716S




Allen Key 9/64
#01-103469S



Armature
#01-100561S



Biological Filter
#01-102119S

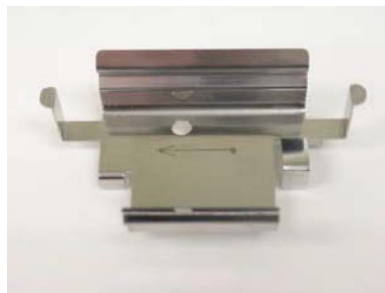

Biological Filter Bracket
#01-104284S



Boiler Alex
#01-108979S



Bubble Level
#01-100684S



Bumper
#01-100780S



Cassette Complete
#01-100008A


Statim 2000 Parts



Cassette Lid
#01-100834S



Cassette Seal & Lube Kit
#01-100028S

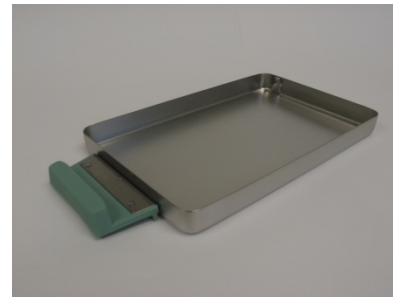

Cassette Tray Only
#01-103475S



Cassette Tray with Rack
#01-100271A



Chassis Kit (Frame)
#01-108388S



Check Valve
#01-100574S



Compressor
#01-100560S



Conductivity Meter
#01-103139S



Control Box
#01-103141S



Copper Tubing
Validation Thermocouple
#01-109128S



Cover Complete
Model 01-121101
#01-106121S




Cover Complete
Model 101102
#01-100564S

Statim 2000 Parts



Cover without Facia
#01-101611S

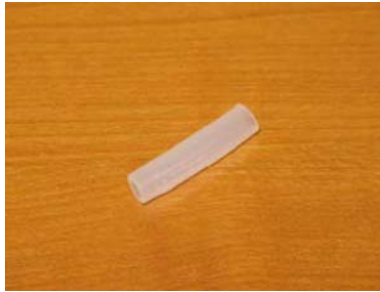

Drain Tube 1"
#74-100335A



Drain Tubing Kit
#01-108152S




Exhaust Tubing
#01-100204S



Facia Complete
Model 01-121101
#01-106122S



Facia Complete
Model 101102
#01-100563S

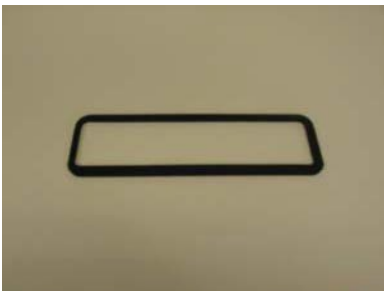

Facia Gasket
#01-100842S



Facia without LCD
Model 01-121101
#01-106124S



Facia without LCD
Model 101102
#01-103518S



Filter Reservoir
#01-109300S



Fitting 3/8 for Biological
Filter Tubes
#01-107817S



Fuse 2A
#01-103498S


Statim 2000 Parts




**Fuse 15A
#01-103472S**




**Fuse .25A
#01-103497S**

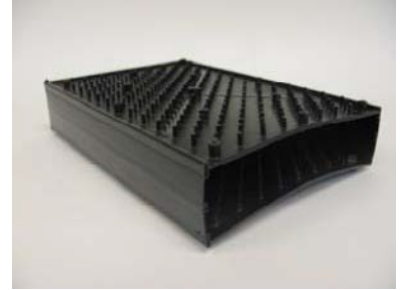

**Isoplate
#01-103856S**

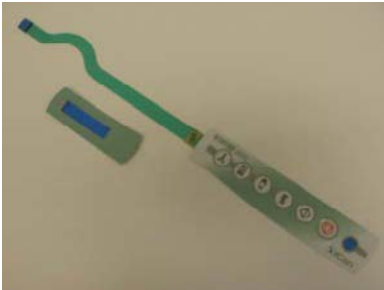


**Keypad Model 01-121101
#01-109680S**

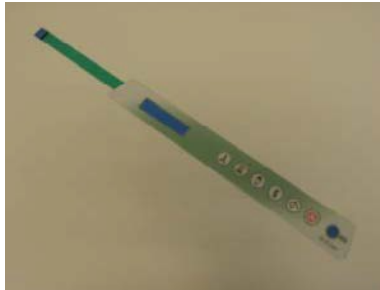

**Keypad Model 101102
#01-101707S**



**Label Kit (Qty 10)
#01-106675S**

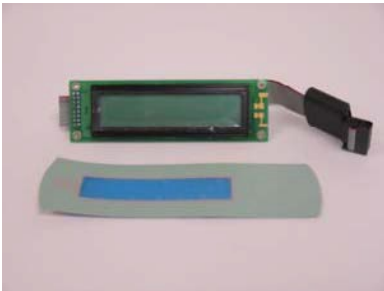

**LCD
#01-104381S**



**LCD Decal
#01-106950S**



**Leveler
#01-101553S**



**Leveler Repair Kit
#01-104180S**



**Line Filter
#01-106087S**




**Microprocessor 4XX
#01-108742S**

Statim 2000 Parts



Microprocessor 5XX
#01-108976S

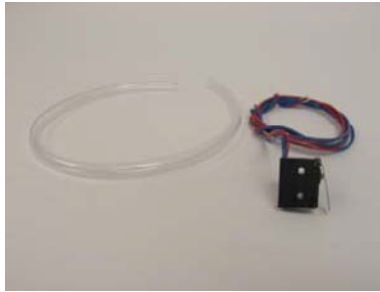


Microswitch
#01-100571S



Operator's Manual
#01-106938S



Packaging with Inserts
#01-108348S

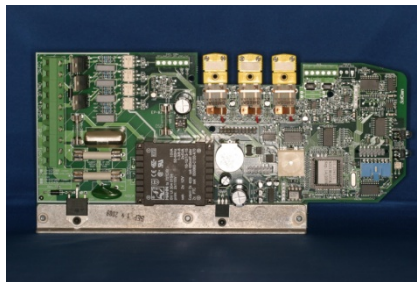


PCB Rev. 7
#01-109365S



PCB Rev. 7 with Printer Cable
#01-109690S



PCB Adapter Board 4XX
#01-108775S

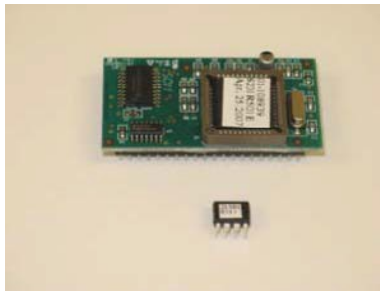

PCB Adapter Board 5XX
#01-108985S



PCB Power Cable
#01-106349S



Plug Drain Tubing
#01-104343S



Power Cable Filter/Switch
#01-108541S



Power Cord
#01-100572S

Statim 2000 Parts




Power Cord 2000
#01-101647S




Power Switch
#01-100573S



Pressure Relief Valve (PRV)
#01-106787S

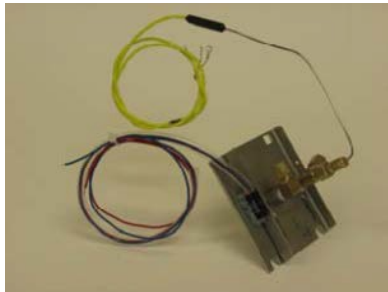

Probe Bracket
#01-100570S

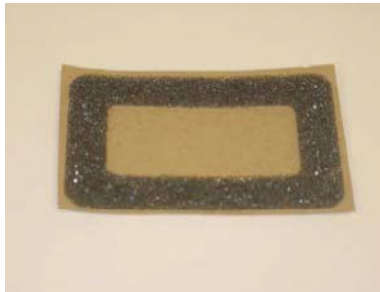

Probe Bracket Gasket
#01-100865S



Pump & Pump Ulka
#01-100568S & #01-110445S



Pump Bracket
#01-108363S



Pump Tester Bottle
#01-100713S



Push In Fitting
#01-100782S



Rack Unwrapped
#01-103945S



Rack Wrapped (Mesh)
#01-106653




Reservoir Cap with Filter
#01-101783S

Statim 2000 Parts



**Reservoir with Float
#01-100565S**

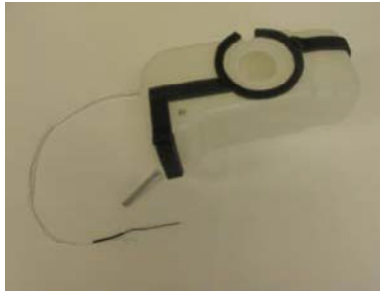

**Reservoir with
Water Quality Sensor
#01-110868S**

**Reservoir with Float & Sensor
#01-106459S**



**Solenoid Coil
#01-104303S**



**Solenoid Plunger Wrench
#01-103471S**




**Solenoid Valve Complete
#01-100557S**



**Solenoid Valve Repair Kit
#01-100998S**



**Strain Relief
#53-100219A**

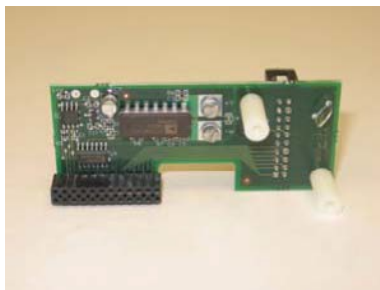

**Temperature Adapter
Board
#01-108981S**



**Test Strip Indicators
#SCI134**



**Thermal Fuse
#01-104461S**



**Validation Thermocouple
#01-108983S**



**Waste Bottle Complete
#01-100812S**

Statim 2000 Parts



**Waste Bottle Fitting
#01-100735S**



**Waste Bottle Only
#01-100724S**



**Water Quality Sensor
#01-103571S**



**Water Reservoir 90°
Float Switch
01-110891S**

Statim 2000 G4 Front View




Item #	Part #	Description
1	01-112415S	Cover w/o Facia
2	01-112401S	Reservoir Cap w/Filter
2	01-109300S	Filter, Water Reservoir
3	01-112414S	Cover Complete
Not Shown	01-112867S	Mat Silicon
4	01-112399S	LCD Module G4
5	01-100842S	Facia Gasket (located inside Facia)
6	01-112398S	Cable USB
Not Shown	USB-STICK2	USB Stick (Flash Drive)
7	01-103856S	Isoplate (not field replaceable requires alignment jig)
8	01-112406S	Cassette Complete
Not Shown	01-112409S	Cassette Lid
Not Shown	01-112408S	Cassette Tray Only
9	01-112410S	Cassette Lid Handles
10	01-112418S	Decal

Statim 2000 G4 Rear View



Item #	Part #	Description
1	01-102119S	Biological Filter
2	01-104284S	Biological Filter Bracket
3	01-100560S	Air Compressor
3	01-100207S	Air Filter (located under round plate)
4	01-107817S	Fitting $\frac{3}{8}$ " Tube Chassis Kit
5	01-100782S	Push-In-Fitting
6	01-110221S	RS232 Port Kit 2000 G4
7	01-106087S	Line Filter
8	01-100573S	Power Switch
9	01-112403S	Ethernet Cable
Not Shown	01-100204S	Exhaust Tubing
Not Shown	01-101647S	Power Cord
Not Shown	01-100780S	Bumper (Left rear stationary leg)
Not Shown	01-101553S	Leveler (Adjustable legs)
Not Shown	01-104180S	Leveler Repair Kit (Adjustable leg including frame insert)

Statim 2000 G4 Top View No Cover

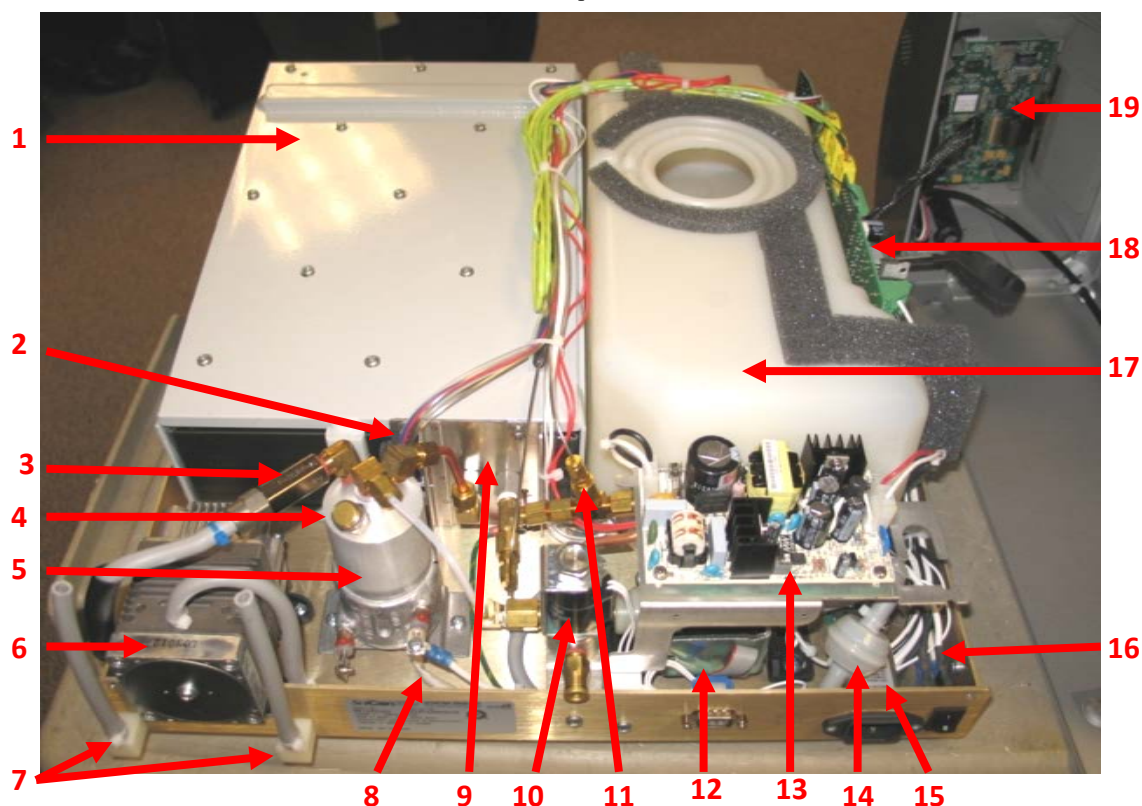

Item #	Part #	Description
1	01-100561S	Armature (not field replaceable requires alignment jig)
2	01-100571S	Microswitch (located behind Boiler)
3	01-100574S	Check Valve
4	01-106787S	Pressure Relief Valve
5	01-108979S	Boiler Alex
6	01-100560S	Compressor
7	01-107817S	Fittings $\frac{3}{8}$ " Tube Chassis Kit (for Biological Filter)
8	01-104461S	Thermal Fuse
9	01-100570S	Probe Bracket (not field replaceable requires complete calibration)
10	01-100557S	Solenoid
10	01-104303S	Solenoid Coil
10	01-100998S	Solenoid Plunger Repair Kit
11	01-108983S	Validation Thermocouple
12	01-110445S	Pump, Ulka
13	01-112405S	DC Power Supply
14	01-106637S	Water Filter, In Line
15	01-106087S	Line Filter
16	01-106349S	PCB Power Cable (from Power Switch to PCB)
17	01-106459S	Reservoir w/Sensor & Float
18	01-112988S	PCB Seven Statim 2000 USA (not field replaceable requires complete calibration)
19	01-112400S	Logic PCB Statim G4
Not Shown	01-108541S	Power Cable Filter/Switch
Not Shown	01-110891S	Float Switch 90 Degree
Not Shown	01-103571S	Water Quality Sensor


Statim 2000 G4 View Inside Cover



Item #	Part #	Description
1	01-100842S	Facia Gasket
2	01-112399S	LCD Module Statim G4
3	01-112404S	Cable Signal Statim G4
4	01-112776S	Cable Power Supply Output 2000 G4
5	01-112403S	Ethernet Cable Statim G4

Statim 5000


Front View

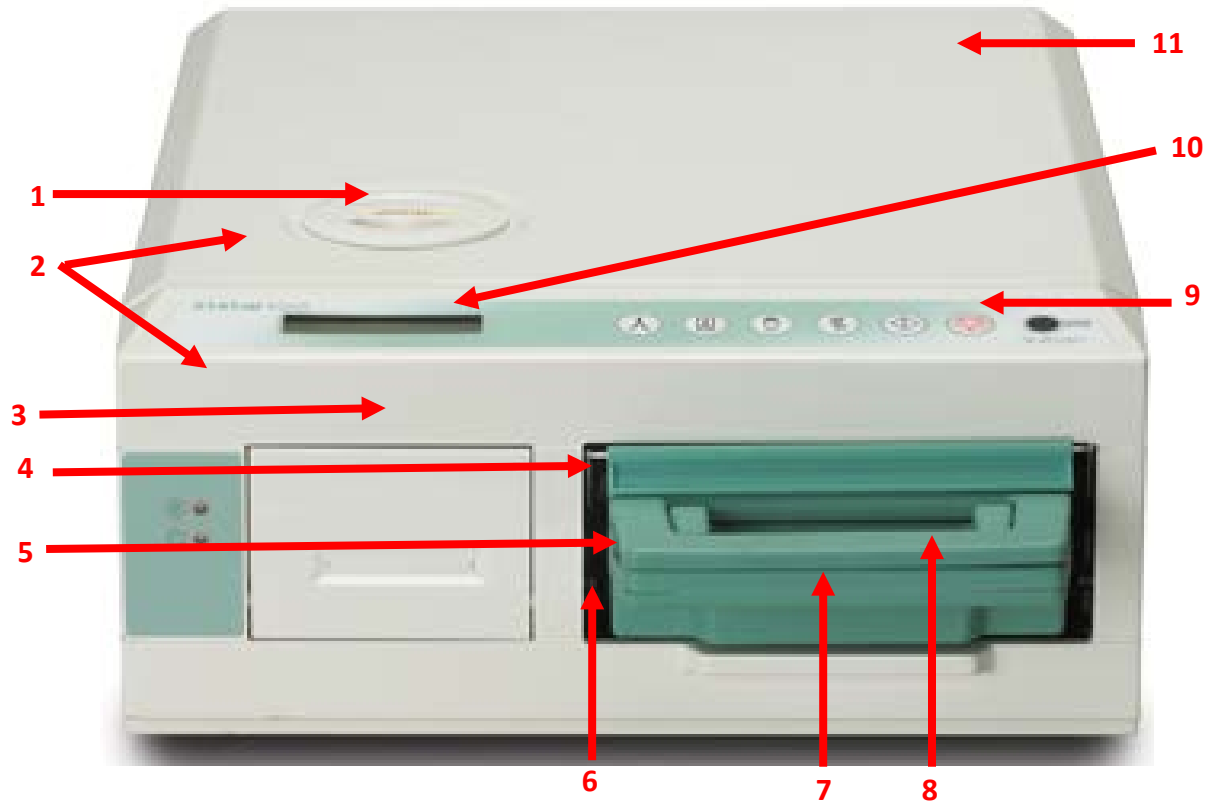

Item #	Part #	Description
1	01-101783S	Reservoir Cap & Filter
1	01-109300S	Filter Only, Reservoir (under Reservoir Cap)
2	01-104300S	Cover Complete
3	01-104299S	Facia Complete
4	01-101650S	Facia Gasket (located inside Facia)
5	01-101613S	Cassette Complete
6	01-103857S	Isoplate (not field replaceable requires alignment jig)
7	01-101658S	Cassette Handles Lid & Tray
8	01-106070S	Cassette Handle Lid Only
9	01-109671S	Keypad
10	01-104382S	LCD (located inside Facia)
11	01-101625S	Cover without Facia
Not Shown	01-112868S	Silicone Mat 5000

Statim 5000


Rear View

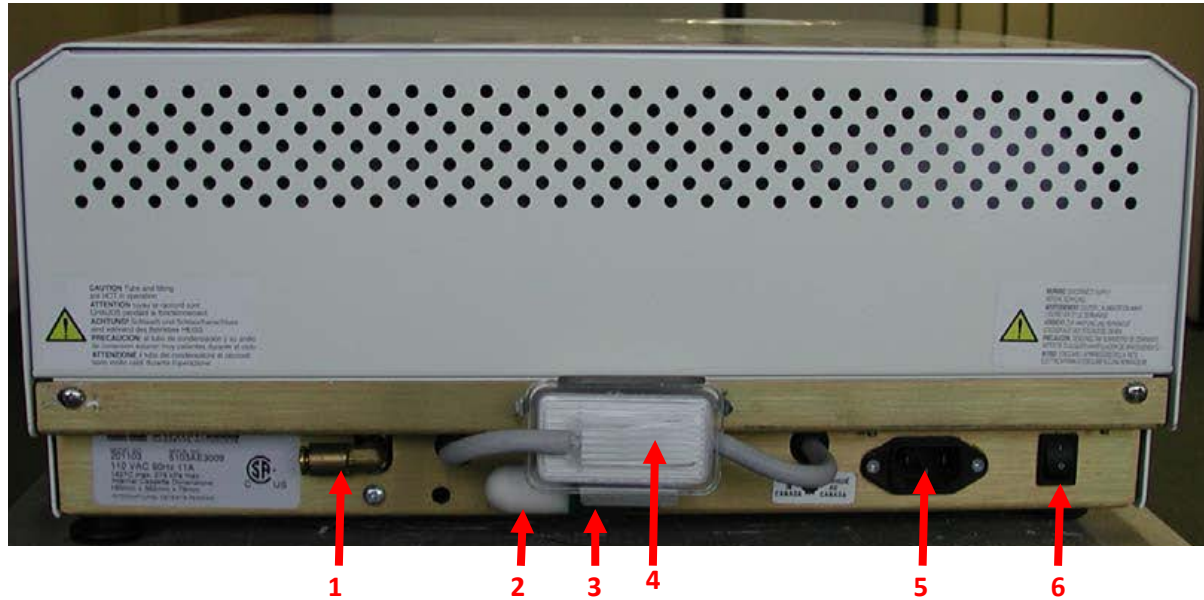

Item #	Part #	Description
1	01-101755S	Push-In-Fitting
2	01-101652S	Air Compressor Filter (not on all units)
3	01-104284S	Biological Filter Bracket
4	01-102119S	Biological Filter
5	01-103993S	Power Socket (older units)
5	01-106087S	Line Filter (newer units)
6	01-100573S	Power Switch

Statim 5000


Top View No Cover

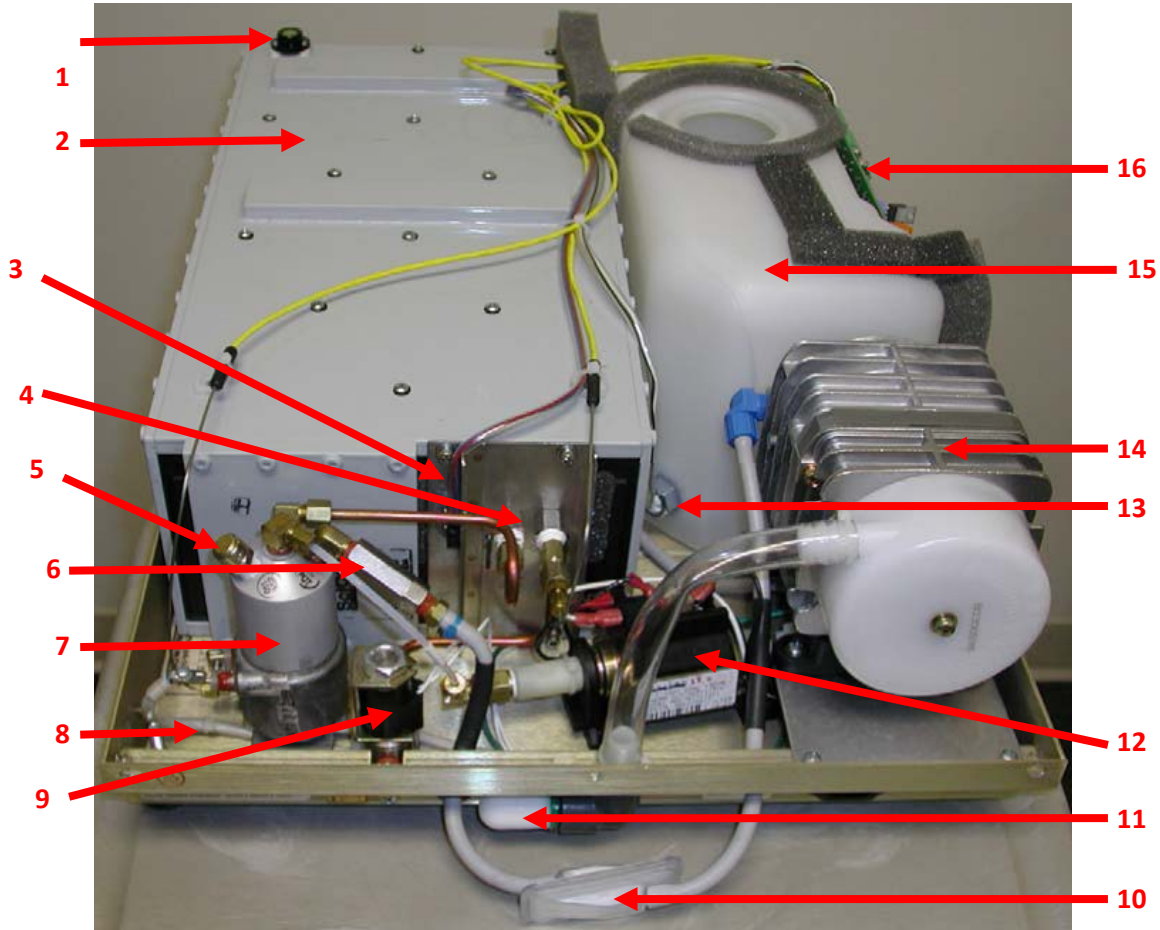

Item #	Part #	Description
1	01-100684S	Bubble Level
2	01-101623S	Armature (not field replaceable requires alignment jig)
3	01-101621S	Microswitch
4	01-101618S	Probe Bracket (not field replaceable requires complete calibration)
5	01-106787S	Pressure Relief Valve
6	01-101627S	Check Valve
7	01-108980S	Boiler (software must be 5XX or higher to replace in the field)
8	01-104462S	Thermal Fuse (under Boiler)
9	01-101628S	Solenoid
10	01-102119S	Biological Filter
11	01-101652S	Air Compressor Filter (not on all units)
12	01-101761S	Pump
12	01-110446S	Pump Ulka (Revision 6XX or higher software)
13	01-103571S	Water Quality Sensor
14	01-112423S	Compressor Medo
14	01-112665S	Compressor Kit, Thomas to Medo
15	01-110869S	Reservoir with Water Quality Sensor
15	01-101622S	Reservoir with Float Switch & Water Quality Sensor
16	01-109395S	PCB Rev. 7 no Printer (not field replaceable requires complete calibration)
16	01-109823S	PCB Rev. 7 with Printer (not field replaceable requires complete calibration)

Statim 5000 Revision 5XX & Higher Software

Alex Parts

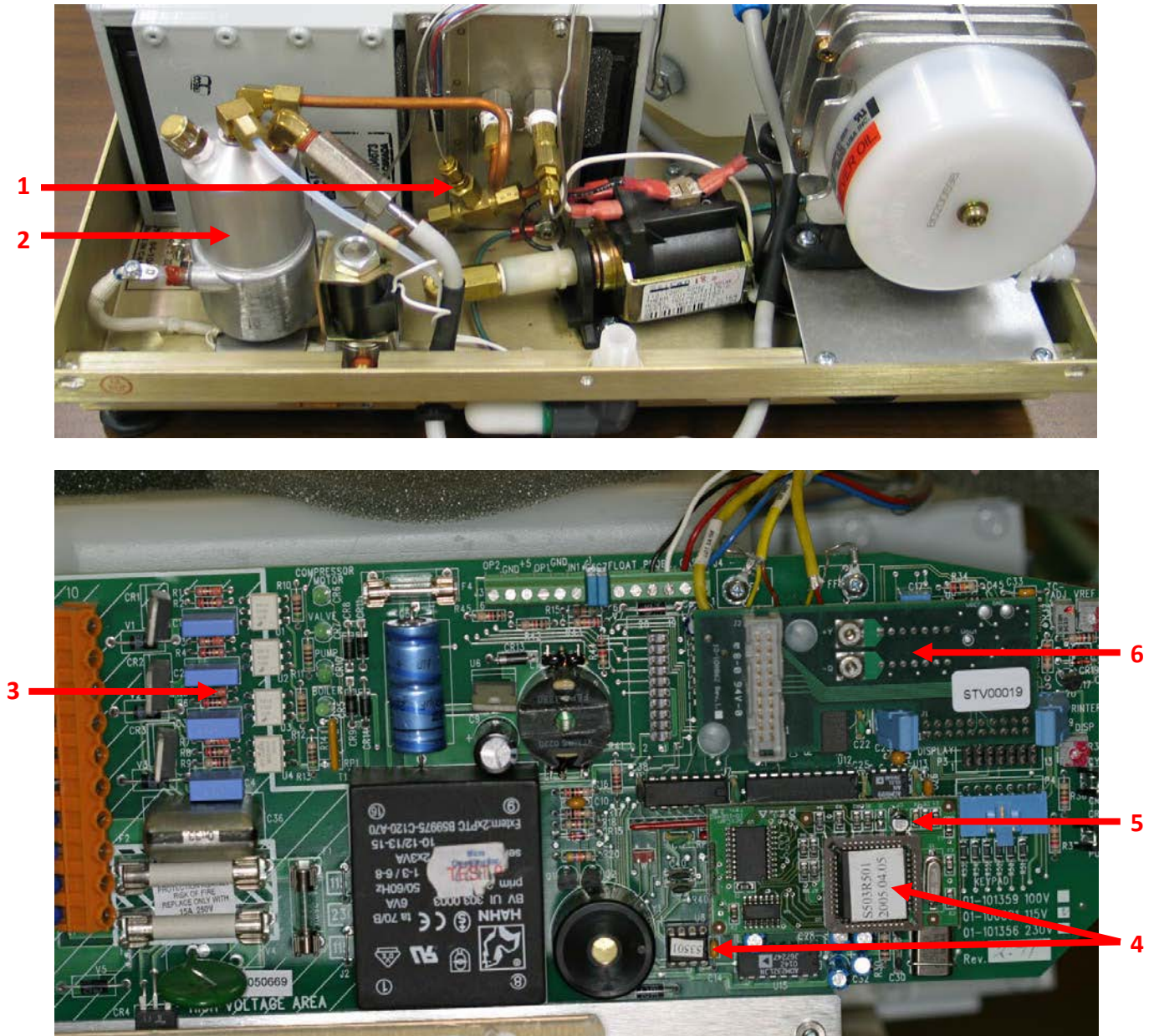

Item #	Part #	Description
1	01-108984S	Validation Thermocouple Statim 2000
2	01-108980S	Boiler (software must be revision 5XX or higher to replace in field)
3	01-109395S	PCB Rev. 7 no Printer (not field replaceable requires complete calibration)
3	01-109823S	PCB Rev. 7 with Printer (not field replaceable requires complete calibration)
4	01-108976S	Microprocessor Kit Revision 5XX Software
4	01-109391S	Microprocessor Kit Revision 6XX Software
5	01-108987S	PCB Adapter Board 5XX Software Statim 5000
6	01-108981S	Temperature Adapter Board for Alex with Validation Thermocouple

Statim 5000 Parts



Air Compressor Filter
#01-101652S



**Air Compressor
Filter Holder**
#01-104329S



Alex Kit 2XX
#01-108998S



Alex Kit 4XX
#01-108995S



Armature
#01-101623S



Boiler Alex
#01-108980S



Bubble Level
#01-100684S




Bumper
#01-100780S



Cassette Complete
#01-101613S



Cassette Extended
#01-104104

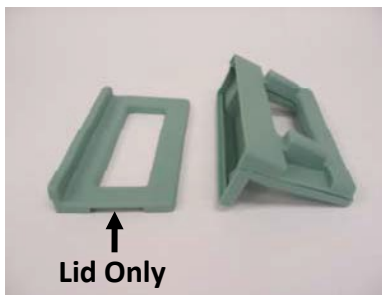

Cassette Handles Lid & Tray
#01-101658S
Cassette Handle Lid Only
#01-106070S



Cassette Handles Extended
#01-104472S


Statim 5000 Parts



Cassette Lid
#01-101757S



Cassette Seal & Lube Kit
#01-101649S

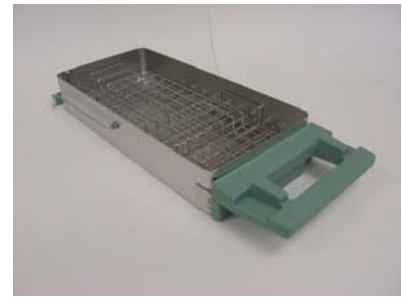

Cassette Tray with Rack
#01-101614S



Chassis Kit (Frame)
#01-108389S



Check Valve
#01-101627S



Compressor
#01-101619S



Conductivity Meter
#01-103139S



Control Box
#01-103141S



Copper Tubing
Validation Thermocouple
#01-109129S



Cover Complete
#01-104300S




Cover without Facia
#01-101625S



Drain Tube 1"
74-100335A

Statim 5000 Parts

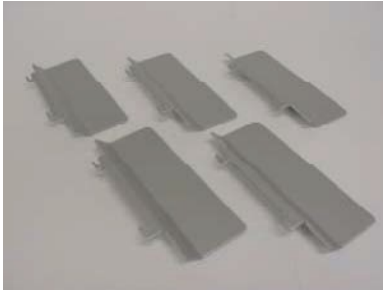

Drying Plates
#01-103935



Exhaust Tubing
#01-100204S



Facia Complete
#01-104299S



Facia Gasket
#01-101650S



Filter, Reservoir
#01-109300S



Fuse 2A
#01-103498S



Fuse 15A
#01-103472S




Fuse .25A
#01-103497S



Isoplate
#01-103857S



Keypad
#01-109671S

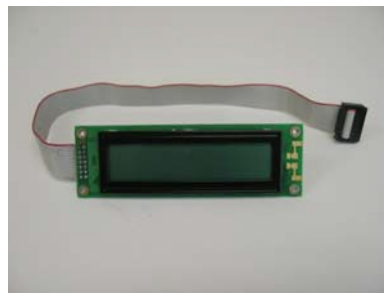

LCD
#01-104382S



Leveler
#01-101553S


Statim 5000 Parts



Leveler Repair Kit
#01-104180S



Line Filter
#01-106087S

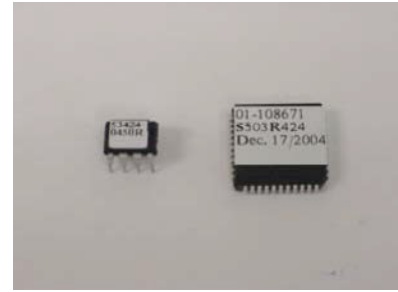

Microprocessor 4XX
#01-108746S



Microprocessor 5XX
#01-108978S



Microswitch
#01-101621S



Operator's Manual
#01-106938S



Packaging with Inserts
#01-108327S



PCB 4XX
#01-108744S



PCB 5XX
#01-109045S



PCB Adapter Board 4XX
#01-108777S



PCB Adapter Board 5XX
#01-108987S




PCB Power Cable
#01-106350S

Statim 5000 Parts



**Plug Drain Tubing
#01-104343S**

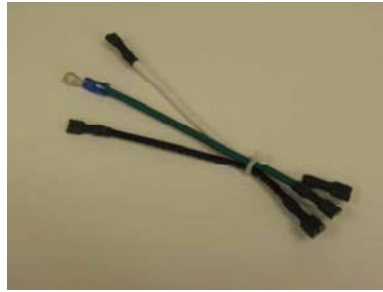

**Power Cable Filter/Switch
#01-108541S**



**Power Cord
#01-101647S**



**Power Socket
#01-103993S**



**Power Switch
#01-100573S**




**Pressure Relief Valve
#01-106787S**



**Printer 5XX> Software
#01-210000**



**Printer Case with Damper
#01-103470S**

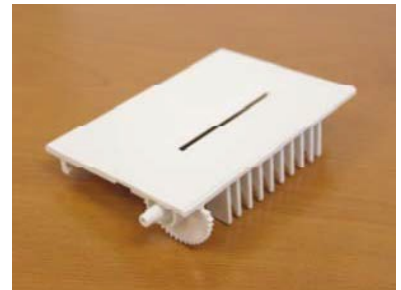

**Printer Door
#01-102159S**



**Printer Paper (Pkg. 10)
#01-101657S**



**Printer Paper Holder
#01-101973S**




**Printer Spring/Latch
#01-101971S**


Statim 5000 Parts



**Probe Bracket
#01-101618S**

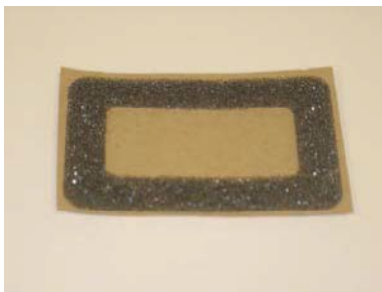

**Probe Bracket Gasket
#01-100865S**

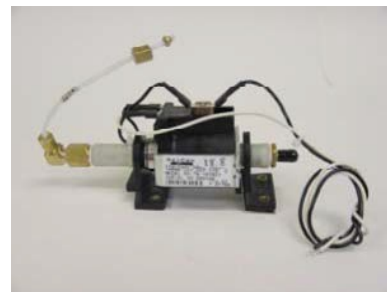

**Pump
#01-101761S
Pump Ulka
(6XX or Higher Software)
#01-110446S**



**Pump Recovery Kit
#01-104313S**




**Pump Tester Bottle
#01-100713S**



**Push In Fitting
#01-101755S**



**Rack Unwrapped Mesh
#01-101709S**

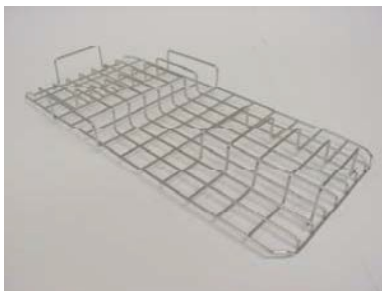

**Rack Wrapped
#01-102054S**



**Reservoir w/Sensor
#01-110869S
Reservoir w/Sensor & Float
#01-101622S**



**Reservoir Cap with Filter
#01-101783S**



**Solenoid Coil
#01-104303S**



**Solenoid Plunger Wrench
01-103471S**


Statim 5000 Parts



Solenoid Valve Complete
01-101628S



Solenoid Valve Repair Kit
01-100998S

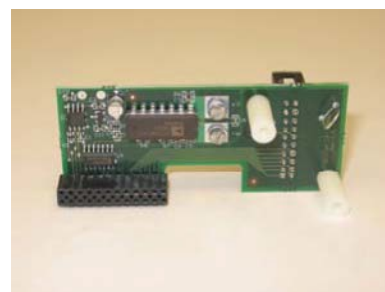

Temperature Adapter Board
01-108981S



Test Strip Indicators
SCI134



Thermal Fuse
01-104462S



Validation Thermocouple
#01-108984S



Waste Bottle Complete
01-100812S



Waste Bottle Fitting
01-100735S



Waste Bottle Only
01-100724S



Water Quality Sensor
01-103571S



Water Reservoir 90° Float Switch
01-110891S

Statim 5000 G4 Front View




Item #	Part #	Description
1	01-112393S	Cover Complete
1	01-112394S	Cover w/o Facia (rear metal part of cover only)
Not Shown	01-112868S	Mat Silicon 5000
2	01-112401S	Reservoir Cap
Not Shown	01-109300S	Filter Water Reservoir
3	01-112398S	Cable, USB
Not Shown	USB-STICK2	USB Stick
4	01-112384S	Cassette Complete (see page 6)
5	01-112397S	Decal

Statim 5000 G4 Rear View




Item #	Part #	Description
1	01-112403S	Ethernet Cable
2	01-101755S	Push-In Fitting
3	01-102119S	Biological Filter
4	01-104284S	Biological Filter Bracket
5	01-101652S	Air Compressor Filter
6	01-104329S	Air Compressor Filter Holder
7	01-110222S	RS232 Port Kit 5000 G4
8	01-106087S	Line Filter
9	01-100573S	Power Switch
Not Shown	01-100204S	Exhaust Tubing
Not Shown	01-101647S	Power Cord
Not Shown	01-100780S	Bumper (Left rear stationary leg)
Not Shown	01-101553S	Leveler (Adjustable legs)
Not Shown	01-104180S	Leveler Repair Kit (Adjustable leg including frame insert)

Statim 5000 G4 Rear No Cover

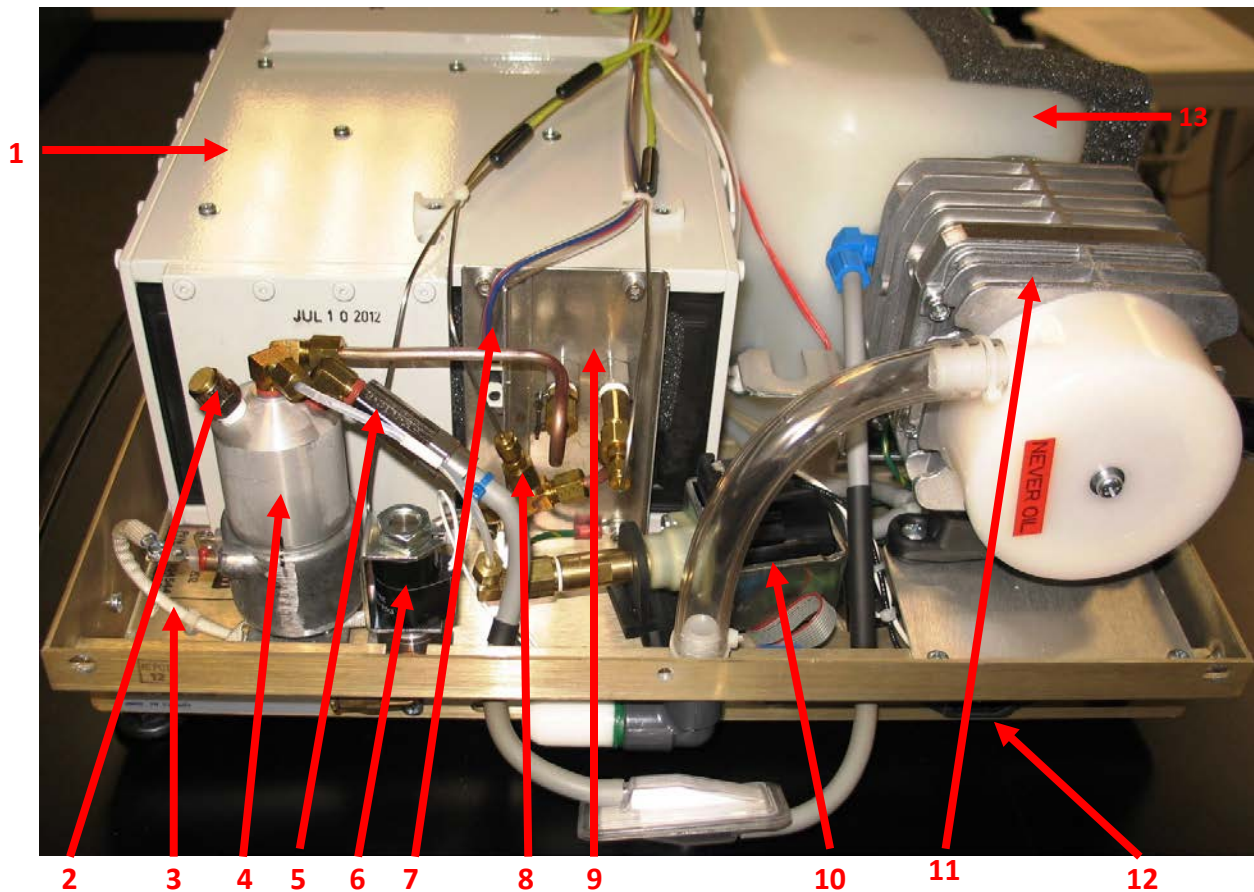


Item #	Part #	Description
1	01-101623S	Armature (not field replaceable requires alignment jig)
2	01-106787S	Pressure Relief Valve
3	01-104462S	Thermal Fuse 5000
4	01-108980S	Boiler Alex
5	01-101627S	Check Valve
6	01-101628S	Solenoid Complete
6	01-104303S	Solenoid Coil
6	01-100998S	Solenoid Plunger Repair Kit
7	01-101621S	Microswitch
8	01-108984S	Validation Thermocouple
9	01-101618S	Probe Bracket (not field replaceable requires complete calibration)
10	01-110446S	Pump, Ulka
11	01-112423S	Compressor, Medo
12	01-106087S	Line Filter
Not Shown	01-108541S	Power Cable Filter/Switch
13	01-101622S	Reservoir w/Water Quality Sensor & Float
Not Shown	01-110891S	Float Switch 90 Degree
Not Shown	01-103571S	Water Quality Sensor

Statim 5000 G4 Front Left No Cover

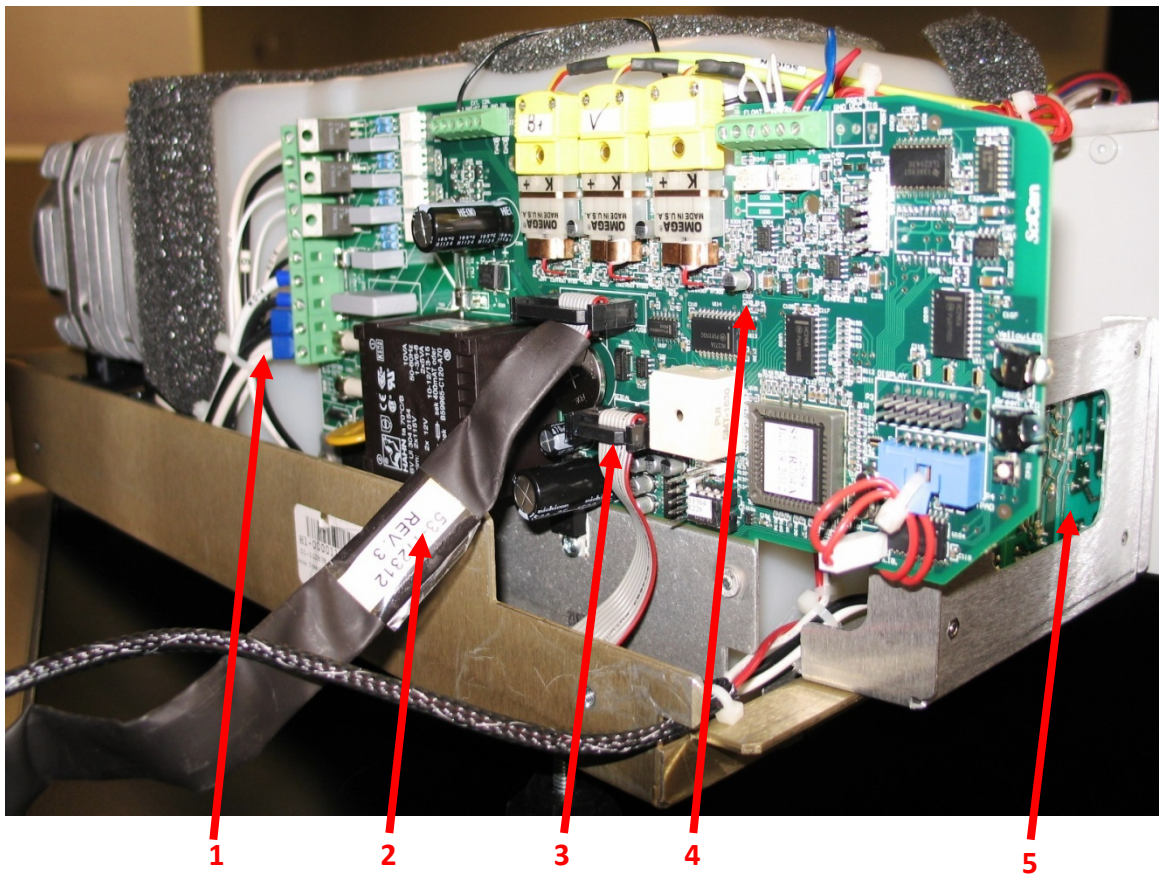


Item #	Part #	Description
1	01-106350S	PCB Power Cable
2	01-112404S	Cable, Signal w/Ferrite
3	01-110222S	RS232 Port Kit 5000 G4
4	01-113389S	PCB Rev. 7 for 5000 G4 (not field replaceable requires complete calibration)
5	01-112405S	DC Power Supply

Statim 5000 G4 View Inside Cover

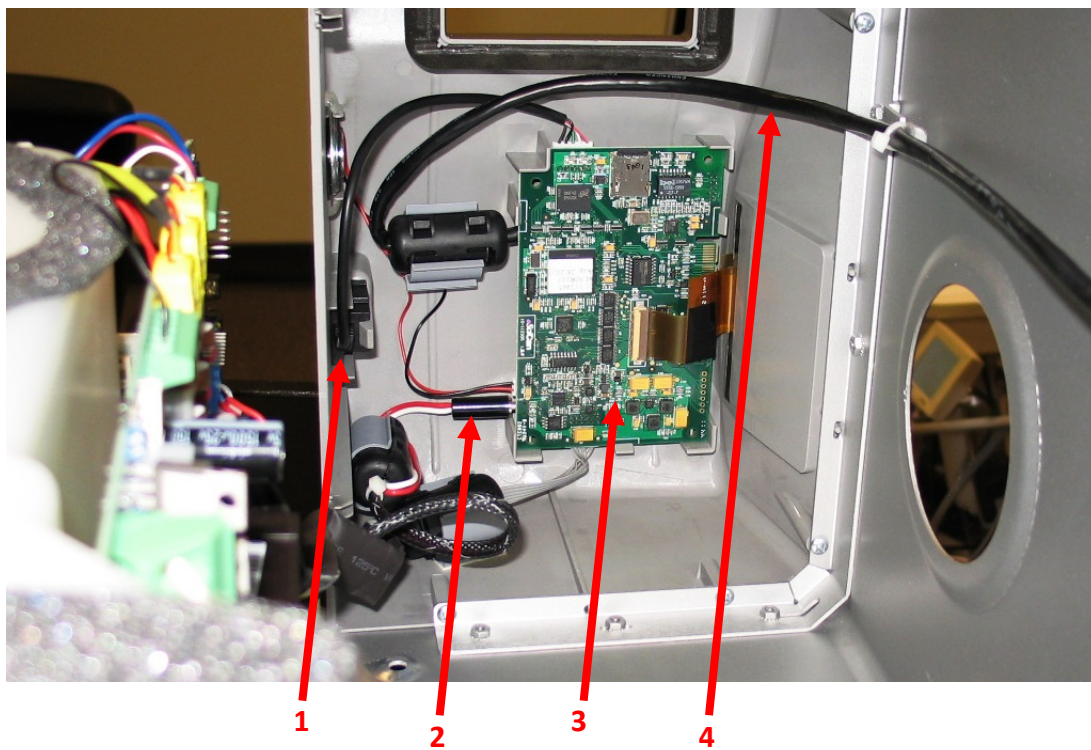

Item #	Part #	Description
1	01-112398S	Cable, USB
2	01-112790S	Cable Power Supply Output 5000 G4
3	01-112400S	Logic PCB Statim G4
4	01-112403S	Ethernet Cable

Statim 5000 G4 Cassette

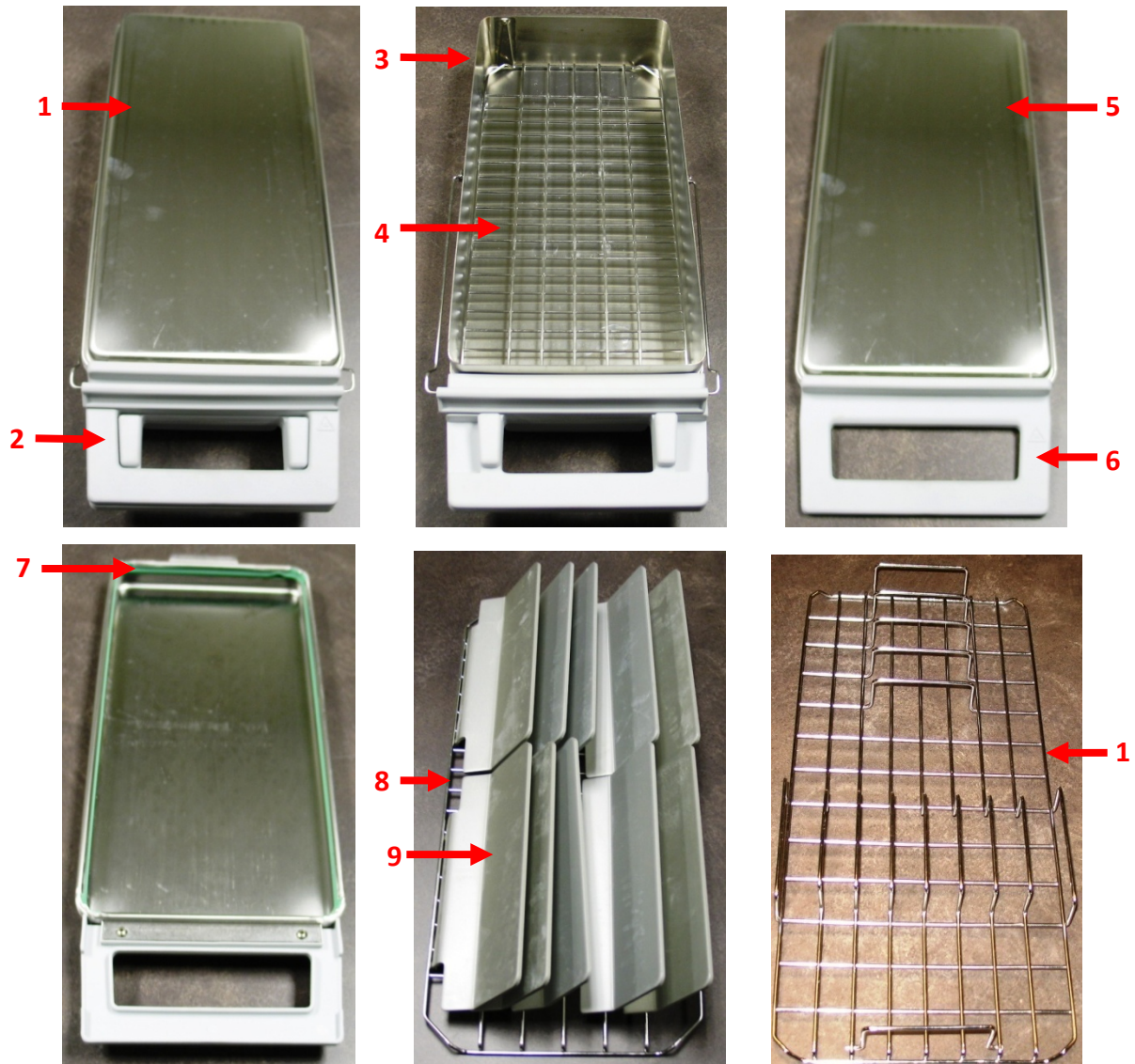

Item #	Part #	Description
1	01-112384S	Cassette Complete (with racks)
1	01-113543S	Cassette Lid & Tray (no racks)
2	01-112387S	Cassette Handles Lid & Tray
3	01-112385S	Cassette Tray w/Mesh Rack
3	01-113544S	Cassette Tray (no rack)
4	01-101709S	Rack Unwrapped Mesh
5	01-112386S	Cassette Lid
6	01-112388S	Cassette Handle Lid Only
7	01-101649S	Cassette Seal & Lube Kit
8	01-113482S	Rack w/Drying Plates
9	01-103935	Drying Plates (Pkg of 5)
10	01-102054S	Rack Wrapped



Statim 7000

Front View



Item #	Part #	Description
1	01-110298S	Cover Complete
2	01-110328S	Reservoir Lid
3	01-110521S	Cover Logo SciCan
4	01-110293S	Reservoir Complete
5	01-110522S	Cover Facia Front Left
6	01-110523S	Cover Facia Front Cassette
7	01-110520S	Cover Reservoir Compartment
8	01-110512S	Leveling Foot
9	01-110288S	Cassette Complete
10	01-110524S	Cover Facia Gasket (located inside cover)
11	01-110526S	Cover Keypad Complete
12	01-110525S	Cover LCD Only





Statim 7000 Front Accessories & Rear View

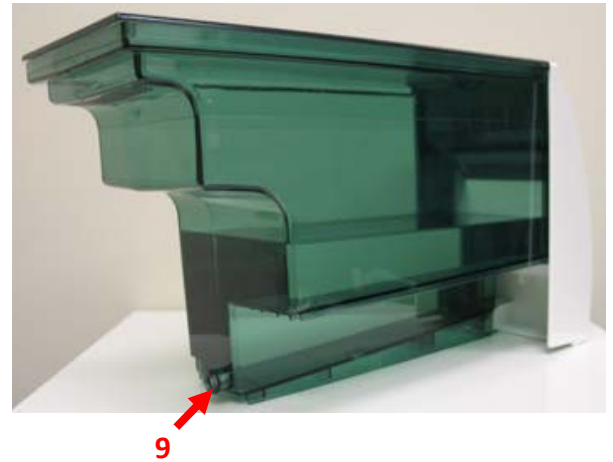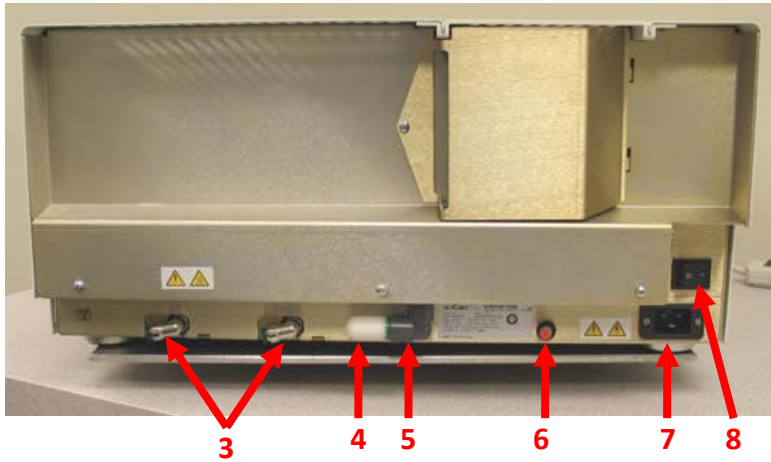

Item #	Part #	Description
1	01-112927	Reservoir Filter Base (must use distilled water)
2	01-102119S	Biological Filter
3	01-110503S	Push-In-Fitting
4	01-101652S	Air Compressor Filter
5	01-104329S	Air Compressor Filter Holder
6	01-104343S	Plug Drain Tubing
7	01-110505S	Line Filter (plug receptable)
8	01-110504S	Power Switch
9	01-110327S	Reservoir Plunger Kit w/Large Outer O-ring

Statim 7000



Front & Side View No Cover

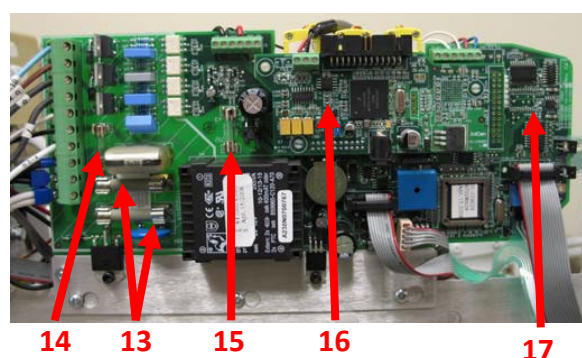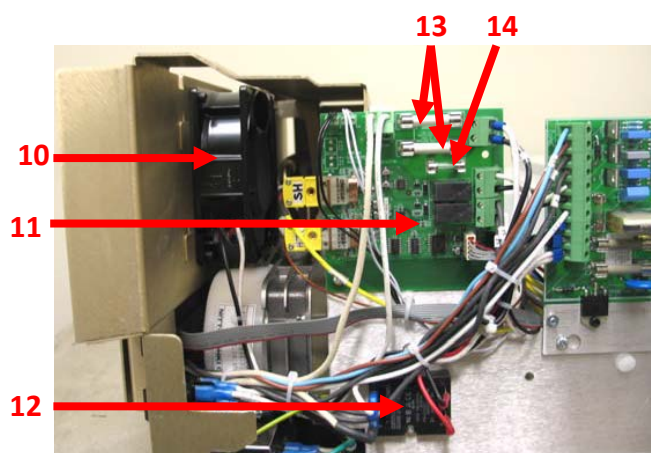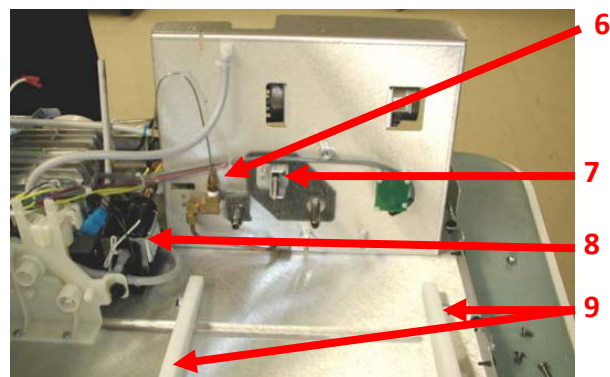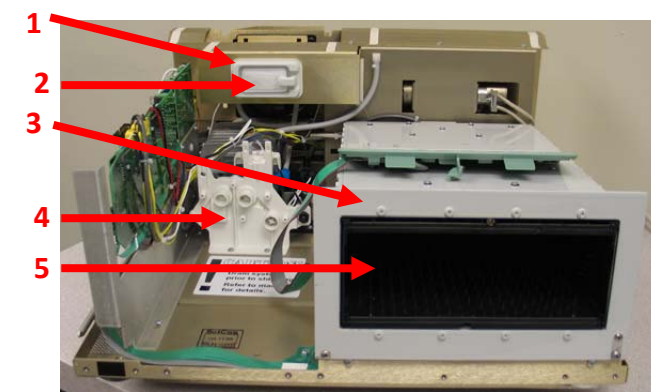

Item #	Part #	Description
1	01-110472S	Biological Filter Holder
2	01-102119S	Biological Filter
3	01-110496S	Armature
4	01-110493S	Reservoir Inlet Assembly
5	01-110498S	Isoplate
6	01-110515S	Validation Thermocouple
7	01-110502S	Microswitch
8	01-110471S	Water Pump
9	01-110497S	Guide Rails (Pkg of 2)
10	01-110490S	Fan
11	01-110482S	PCB Power Board Auxiliary Heater
12	01-110477S	Auxiliary Heater Triac (Solid State Relay)
13	01-110716S	Fuse 10A 250V (Pkg of 2)
14	01-103498S	Fuse 2A
15	01-109816S	Fuse 0.8A
16	01-110483S	PCB RFID Board
17	01-110480S	PCB 7000 (not field replaceable requires complete calibration)





Statim 7000

Rear View No Cover

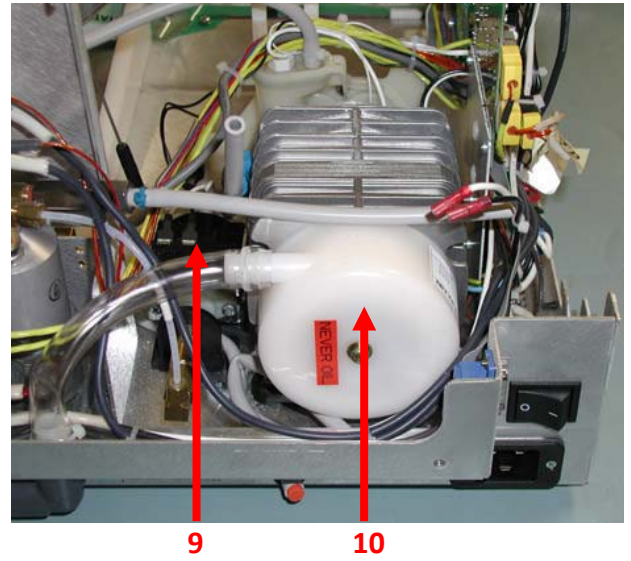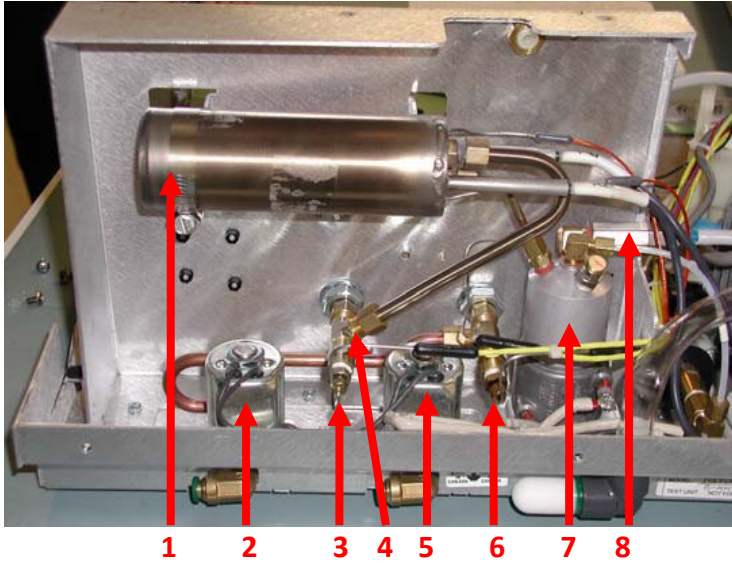

Item #	Part #	Description
1	01-110474S	Auxiliary Heater
2	01-110527S	Solenoid Chamber (Valve 2)
3	01-110513S	Thermocouple Inlet/Outlet
4	01-110499S	Armature Probe Inlet Fitting
5	01-110528S	Solenoid Validation (Valve 1)
6	01-110514S	Thermocouple Outlet (not field replaceable requires complete calibration)
7	01-110472S	Boiler
8	01-110510S	Check Valve
9	01-110471S	Pump
10	01-110478S	Compressor

Statim 7000 Cassette

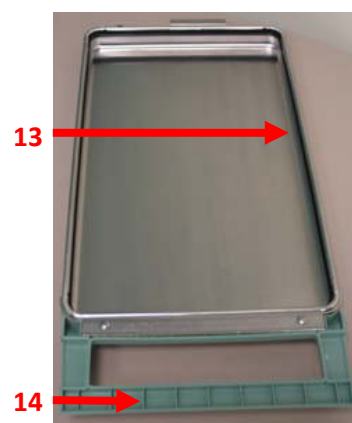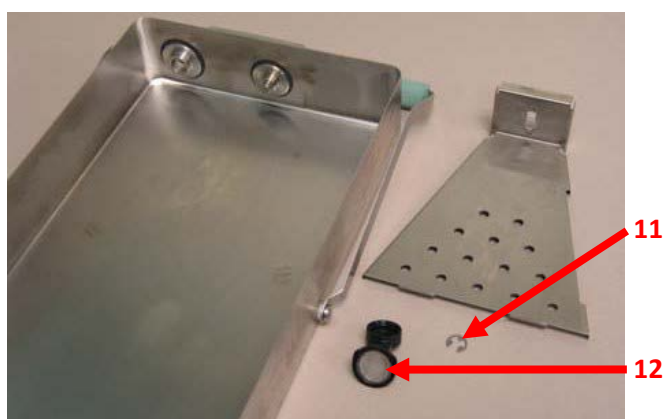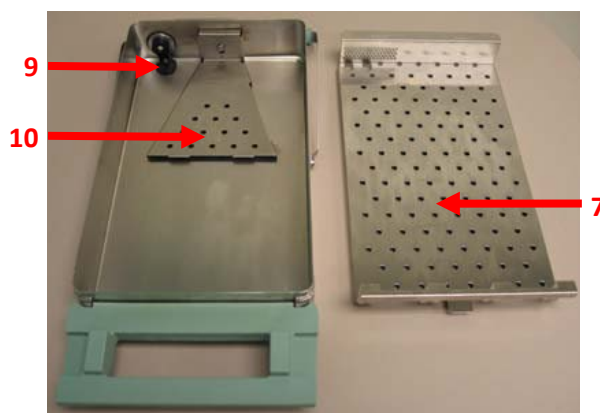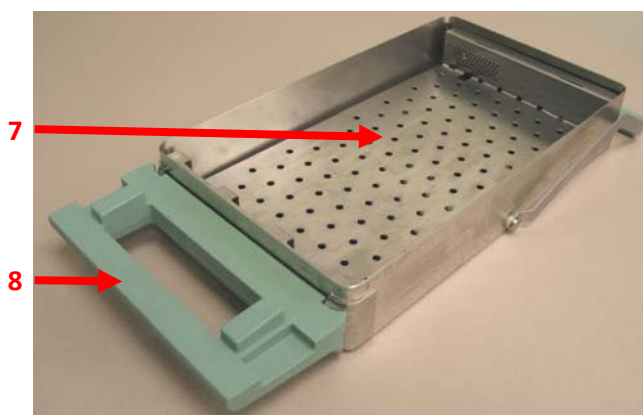

Item #	Part #	Description
1	01-110288S	Cassette Complete
2	01-110290S	Cassette Lid
3	01-110289S	Cassette Tray
4	01-110291S	Cassette Inlet Coupling
5	01-110296S	Cassette Coupling Seal Kit (Seals & O-rings for both Couplings included)
6	01-110292S	Cassette Outlet Coupling
7	01-110294S	Cassette Perforated Rack
8	01-110330S	Cassette Tray Handle
9	01-110297S	Cassette Tray Exhaust Duct
10	01-110825S	Cassette Steam Deflector
11	01-110824S	Cassette Steam Deflector Clip
12	01-106848S	Filter Exhaust Duct Insert
13	01-110295S	Cassette Seal
14	01-110329S	Cassette Lid Handle
Not Shown	01-111590S	Cassette O-ring Lubricator Kit

Statim 7000


Waste, Condenser & Overflow Bottles



Item #	Part #	Description
1	01-112922S	Stem 1/4" X 5/16"
2	01-100735S	Waste Bottle Fitting
3	01-110506S	Bottle Waste Complete
4	01-111660S	Condenser Bottle Fitting 3/8"
5	01-111395S	Bottle Condenser
6	01-110300S	Bottle Waste Overflow Kit
7	01-111497S	Bottle Tubing Condenser to Waste (with Quick Disconnect
8	01-110508S	Waste to Condenser Fitting Quick Disconnect
Not Shown	01-110509S	Bottle Exhaust Tubing Kit(from Statim to Condenser Bottle

Statim 900

Front View

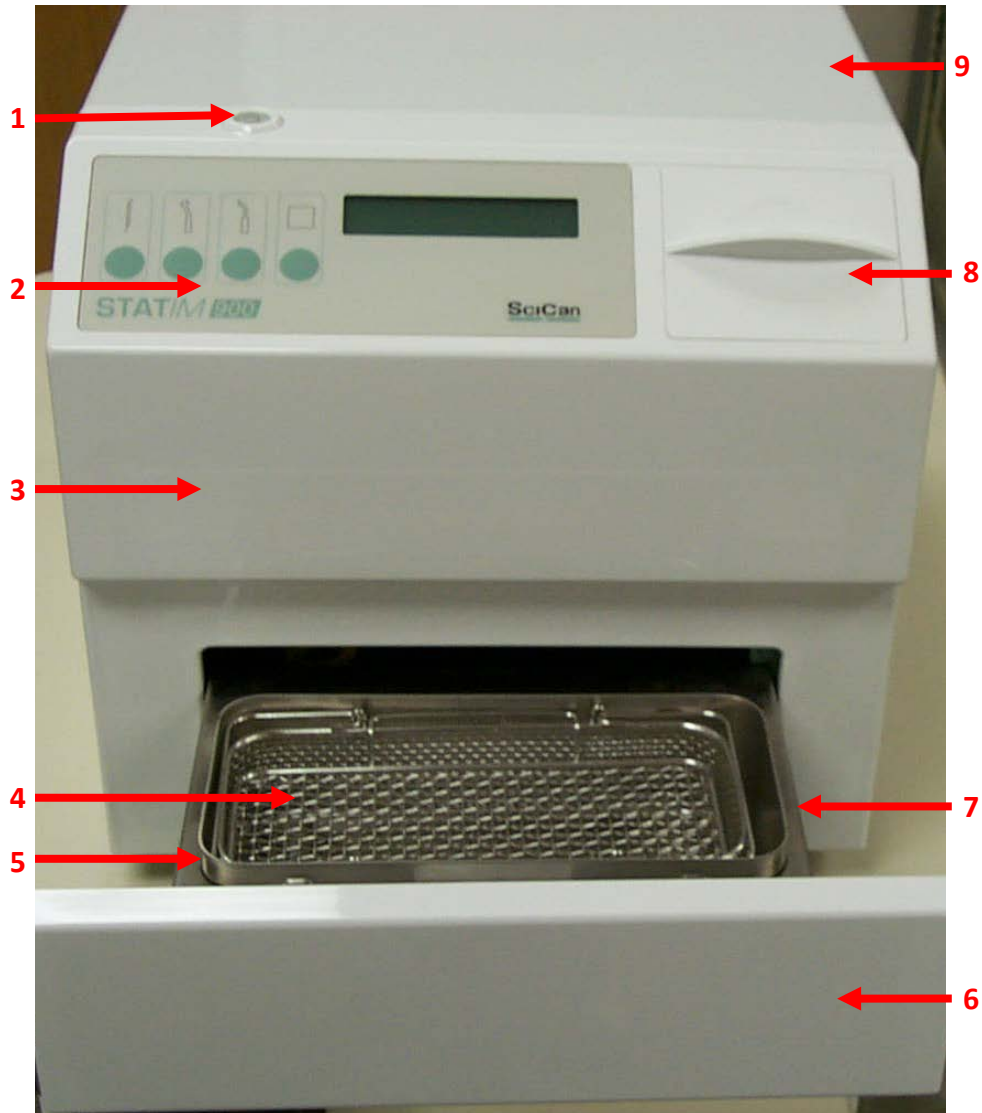

Item #	Part #	Description
1	01-101554S	Bubble Level
2	01-103527S	Keypad
3	01-103508S	Facia
4	01-103636S	Wire Basket w/Insert
5	01-103608S	Sterilization Chamber
6	01-101560S	Drawer Front
7	01-101562S	Drawer
8	01-103559S	Reservoir Cover
9	01-101558S	Cover
Not Shown	01-101543S	Seal Kit (Exhaust & Pressure)
Not Shown	01-106898S	Seal Assembly (complete with plate)

Statim 900


Rear View



Item #	Part #	Description
1	01-104343S	Plug Drain Tubing
2	01-100573S	Power Switch
3	01-106087S	Line Filter
4	01-100782S	Push-In-Fitting


Statim 900

Left Side No Cover

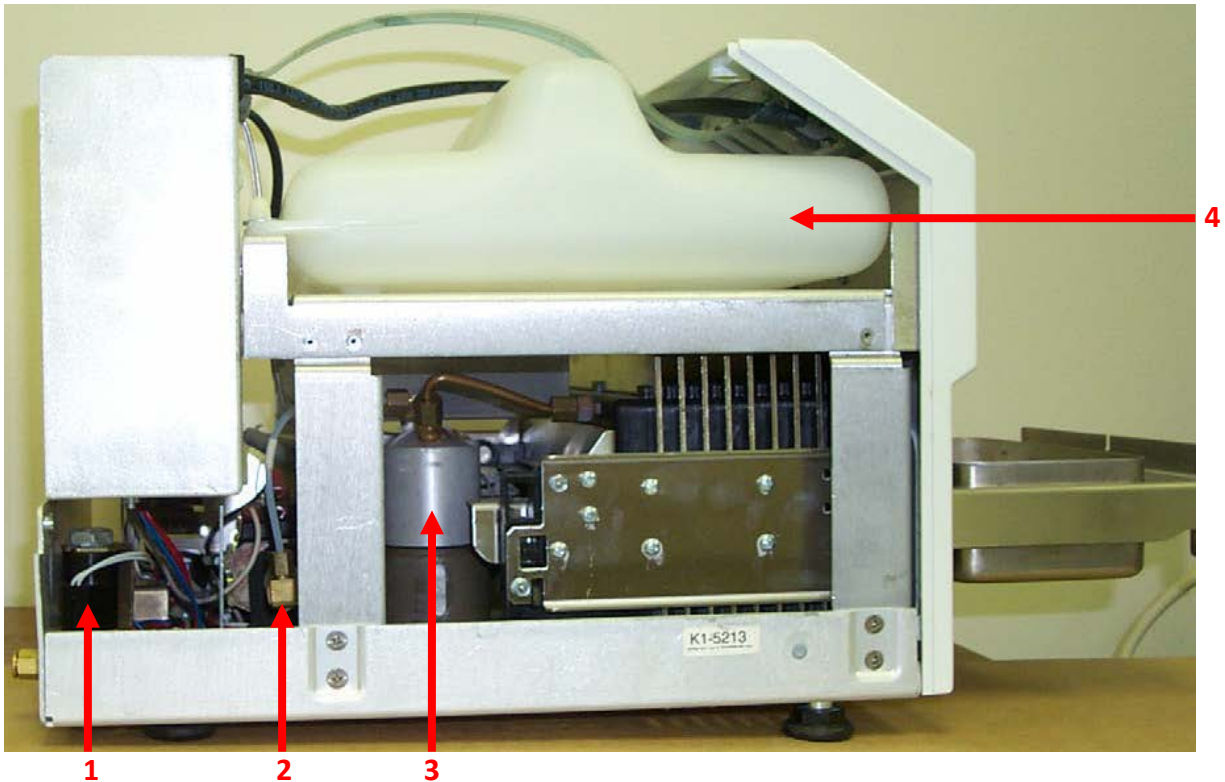

Item #	Part #	Description
1	01-101550S	Solenoid
2	01-102038S	Pump
3	01-110242S	Boiler (software must be 5XX or higher to replace in the field)
4	01-101542S	Reservoir


Statim 900 Right Side No Cover

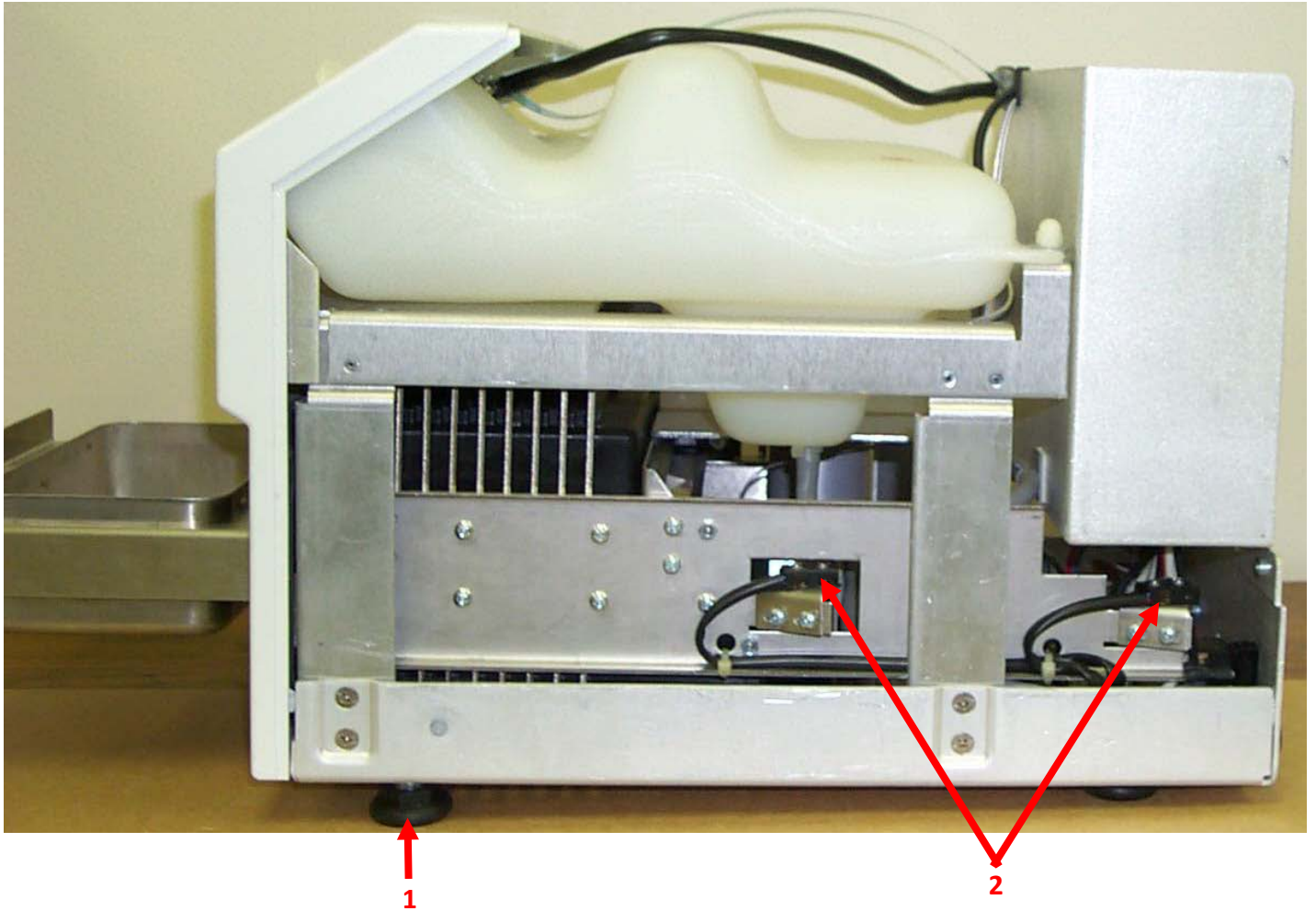

Item #	Part #	Description
1	01-101553S	Leveler
1	01-104180S	Leveler Repair Kit (includes Leveler & Insert for Leveler)
2	01-101552S	Microswitch

Statim 900

Top View No Cover

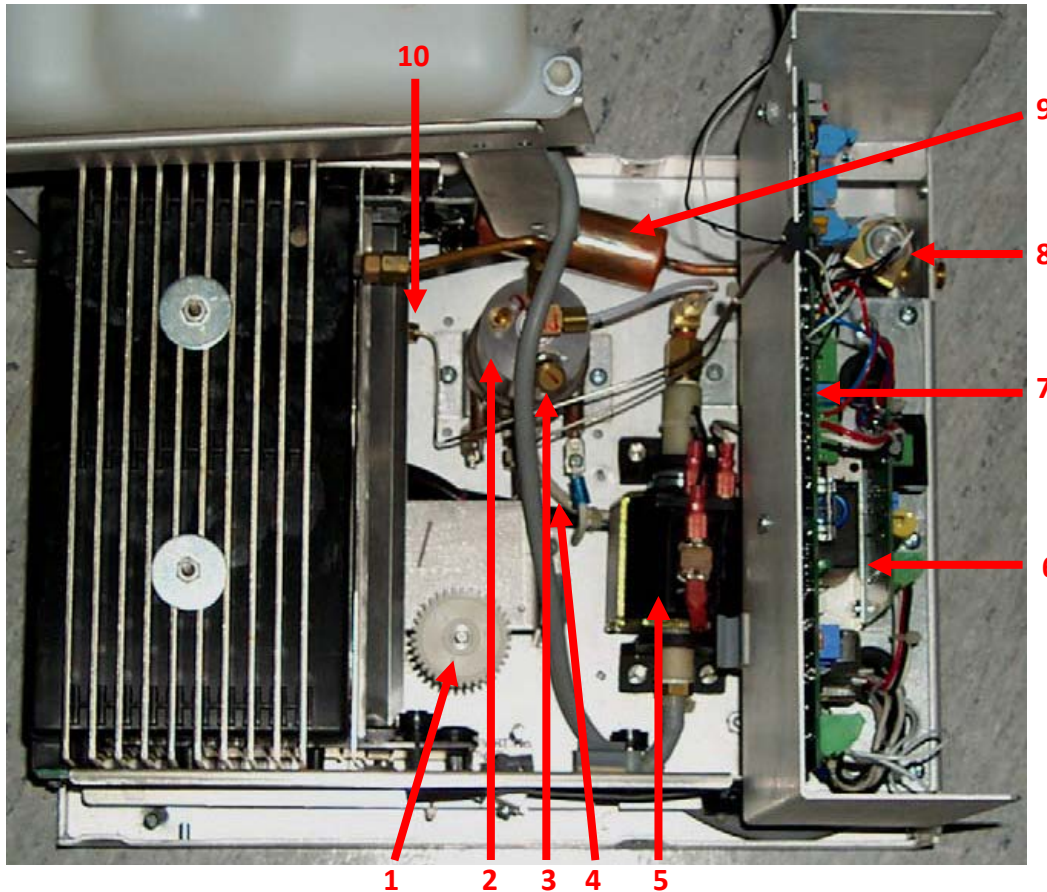

Item #	Parts #	Description
1	01-101545S	Motor
2	01-110242S	Boiler (software must be 5XX or higher to replace in the field)
3	01-103085S	Pressure Relief Valve
4	01-104461S	Thermal Fuse
5	01-102038S	Pump
6	01-102061S	Motor Control PCB
7	01-110462S	PCB Rev. 7 5XX Software (not field replaceable requires complete calibration)
7	01-110461S	PCB Rev. 7 6XX Software (not field replaceable requires complete calibration)
8	01-101550S	Solenoid
9	01-102060S	Exhaust Tube with Accumulator
10	01-101586S	Exhaust Shoe (Chamber Thermocouple)
Not Shown	01-106637S	Water Filter In Line (for Water Pump)

Statim 900 Parts



Basket
01-103636S



Boiler Alex
01-110242S



Basket
01-101554S



Cap Replacement
01-101582S



Chamber Sterilization
01-103608S




Cover
01-101558S



Drawer
01-101562S



Drawer Front
01-101560S

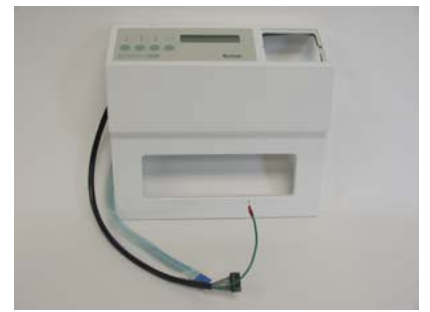

Facia
01-103508S



Filter Insert Exhaust
01-106848S



Fuse .25A
01-103497S



Fuse 15A
01-103472S

Statim 900 Parts



Fuse 2A
01-103498S



Keypad
01-103527S



LCD
01-103138S



Leveler
01-101553S




Leveler Repair Kit
01-104180S



Op Manual
01-106847S



Microswitch
01-101552S

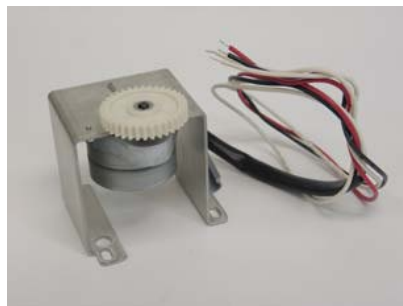

Motor
01-101545S



Motor Control PCB
01-102061S



Plug Drain Tubing
01-104343S



Power Cord
01-101647S



Power Switch
01-100573S

Statim 900 Parts



Pressure Relief Valve
01-103085S



Pump
01-102038S



Push-in Fitting
01-100782S



Reservoir
01-101542S



Reservoir Cap
01-103559S



Seal (Exhaust Only)
01-103144S



Seal (Pressure Only)
01-103143S



Seal Assembly
(Complete w/Plate)
01-106898S



Seal Frame
01-102064S



Seal Kit
(Exhaust & Pressure)
01-101543S



Seal Plate Flat
01-102065S



Seal Screw
01-102063S

Statim 900 Parts



Solenoid Coil
01-104303S



Solenoid Valve Complete
01-101550S



Solenoid Valve Repair Kit
01-100998S



Thermal Fuse
01-104461S




Waste Bottle Complete
01-100812S




Water Filter In-Line
01-106637S

Bravo Accessories

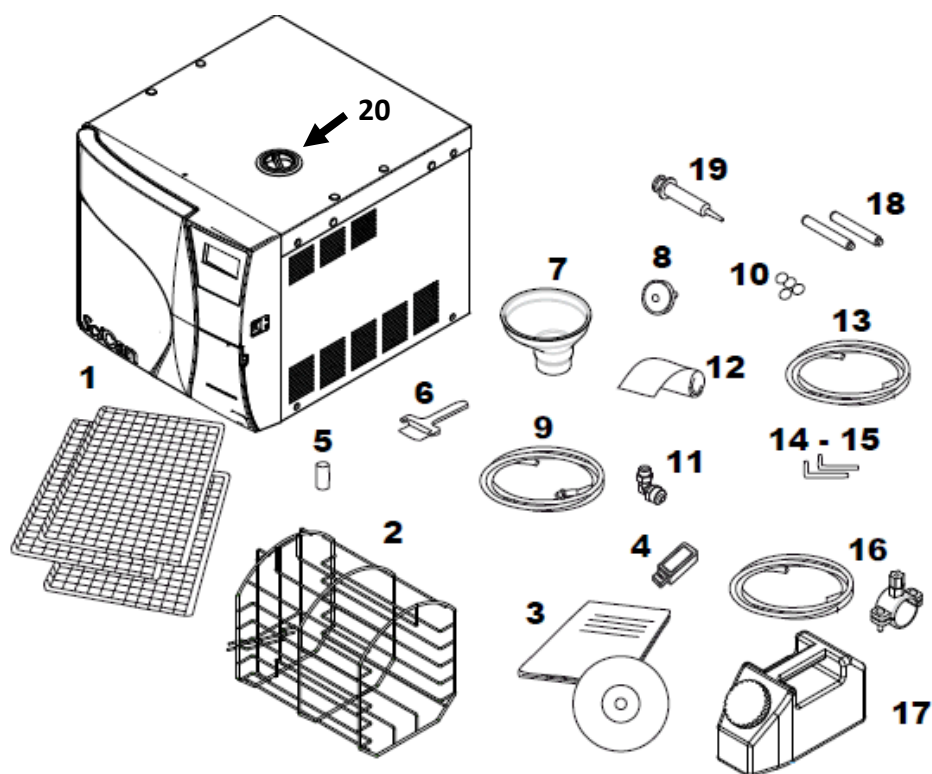

Item #	Part #	Description
1	C1XP076000Y	Tray for 17 & 17V
1	C1BG345000Y	Tray for 21V
2	C1BP583000Y	Rack for Chamber 17 & 17V
2	C1BG534000Y	Rack for Chamber 21V
3	95530358	Operator's Manual w/CD for Bravo2
4	USB-STICK2	USB Drive
5	47200030000	Chamber Filter Screen Metal
6	STXX0080000	Tray Handle
7	97467070	Funnel
8	47200010000	Biological Filter
9	95509115	Drain Tubing w/Fitting
10	97550083	Rubber Caps X 4
11	97960018	Elbow Barbed Fitting 1/8" (part #99950946 is same part)
12	STXX0250000	Printer Paper Thermal Roll
13	97340162	Blue Tubing for Automatic Filling
14	49800010000	Allen Wrench 3MM for Rear Cap Removal
15	58008036	Allen Wrench 5MM for Handle Removal
16	01-111775S	Direct to Drain Kit
17	A0XP0010000	Fill Bottle 2500 ml
18	99950914	Back Spacers Pkg. of 2
19	97270148	Syringe for Priming Pump
20	97467023	Reservoir Cap Bravo II
Not Shown	SD-306	Operator's Manual Original Bravo
Not Shown	18300000400	Chamber Cleaner – Stahl Fix Classic
Not Shown	A7BP5920000	Packaging & Foam Bravo 17
Not Shown	A7BG5380000	Packaging & Foam Bravo 21
Not Shown	01-112320S	Clean Water Reservoir Drain Kit (Bravo 1)


Bravo Parts Table 1

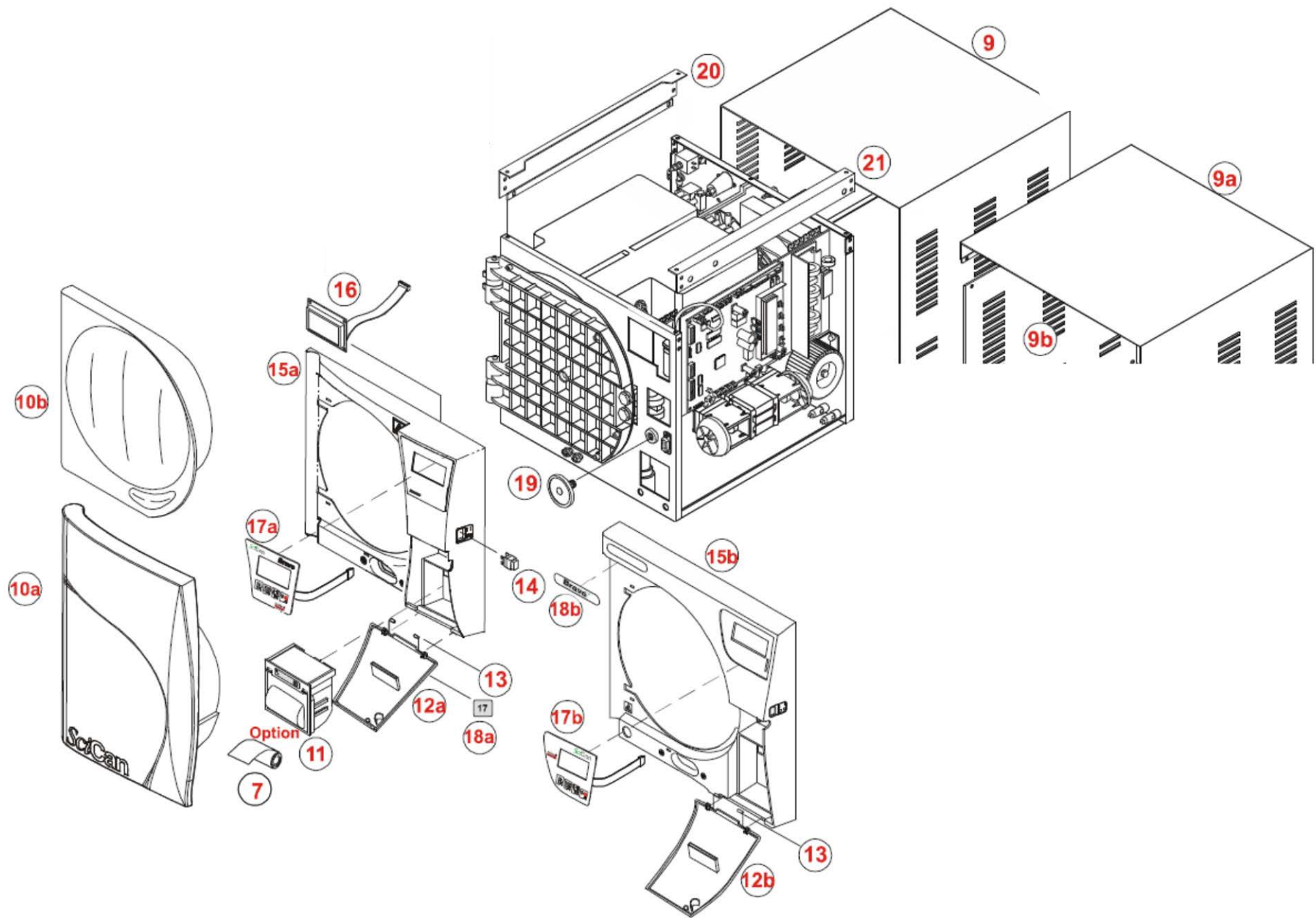

Parts List for Table 1

Item #	Table #	Part #	Description
9	1	C1BH0060001	Cover for Bravo 17 & 17V (1 Piece Cover)
9	1	C1BG4680001	Cover for Bravo 21V (1 Piece Cover)
9a	1	C1BP6230002	Cover Top/Right Side Bravo 17 & 17V (2 Piece Cover)
9a	1	C1BG5760002	Cover Top/Right Side Bravo 21V (2 Piece Cover)
9b	1	C1BP6240002	Cover Left Side Bravo 17 & 17V (2 Piece Cover)
9b	1	C1BG5770002	Cover Left Side Bravo 21V (2 Piece Cover)
10a	1	97467027	Door Cover Plastic External Bravo 2
10b	1	97467028	Door Cover Plastic Internal Bravo 2
10c	1	C3BP1850000	Door Cover Plastic Original Bravo
11	1	C5BP5500000	Printer Thermal Bravo (Printer only)
11	1	A9BP5430000	Printer Kit Bravo Gam (to upgrade Bravo with no printer)
11	1	95530152	Printer Kit Bravo Troll (to upgrade Bravo with no printer)
12a	1	C3JP0140000	Printer Door Bravo 2 (New Style)
12b	1	C3BP1840000	Printer Door Bravo (Original Style)
Not	Shown	99950374	Round Metal Locking Plate for Printer Door
13	1	230B04L20K0	Printer Door Pin 4 x 20 Bravo
14	1	97600130	Power Switch Bravo (Original Style)
15a	1	C3JP0110000	Facia Front Plastic Bravo 2
15b	1	C3BP1830000	Facia Front Plastic Bravo (Original Style)
Not	Shown	95509172	Facia Front Label Kit
16	1	C5BP1230000	LCD Bravo (sn 08xxnnnn to 10xxnnnn)
16	1	97901487	LCD Bravo (sn 11xxnnnn & higher)
17a	1	C6JP0190000	Keypad Bravo 2
17b	1	C6JP0010000	Keypad Bravo (Original Style)
18b	1	C6JM0010000	Label Bravo 17 (Original Style)
18b	1	C6JP0020000	Label Bravo 17V (Original Style)
18b	1	97781388	Label Bravo 21V (Original Style)
19	1	47200010000	Biological Filter Bravo
20	1	99950410	Left Crossbar Bravo 17 & 17V
20	1	C1BG317000M	Left Crossbar Bravo 21V (1 Piece Cover)
20	1	C1BG575000M	Left Crossbar Bravo 21V (2 Piece Cover)
20	1	99950333	Left Crossbar Bravo 2, 21V
21	1	C1BP152000M	Right Crossbar Bravo 17 & 17V (1 Piece Cover)
21	1	C1BP654000M	Right Crossbar Bravo 17 & 17V (2 Piece Cover)
21	1	99950409	Right Crossbar Bravo 2, 17V
21	1	C1BG318000M	Right Crossbar Bravo 21V (1 Piece Cover)
21	1	C1BG574000M	Right Crossbar Bravo 21V (2 Piece Cover)
21	1	99950332	Right Crossbar Bravo 2, 21V
Not	Shown	201C04L10K0	Screws for Cover & Crossbar
Not	Shown	97668116	USB Interface Board (for Bravo 2)
Not	Shown	97526082	USB Flat Cable (for Bravo 2)
Not	Shown	97781393	Label for Printer Door 17V (for Bravo 2)
Not	Shown	97781389	Label for Printer Door 21V (for Bravo 2)

Bravo Parts Table 2

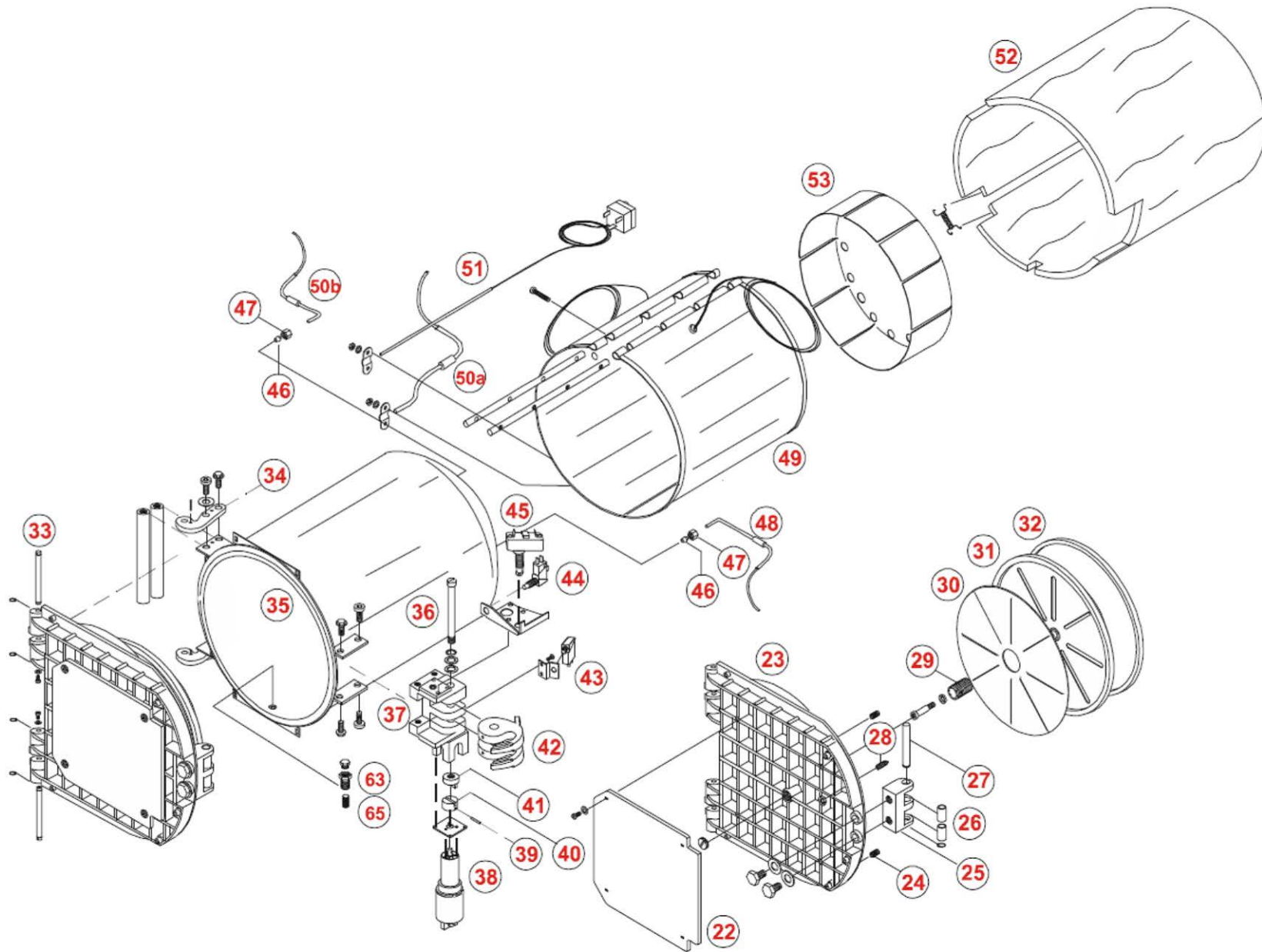


Parts List for Table 2

Item #	Table #	Part #	Description
22	2	C4BP2740000	Door Thermo insulating panel Bravo
23	2	C0BP260000P	Door Front Metal Bravo
24	2	C7BP2640000	Door Spring Bravo
25	2	C0BP060000Z	Door Fork Bravo
26	2	49000050000	Door Bushing Bravo
27	2	C0BP0500000	Door Fork Pin Bravo
28	2	203C06L30K0	Door Center Screw Bravo
29	2	C0BP079000N	Door Plate Adjustment Screw Bravo
30	2	C4BP5690000	Door Plate Thermo Insulating Bravo
31	2	A1BP027000Y	Door Seal Plate Bravo
32	2	48000050000	Door Gasket Bravo
33	2	C0BP2720000	Door Hinge Pin Bravo
34	2	99950138	Door Hinge Bravo
35	2	C2BP0010000	Chamber Bravo 17 & 17V
35	2	C2BG5300000	Chamber Bravo 21 & 21V
36	2	C0BP0430000	Door Locking System Pin Bravo
37	2	A1BP2710000	Door Locking System Complete Bravo
38	2	95530349	Door Locking System Motor Assy. Complete Bravo
39	2	232A03L25K0	Door Locking System Pin 3 x 25 Bravo
40	2	C0BP0450000	Door Locking System Motor Coupling Bravo
41	2	C0BP0460000	Door Locking System Intermediate Coupling Bravo
42	2	C0BP0590000	Door Locking System Closing Hook Bravo
43	2	43300010000	Door Locking System Hook Microswitch (open) Bravo
44	2	43300010000	Door Locking System Door Microswitch Bravo
45	2	43300080000	Door Locking System Hook Microswitch (close) Bravo
46	2	28600160000	Temperature Sensor Ferrule Bravo
47	2	2860014000N	Temperature Sensor Nut Bravo
48	2	43000190000	Temperature Sensor PT1000 – PT1 Bravo
49	2	41000180000	Chamber Heater 17 & 17V 1500W 120V Bravo
49	2	41000280000	Chamber Heater 17 & 17V 2000W 220V Bravo
49	2	41000190000	Chamber Heater 21V 1500W 120V Bravo
49	2	41000290000	Chamber Heater 21V 2000W 220V Bravo
50a	2	43000180000	Temperature Sensor PT1000 – PT3 Bravo
50b	2	43000180000	Temperature Sensor PT1000 – PT4 Bravo
50c	4	43000180000	Temperature Sensor PT1000 – PT2 Bravo
51	2	97730010	Safety Thermostat TG400 Chamber Heater Bravo
52	2	A8BPR140000	Thermo Insulating Panel Kit for Chamber Bravo 17 & 17V
52	2	A8BGR010000	Thermo Insulating Panel Kit for Chamber Bravo 21V
53	2	99950938	Thermo Insulating Panel Rear for Chamber Bravo
63	2	99950911	Fitting for Chamber Filter Bravo
65	2	47200030000	Chamber Filter Screen Metal Bravo

Note: Item #52 Panel kits include 3 springs & 6 hooks each


Bravo Parts Table 3

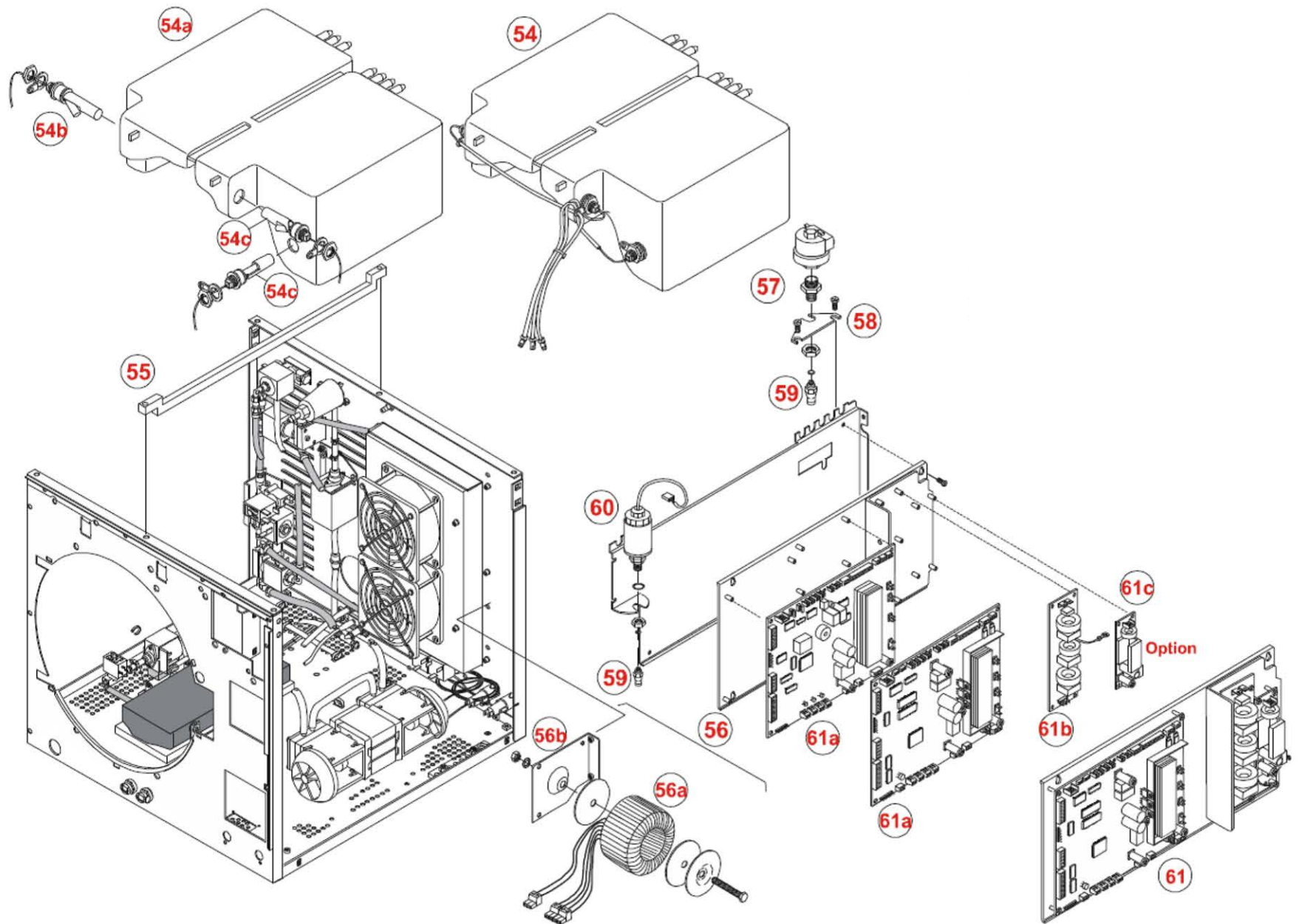

Parts List for Table 3

Item #	Table #	Part #	Description
54	3	A0BP2900000	Reservoir Assembly Complete Bravo
54a	3	C3BP1700000	Reservoir and Waste Water Tank Bravo
54b	3	97668046	Float Switch Waste Water Tank Bravo
54c	3	97668045	Float Switch Clean Water Reservoir Bravo
55	3	C1BP175000M	Support Bar Reservoir Bravo 17 & 17V
55	3	C1BG320000M	Support Bar Reservoir Bravo 21 & 21V
56	3	A9BP5860000	Transformer Complete Bravo 120V (Gam Board)
56	3	A9BP5970000	Transformer Complete Bravo 220V (Gam Board)
56a	3	41200170000	Transformer Only Bravo 120V (Gam Board)
56a	3	41200080000	Transformer Only Bravo 220V (Gam Board)
56a	3	41200170000	Transformer Only Bravo 120V (Troll Board)
56a	3	41200110000	Transformer Only Bravo 220V (Troll Board)
56b	3	C1BP161000M	Transformer Support Plate Bravo
57	3	43100060000	Pressure Switch Bravo
58	3	C1BP538000M	Pressure Switch Support Bravo
59	3	97985500	Pressure Switch/Transducer Quick Connect Fitting Bravo
60	3	43100090000	Pressure Transducer Bravo
61a	3	See below	Basic Board (Gam) Bravo 120V/220V (See conversion kit below)
61a	3	C5BM1400200	CPU Board (Troll) Bravo 120V/220V
61b	3	C5BP4350000	PCB Filter Board (GAM) Bravo 120V
61b	3	C5BP1420000	PCB Filter Board (GAM) Bravo 220V 50/60Hz
61b	3	C5BM1420000	PCB Filter Board (Troll) Bravo 120V/220V
61c	3	C5BM1430000	PCB Printer Board Bravo
Not	Shown	SB17V120V	Bravo Software (Gam Board 17V/120V) Serial # <10JP0033
Not	Shown	SB17V220V	Bravo Software (Gam Board 17V/220V) Serial # <10JP0040
Not	Shown	SB17120V	Bravo Software (Gam Board 17/120V) Serial # <10JM0008
Not	Shown	SB17220V	Bravo Software (Gam Board 17/220V) Serial # <10JM0003
Not	Shown	SB21V120V	Bravo Software (Gam Board 21V/120V) Serial # <10JG0076
Not	Shown	SB21V220V	Bravo Software (Gam Board 21V/220V) Serial # <10JG0031
Not	Shown	A9BP5430000	Printer Upgrade Kit (Gam) Bravo
Not	Shown	95530085	Printer Upgrade Kit (Troll) Bravo
Not	Shown	BRAVO-GT2P120	Bravo Conversion Kit Gam to Troll2 w/Printer 120V
Not	Shown	BRAVO-GT2NP120	Bravo Conversion Kit Gam to Troll2 no Printer 120V
Not	Shown	BRAVO-GT2P220	Bravo Conversion Kit Gam to Troll2 w/Printer 220V
Not	Shown	BRAVO-GT2NP220	Bravo Conversion Kit Gam to Troll2 no Printer 220V
Not	Shown	BRAVO FUSEHOLD	Bravo Fuse Holder for GAM PCB

Bravo Parts Table 4

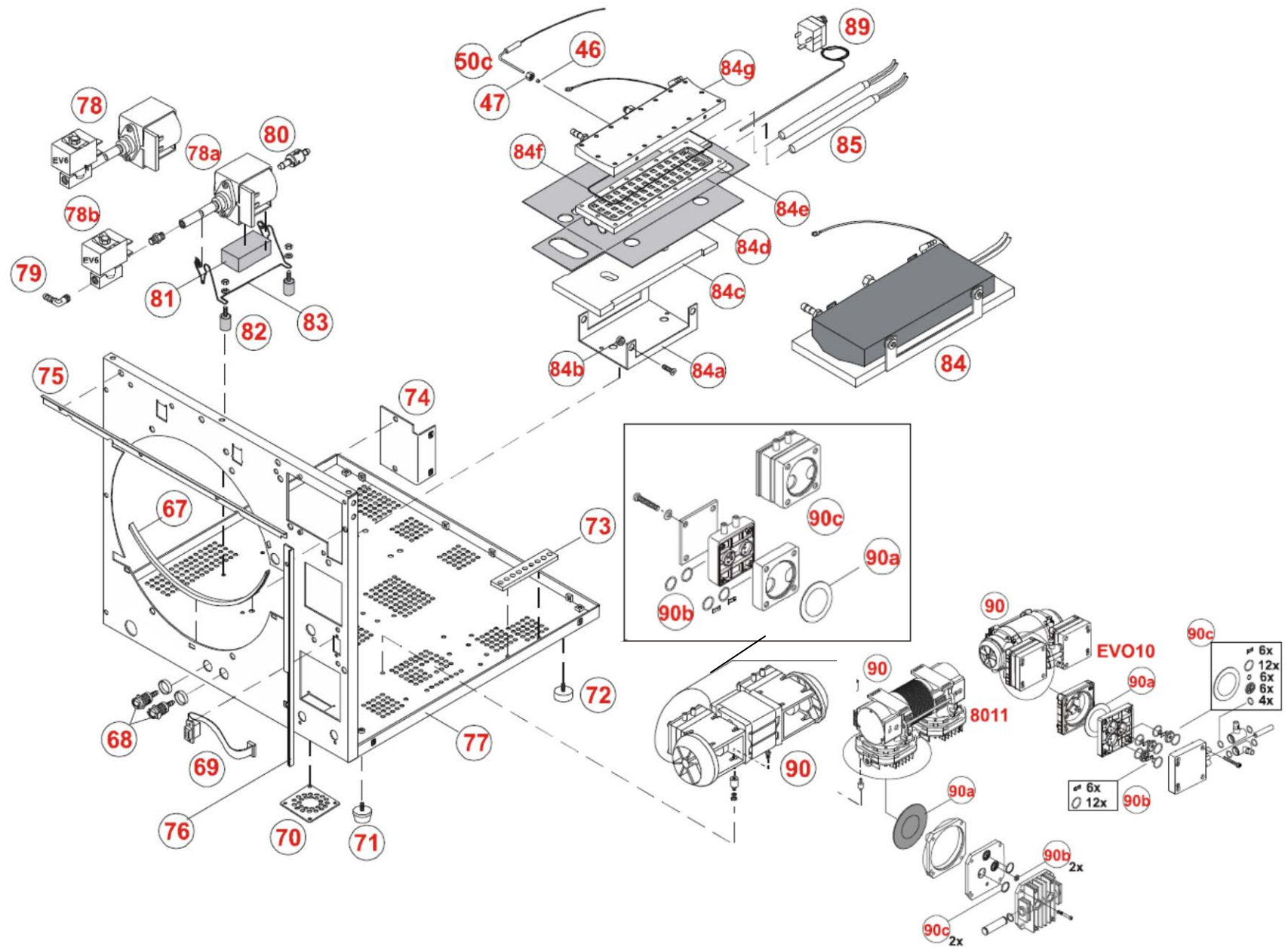


Parts List for Table 4


Item #	Table #	Part #	Description
63	2	99950911	Filter Fitting Chamber Front Bravo
65	2	47200030000	Filter Screen Chamber Front LP1 Bravo
67	4	48300010000	Gasket Rubber Bravo
68	4	260000006V0	Fitting CPC Female Water Tap Chamber Front Bravo
69	4	A2BP2380000	Printer Plug RS232 9 pin w/Cable Bravo
70	4	C3BP5610000	Plate ABS Bravo
71	4	25600000400	Foot Front PVC Bravo
72	4	95509127	Rear Leg Kit (if rear leg insert is broken order kit and next 4 parts)
Not	Shown	97467015	Plastic Spacer (required to replace broken rear leg insert only)
Not	Shown	51242040	M6x40 TPSEI screw (required to replace broken front & rear leg
Not	Shown	53104010	D6,4x12,5 washer (required to replace broken front & rear leg
Not	Shown	52210031	M6 nut (required to replace broken front & rear leg insert)
73	4	99950119	Plate Ground Bravo
74	4	C1BP147000M	Support Electronic Angle Bravo
75	4	C1BP269000M	Support Front Top Angle Bar Bravo
76	4	C1BP270000M	Support Front Side Angle Bar Bravo
77	4	99950359	Chassis Bravo 17 & 17V
77	4	A1BG350000M	Chassis Bravo 21 & 21V
78	4	95509145	Water Pump Complete w/Solenoid Bravo
78a	4	40000220000	Water Pump 24V Bravo
78b	4	40100370000	Valve EV6 for Water Pump Bravo
79	4	97960018	Fitting Elbow for Water Pump Solenoid Bravo
80	4	97290166	Water Filter for Water Pumps Bravo (1 Each)
81	4	111000006W0	Water Pump Adhesive Mousse Bravo
82	4	25600000800	Water Pump Mounting Bracket Foot Bravo
83	4	40000050000	Water Pump Mounting Bracket Bravo
84	4	A9BM5510000	Steam Generator Complete Bravo 120V
84	4	A9BP5250000	Steam Generator Complete Bravo 220V
84a	4	C1BP136000M	Steam Generator Support Bravo
84b	4	C0BP1310000	Steam Generator Support Spacer Insulator Bravo
84c	4	C4BP2360000	Steam Generator Thermo Insulating Panel End Bravo
84d	4	C4BP5700000	Steam Generator Thermo Insulating Cover Bravo
84e	4	C0BP548000P	Steam Generator Lower Part Bravo
84f	4	48100080000	Steam Generator O-ring Bravo
84g	4	C0BP547000P	Steam Generator Upper Part Bravo
85	4	41000170000	Steam Generator Heating Element Bravo 115V 850W
85	4	41000260000	Steam Generator Heating Element Bravo 220V 1000W
86	4	43000180000	Temperature Sensor PT1000 – PT2 Bravo
87	4	2860014000N	Steam Generator Temperature Sensor Nut Bravo
88	4	28600160000	Steam Generator Temperature Sensor Ferrule Bravo
89	4	97730011	Steam Generator Safety Thermostat TG400 Bravo
90	4A	A9BM5520000	Vacuum Pump Small Complete Bravo 17/120V
90	4A	A9BM5270000	Vacuum Pump Small Complete Bravo 17/220V
90	4B	40000420000	Vacuum Pump EVO10 Complete Bravo 17V/120V
90	4B	A8BPR010000	Vacuum Pump EVO10 Complete Bravo 17V/220V
90	4B	40000420000	Vacuum Pump 8011 Large Complete Bravo 21V/120V
90	4B	A9BP5270000	Vacuum Pump 8011 Large Complete Bravo 21V/220V
Not	Shown	CBB60-D45	Capacitor 5uF for 8011 Thomas Vacuum Pump Bravo 21V
Not	Shown	CBB60-D4	Capacitor 4uF for 8011 Thomas Vacuum Pump Bravo 21V

Note: See Table 4a & 4b for Vacuum Pump parts.

Bravo Parts Table 4A

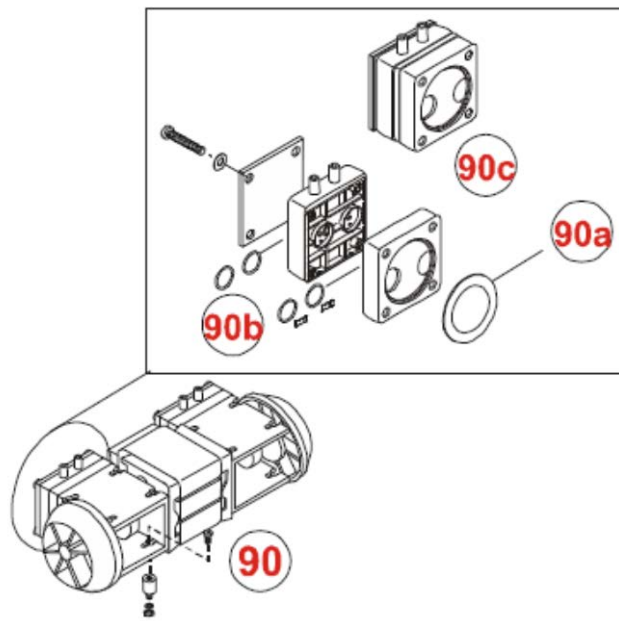

Bravo Parts Table 4B

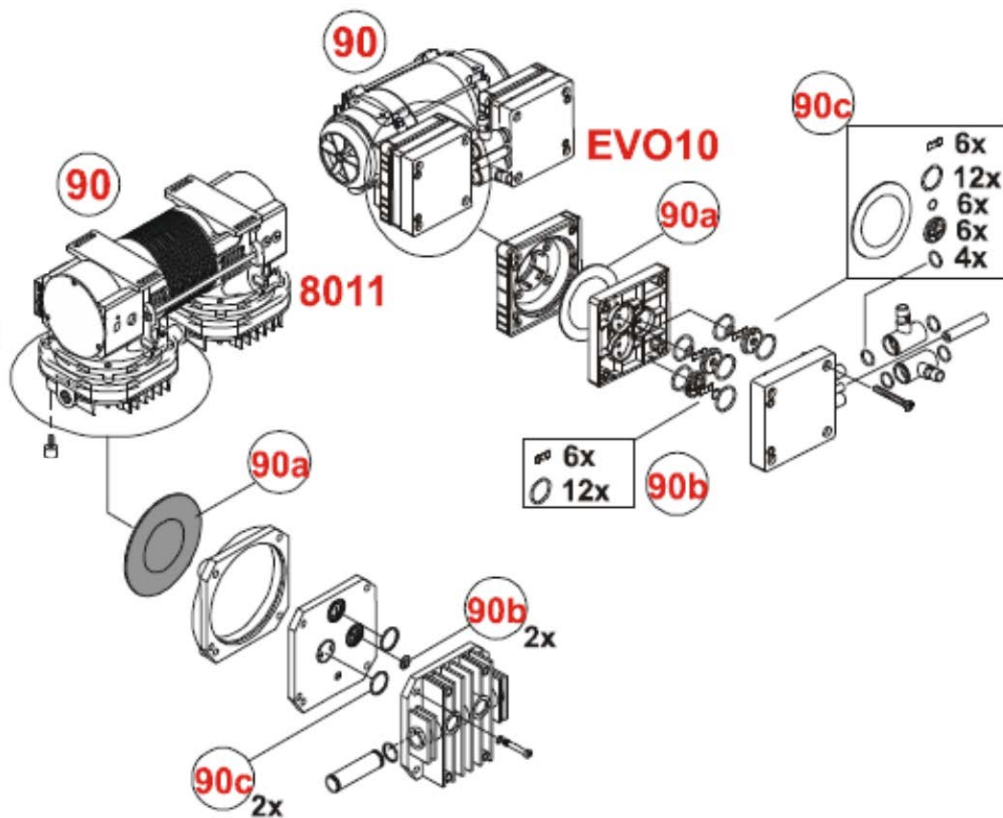

Parts List for Table 4A

H5P3 Vacuum Pump Bravo 17

Item #	Table #	Part #	Description
90	4A	A9BM5520000	H5P3 Vacuum Pump Small Complete Bravo 17/120V
90	4A	A9BM5270000	H5P3 Vacuum Pump Small Complete Bravo 17/220V
90a	4A	SCI-902-02	H5P3 Diaphragm (Pkg. of 2)
90b	4A	SCI-900-12	H5P3 Single Head Repair Kit (4 O-rings, 2 Shutters & 1 Diaphragm) Bravo 17
90b	4A	SCI-900-18	H5P3 Single Head Repair Kit (4 O-rings, 2 Shutters no Diaphragm) Bravo 17
90b	4A	SCI-901-03	H5P3 Dual Head Repair Kit (8 O-rings, 4 Shutters & 2 Diaphragms) Bravo 17
90b	4A	SCI-901-01	H5P3 Dual Head Repair Kit (8 O-rings, 4 Shutters no Diaphragms) Bravo 17
90c	4A	SCI-200-06	H5P3 Complete Head no Diaphragm Bravo 17
90c	4A	SCI-900-06	H5P3 Complete Head with Diaphragm Bravo 17
90d	4A	SCI-70-303	H5P3 Shutter

Parts List for Table 4B

EVO10 Vacuum Pump Bravo 17V/21V

Item #	Table #	Part #	Description
90	4B	40000420000	EVO10 Vacuum Pump Complete Bravo 17V/21V 120V
90	4B	A8BPR010000	EVO10 Vacuum Pump Complete Bravo 17V/21V 220V
90a	4B	SCI-900-44	EVO10 Diaphragm (Pkg. of 2)
	4B	SCI-900-40	EVO10 Head Repair Kit (6 Discs, 12 O-rings & 2 Diaphragms)
	4B	SCI-901-30	EVO10 Repair Kit 6 Shutters & 12 O-rings
	4B	SCI-900-41	EVO10 C-clip (Pkg. of 6)
Not	Shown	CBB60-D4	EVO10 Capacitor 4uF
Not	Shown	CBB60-D45	EVO10 Capacitor 5uF
Not	Shown	SCI-388-224	EVO 10 Screw for Rubber Foot
Not	Shown	SCI-531-058	EVO10 Pump Head Screw
Not	Shown	SCI-50-200	EVO10 Rubber Mounting Feet

Parts List for Table 4B


8011 Vacuum Pump Bravo 21V

Item #	Table #	Part #	Description
90	4B	A9BP5270000	Vacuum Pump 8011 Complete Bravo 21V/220V
90a	4B	40000260000	Vacuum Pump 8011 Membrane (Qty 2)
90b	4B	40000250000	Vacuum Pump 8011 Shutter (Qty 4)
90c	4B	48100150000	Vacuum Pump 8011 O-ring (Qty 4)

Note: A9BP5870000 Vacuum Pump 8011 21V/120V no longer available use 40000420000

Note: Items with part# & description in red are required to rebuild the vacuum Pump






Bravo Parts Table 5

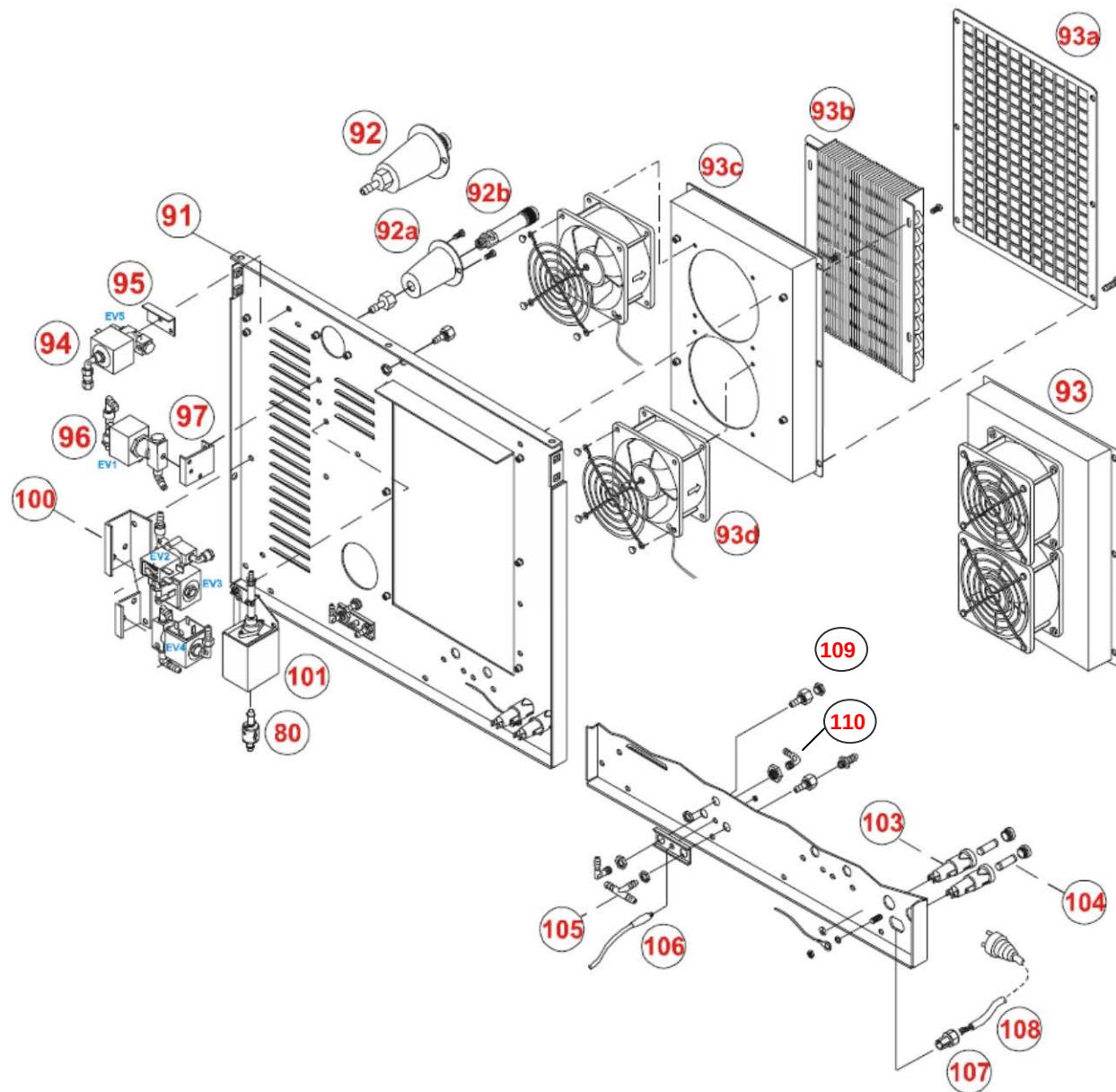

Parts List for Table 5

Item #	Table #	Part #	Description
80	4	97290166	Water Filter for Water Pumps Bravo (1 Each)
91	5	99950913	Chassis Rear Bravo (also need part #97440043)
92a	5	95530084	Pressure Relief Valve Assy. Bravo
92b	5	47000020000	Pressure Relief Valve Only Bravo
93	5	C1BP6220000	Heat Exchanger Complete Bravo
93a	5	40400040000	Heat Exchanger Protection Grid Bravo
93b	5	C1BP6220000	Heat Exchanger Only Bravo
93c	5	C1BP159000M	Heat Exchanger Support Bravo
93d	5	40400030000	Heat Exchanger Fan
94	5	40100240000	Valve EV 3 Way 24V Bravo
95	5	55V432011000	Valve EV5 Support Bravo
96	5	40100240000	Valve EV 3 Way 24V Bravo
97	5	C1BP241000M	Valve EV1 Support Bracket Bravo
100	5	C1BP173000M	Valve EV2/EV3/EV4 Support Bracket Bravo
101	5	40000220000	Water Pump 24V Bravo
102	5	97290166	Water Filter for Water Pumps Bravo (1 each)
103	5	01-112545S	Fuse Holder Bravo 120V/220V (Pkg of 2)
104	5	97760031	Fuse 15A 250V Bravo
105	5	261000014V0	Tee Fitting Bravo
106	5	A2BP2630000	Level Sensor Cable External Bravo
107	5	27700003500	Strain Relief Bravo
108	5	97526181	Power Cord Bravo 120V
108	5	97526182	Power Cord Bravo 220V
109	5	266000007MN	Vent Hole Nut Bravo
109	5	97960047	Fitting 1/4 x 1/8 Auto Fill Bravo
109	5	97960048	Fitting 1/8 Ext.
109	5	97960056	Fitting 1/4 Cap
110	5	97960018	Elbow Barbed Fitting 1/8" (part #99950946 is same part)


Bravo Parts Table 6

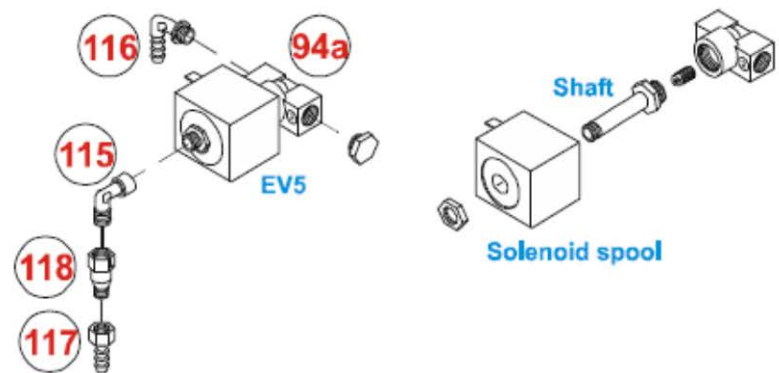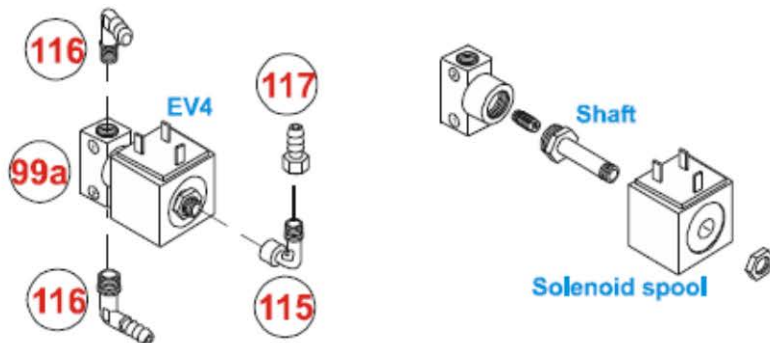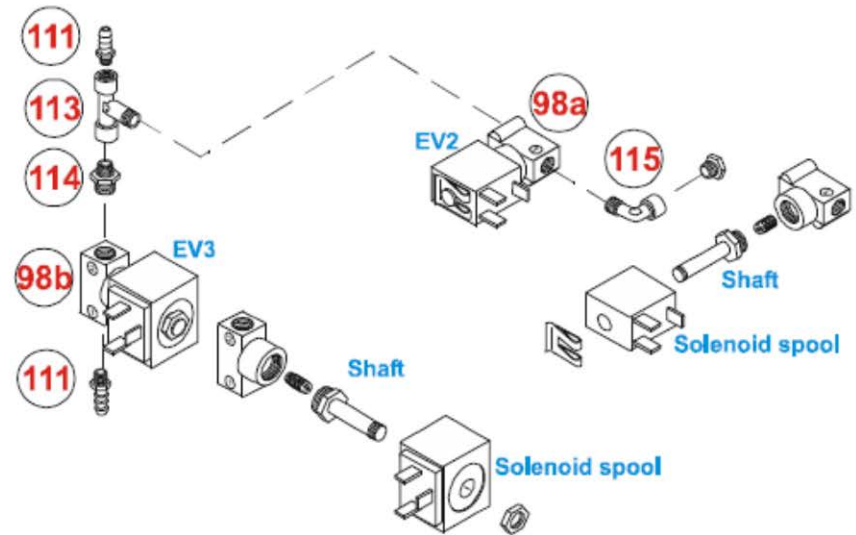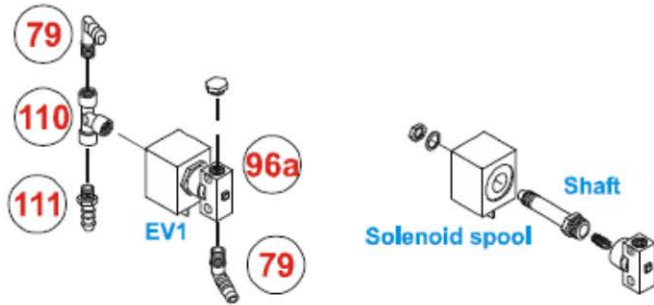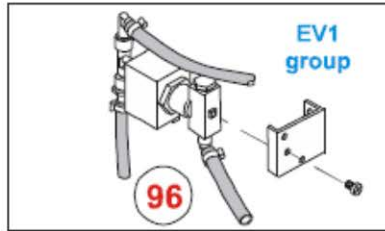

Parts List for Table 6

Item #	Table #	Part #	Description
79	6	97960018	Fitting Elbow Barbed 1/8" Bravo
94a	6	40100240000	Valve EV 3 Way 24V Bravo
96	6	40100240000	Valve EV 3 Way 24V Bravo
96a	6	40100240000	Valve EV 3 Way 24V Bravo
98a	6	40100370000	Valve EV 2 Way 24V Bravo
98b	6	40100230000	Valve EV 2 Way 24V Bravo
99a	6	97720185	Valve EV4 24V Bravo
110	6	263000002MN	Fitting Tee Female/Female/Female Bravo
111	6	99950948	Fitting Pipe Barbed Bravo
113	6	263000003MN	Fitting Tee Female/Male/Female Bravo
114	6	265000009MN	Fitting Coupling Bravo
115	6	262000003MN	Fitting Elbow Female/Male Bravo
116	6	99950947	Fitting Elbow Barbed Bravo
117	6	261000004MN	Fitting Pipe Barbed Bravo
118	6	47000010000	Fitting One Way Valve Connector Bravo


Bravo Parts Table 7

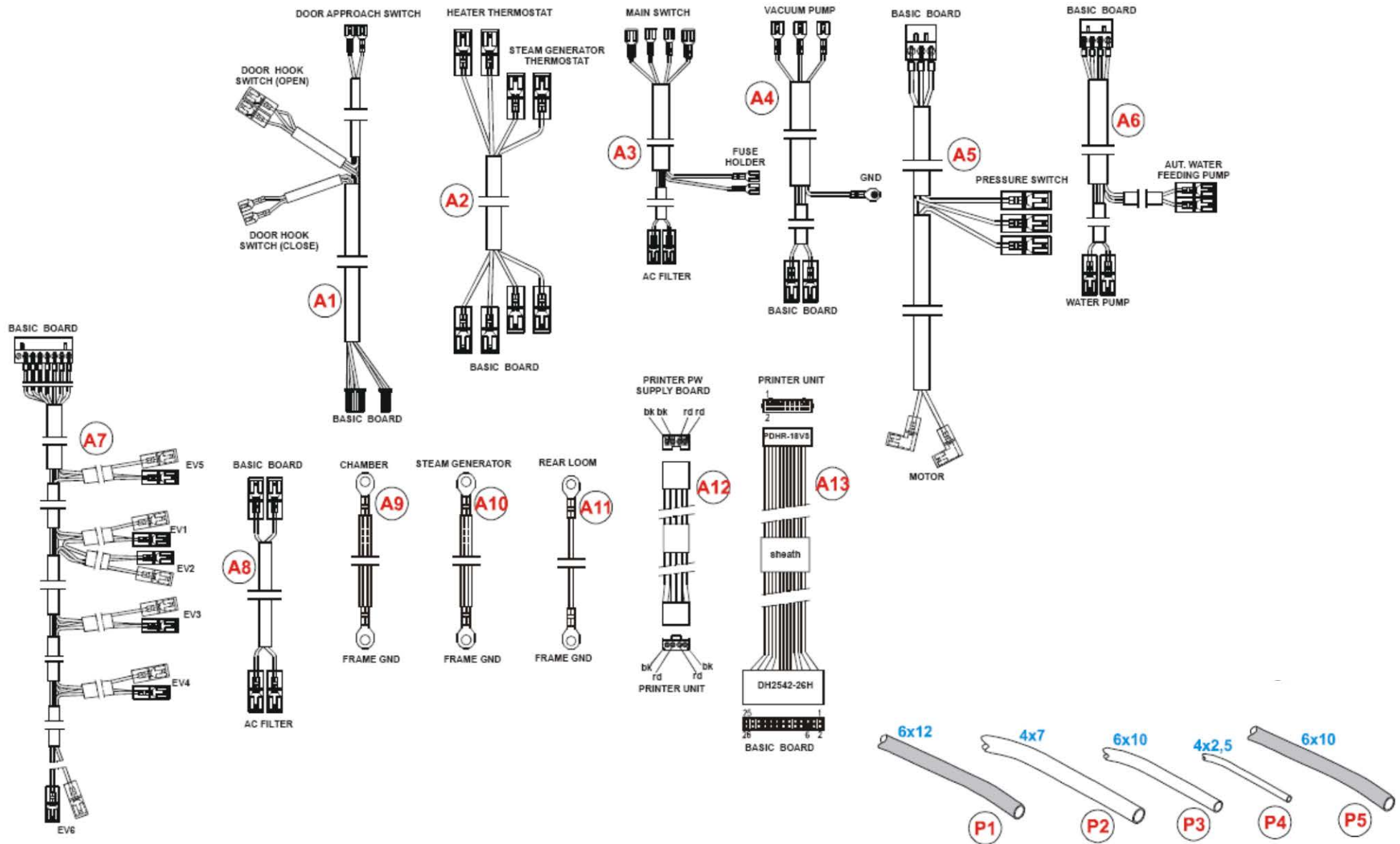

Parts List for Table 7

Item #	Table #	Part #	Description
A1	7	A2BP2220000	Cable Microswitch Bravo
A2	7	A2BP4010000	Cable Safety Thermostat Bravo
A3	7	97526203	Cable Main Switch Bravo
A4	7	97526046	Cable Vacuum Pump Bravo 17
A4	7	97526040	Cable Vacuum Pump Bravo 17V & 21V
A5	7	A2BG3520000	Cable Motor Reducer Bravo
A6	7	A2BP2270000	Cable Water Pump Bravo
A7	7	A2BP2280000	Cable Electro Valve Bravo
A8	7	A2BP4070000	Cable Filter Bravo
A9	7	A2BP4820000	Cable Ground Chamber Bravo
A10	7	A2BP4820000	Cable Ground Steam Generator Bravo
A11	7	A2BP4810000	Cable Ground Rear Frame Bravo
A12	7	A2BP4700000	Printer Cable Power Bravo
A13	7	A2BP4690000	Printer Cable Signal Bravo
P1	7	110000005W0	Silicone Tube 6 x 12 Black Bravo
P2	7	110000011W0	Silicone Tube 4 x 7 Transparent Bravo
P3	7	95509115	Silicone Tube 6 X 10 w/Quick Connect for Drain Bravo
P4	7	110000002V0	Teflon Tube 4 X 2.5 Bravo
P5	7	110000018W0	Silicone Tube 6 X 10 Bravo (Steam Generator to Chamber)

Bravo Fuses

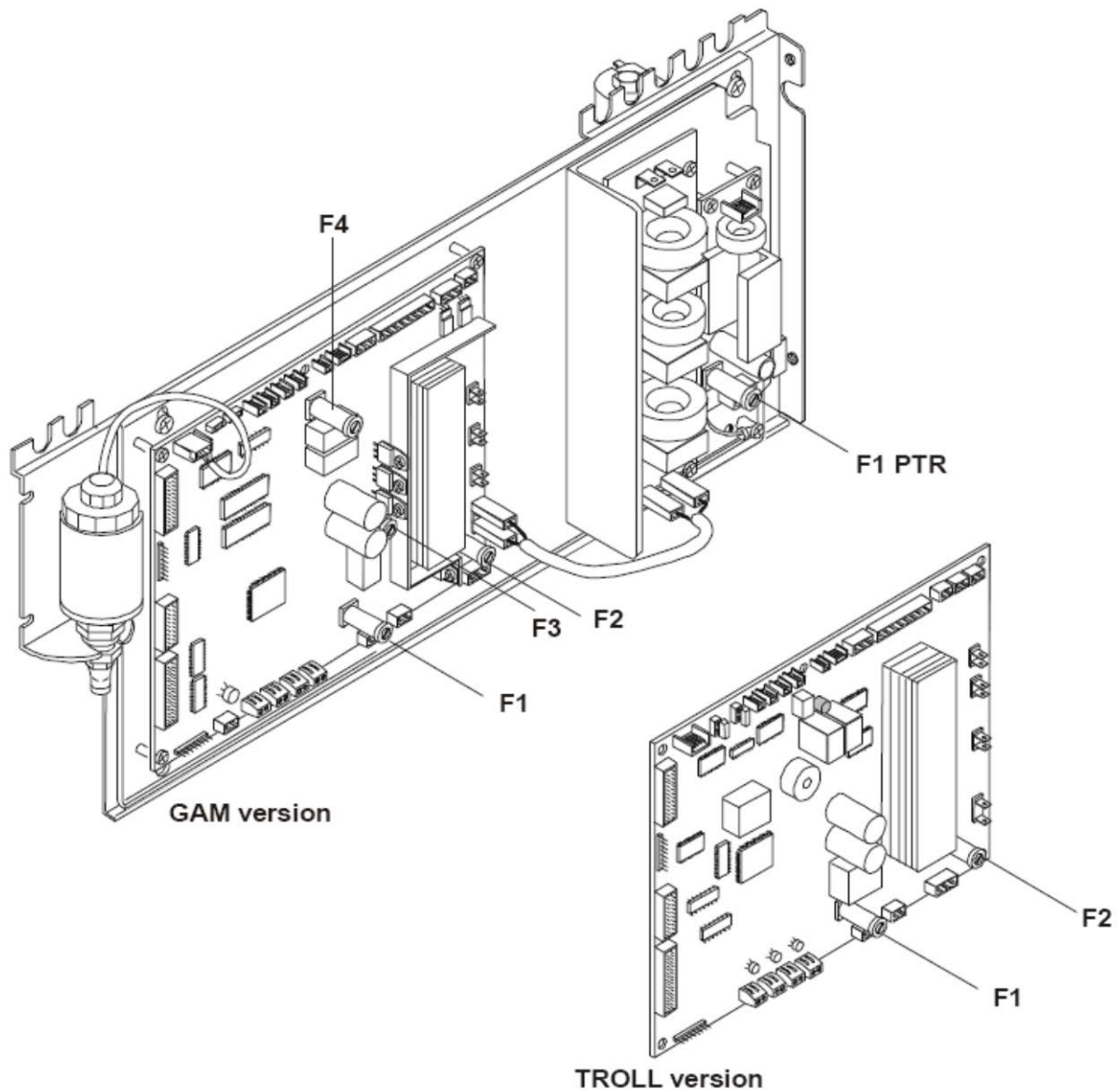

Basic board: GAM version

F1: T 5A 250V (secondary transformer winding)

F2: T 4A 250V (120V Bravo)(Primary transformer winding)

F2: T 2A 250V (220V Bravo)(Primary transformer winding)

F3: T 200mA 250V (door-lock accidental activation)

F4: F 1.25A 250V (door-lock overload)

part # Fuse-T-5A

part # Fuse-T-4A

part # Fuse-TT-2A

part # Fuse-F-200MA

part # Fuse-F-1.25A

Basic board: TROLL version

F1: T 6.3A 250V (secondary transformer winding)

F2: T 3.15A 250V (primary transformer winding)

part # Fuse-T-6.3A

part # Fuse-T-3.15A

Printer PS board

F1 PTR: T 5A 250V (printer protection)


part # Fuse-T-5A

Line Fuses 15A 250V ABC (Fast Acting)(not shown)

part # 97760031

Bravo Fittings


Pressure Transducer & Safety Pressure Switch

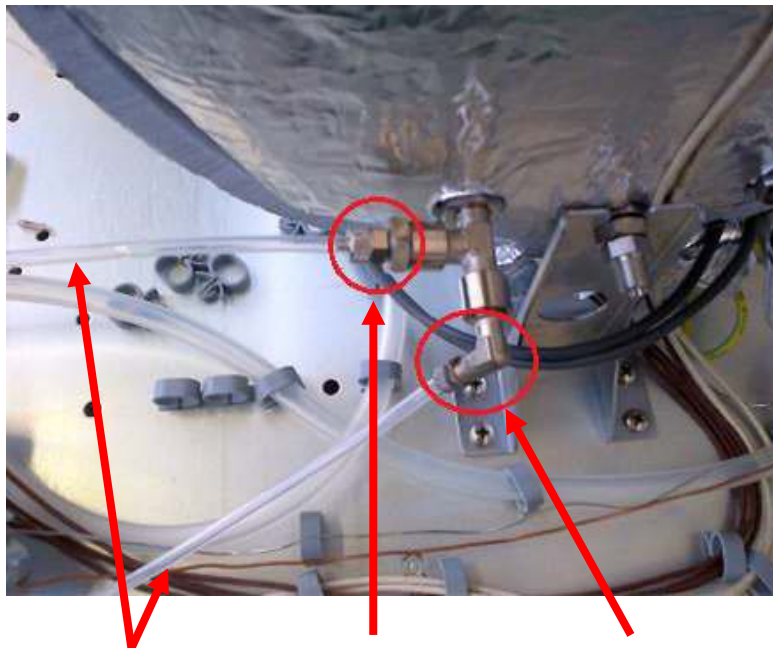

**Teflon Tubing
D4 x2.7
97340164**

**Fitting
Pressure Switch
97985500**

**Elbow Fitting
Assembly
95509171**

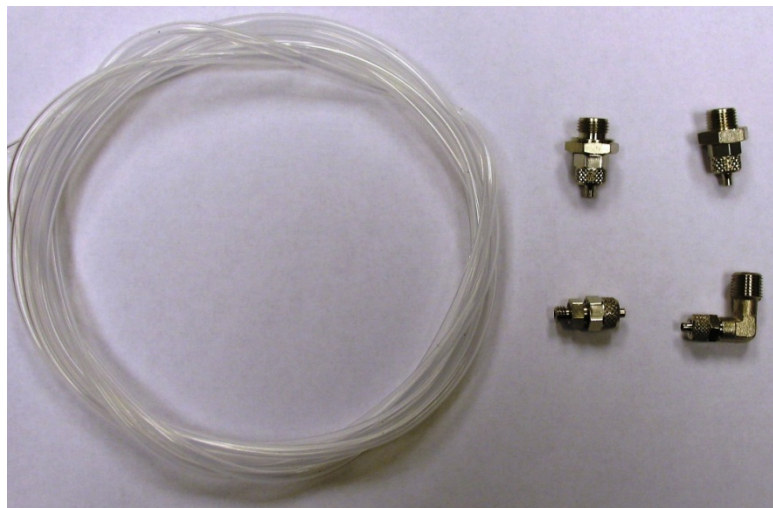

**Fitting Kit w/Tubing
Pressure Transducer & Switch
95509138**

**Pressure
Transducer
43100090000**



**Fitting
Transducer
97985600**

**Pressure
Switch
43100060000**




**Fitting
Pressure Switch
97985500**

Bravo Individual Parts Pictures




Bravo ABS Plate
#C3BP5610000

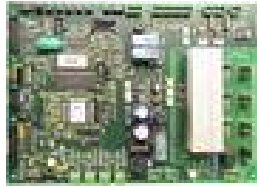

Bravo Basic Board (Troll)
#C5BM1400200
For Gam PCB see Gam to
Troll Conversion Kits



Bravo Cable Flat 9 Pin
#A2BP2380000

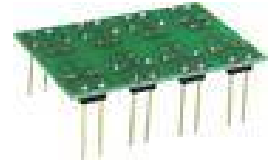

Bravo Calibration Board
#STXX0120000



Bravo Chamber Heater
Safety Thermostat TG400
#97730010



Bravo Chamber Thermo-
Insulating Kit 17
#A8BPR140000



Bravo Chamber Thermo-
Insulating Kit 21
#A8BGR010000



Bravo Chamber Thermo-
Insulating Panel Rear
#99950938



Bravo Door Fork
#C0BP060000Z



Bravo Door Fork Pin
#C0BP0500000



Bravo Door Locking
System Complete
#A1BP2710000



Bravo Door Locking
System Pin 3 x 25
#232A03L25K0



Bravo Door Motor
Complete
#95530349



Bravo Door Plate
Adjustment Screw
#C0BP079000N



Bravo Door Thermo-
Insulating Plate
#C4BP2740000



Bravo Drain Tubing
With Fitting 6 x 10
#95509115

Bravo Individual Parts Pictures



Bravo Facia Front Plastic
(Original Style)
#C3BP1830000



Bravo Fitting for
Chamber Filter
#99950911



Bravo Fitting CPC
Female Water
#260000006V0



Bravo Float Switch Clean
Water Reservoir Min-Max
#97668045



Bravo Float Switch Waste
Water Tank
#97668046



Bravo Foot Front PVC
#25600000400



Bravo Foot Rear Kit
#95509127




Bravo Fan
#40400030000



Bravo Keypad
#C6JP0010000




Bravo Label
Bravo 17
#C6JM0010000

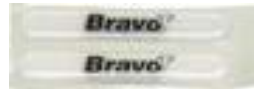


Bravo Label
Bravo 17V
#C6JP0020000



Bravo Label
Bravo 21V
#97781388

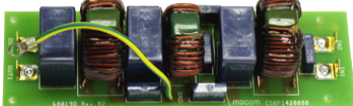

Bravo PCB Filter
Board GAM 120V
#C5BP4350000

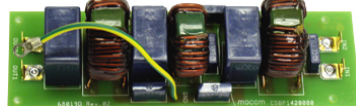

Bravo PCB Filter
Board GAM 220V
#C5BP1420000



Bravo Plate Ground
#99950119



Bravo Plug Plastic
#C3BP2790000

Bravo PCB Filter
Board Troll 120V & 220V
#C5BM1420000

Bravo Individual Parts Pictures



Bravo Power Switch
#97600130



Bravo Pressure Relief
Valve Only
#47000020000



Bravo Pressure Switch
#43100060000



Bravo Pressure Switch
Transducer Fitting
#97985500



Bravo Printer Cable Power
#A2BP4700000



Bravo Printer Cable Signal
#A2BP4690000



Bravo Printer Door
(Original Style)
#C3BP1840000



Bravo Printer Door
Pin 4 x 20
#230B04L20K0



Bravo Printer Paper
Thermal 57mm
#STXX0250000



Bravo Printer Thermal
#C5BP5500000



Bravo Reservoir Fill
Bottle 2500ml
#A0XP0010000



Bravo Vacuum Pump 8011
Shutter
#40000250000



Bravo Steam Generator
Heating Element 115V 850W
#41000170000



Bravo Steam Generator
Heating Element 220V 1000W
#41000260000



Bravo Steam Generator
O-ring
#48100080000

Bravo Individual Parts Pictures



Bravo Steam Generator
Lower Part
#COBP548000P



Bravo Steam Generator
Safety Thermostat TG400
#97730011



Bravo Steam Generator
Temperature Sensor Nut
#2860014000N



Bravo Steam Generator
Temperature Sensor Ferrule
#28600160000



Bravo Steam Generator
Upper Part
#COBP547000P



Bravo Temperature Sensor
PT1000-PT1
#43000190000



Bravo Temperature Sensor
PT1000-PT2/PT3/PT4
#43000180000



Bravo Transformer Only
GAM 120V
#97750175



Bravo Transformer Only
Troll 120V
#41200170000



Bravo Tray Handle
#STXX0080000



Bravo Vacuum Pump 8011
Large 21V/120V
40000420000



Bravo Vacuum Pump 8011
Large 21V/220V
#A9BP5270000

Bravo Transformer Only
GAM 220V
#41200080000

Bravo Transformer Only
Troll 220V
#41200110000



Bravo Vacuum Pump
Small Membrane
w/Screw
#SCI-902-02



Bravo Vacuum Pump Small
Shutter
#SCI-70-303



Bravo Valve EV Parker 3
Way 24V
#40100240000

Bravo Individual Parts Pictures



Bravo Valve EV1
Elbow Fitting
#97960018



Bravo Valve EV1 Support
#C1BP241000M



Bravo Valve EV
Parker 2 Way 24V
#40100370000



Bravo Valve EV
Parker 2 Way 24V
#40100230000




Bravo Valve EV4
Parker 3 Way 24V
#97720185



Bravo Valve EV5 One Way
Valve Connector
#47000010000



Bravo Vent Hole Nut
#266000007MN

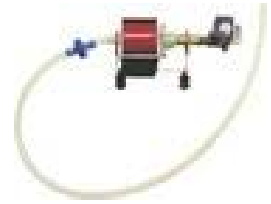


Bravo Water Pump Group
With Solenoid
#95509145



Bravo Water Pump
Mounting Bracket Foot
#25600000800

**Bravo Water Intake Pull Kit
#01-111774S**

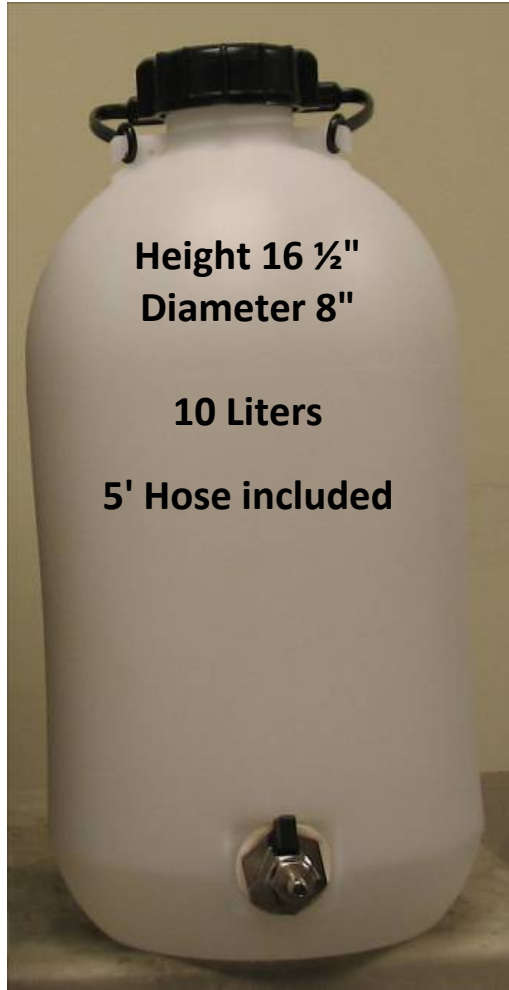


Height 16 ½"
Diameter 8"


10 Liters

5' Hose included

Bravo Direct to Drain Kit #01-111775S

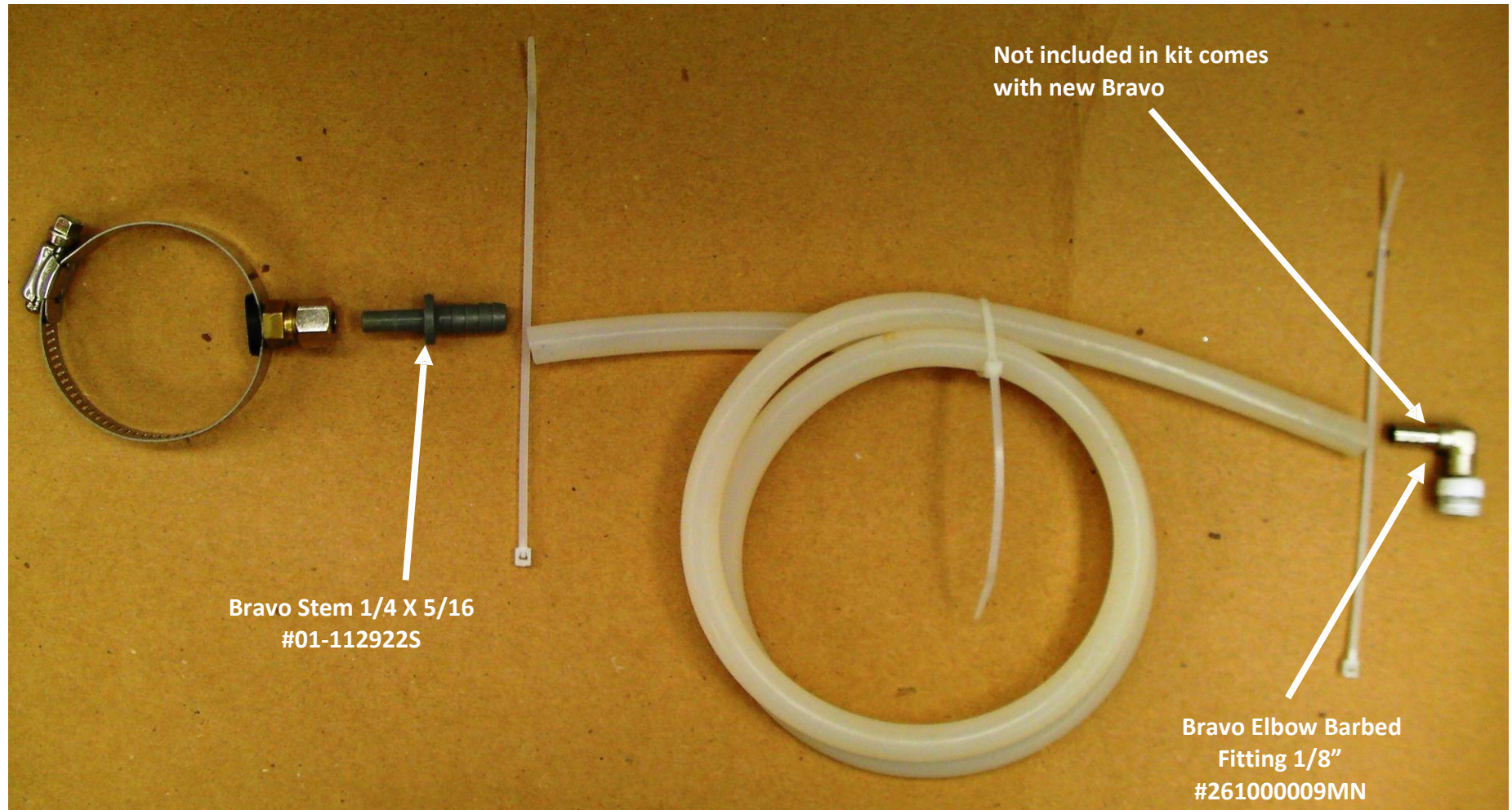


Quantim Front Door Closed

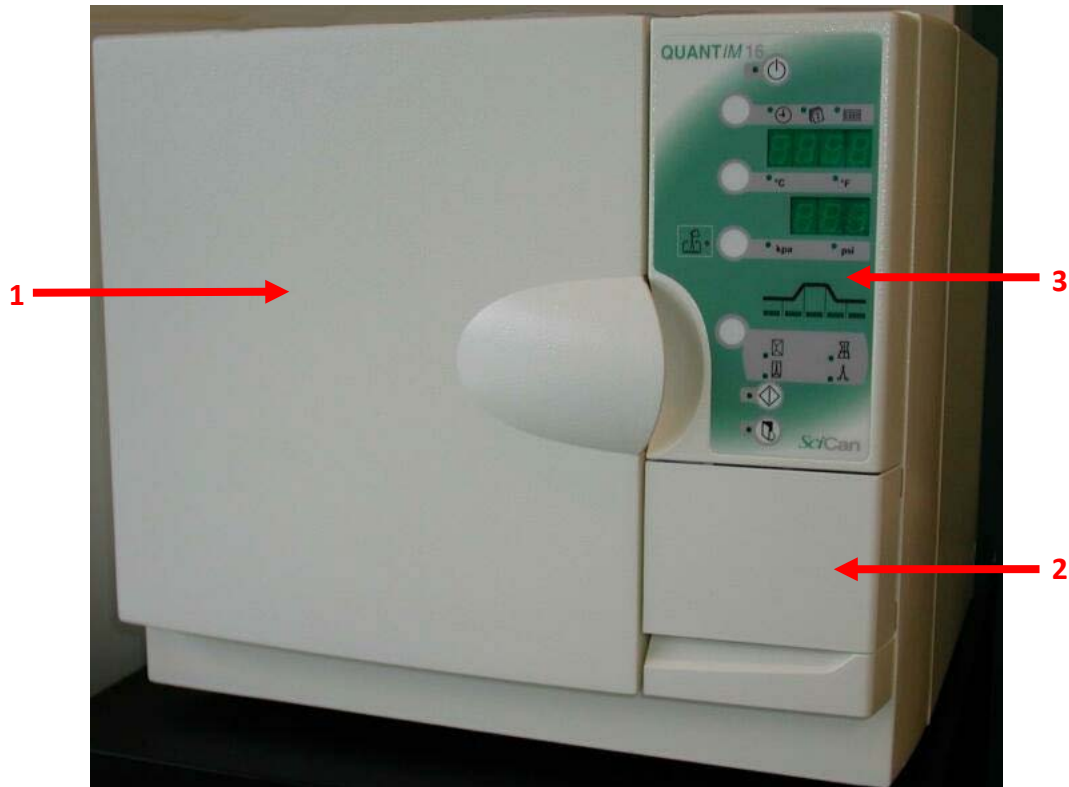


Item #	Part #	Description
1	289124	Door Molding Kit
2	289125	Printer Door w/Hinges
3	289162	Decal (no liquids cycle)
Not Visible	279159	Door Microswitch 2 Wire (Under Door Molding)
Not Visible	279494	Door Microswitch 3 Wire (Under Door Molding)
Note: Door Microswitch 2 Wire, wires are connected to switch terminals. Door Microswitch 3 Wire, switch terminals are connected to door PCB.		

Quantim Front Door Open

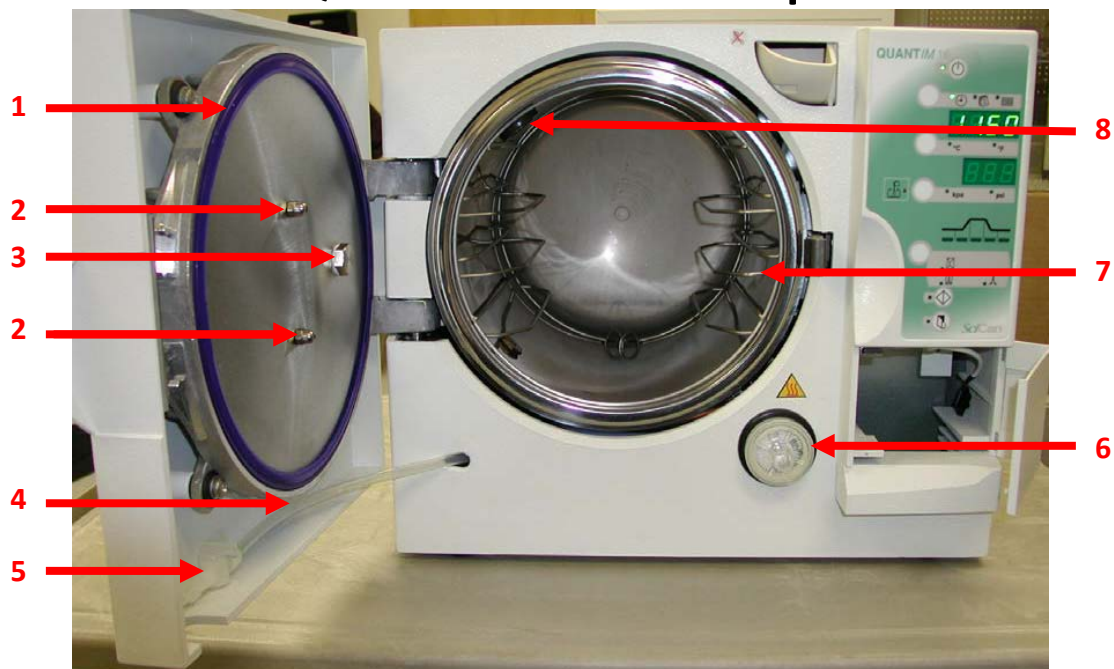


Item #	Part #	Description
1	279011	Door Gasket (Includes Door Plate Seals)
2	279490	Acorn Nut (Pkg of 2) (Pictured Above)
2	279491	Dome Nut (Pkg of 2) (Not Shown in Picture)
3	279107	O-ring Kit Entry Port White (Pkg of 2)
3	279467	Bolt & O-ring Entry Port
4	279527	Tubing & Fitting Kit
5	279102	Drain Tap
6	309077	Air Filter
7	289132	Rack for Chamber 3 Tray (Wire)
8	289006	Temperature Probe Chamber

Quantim Rear

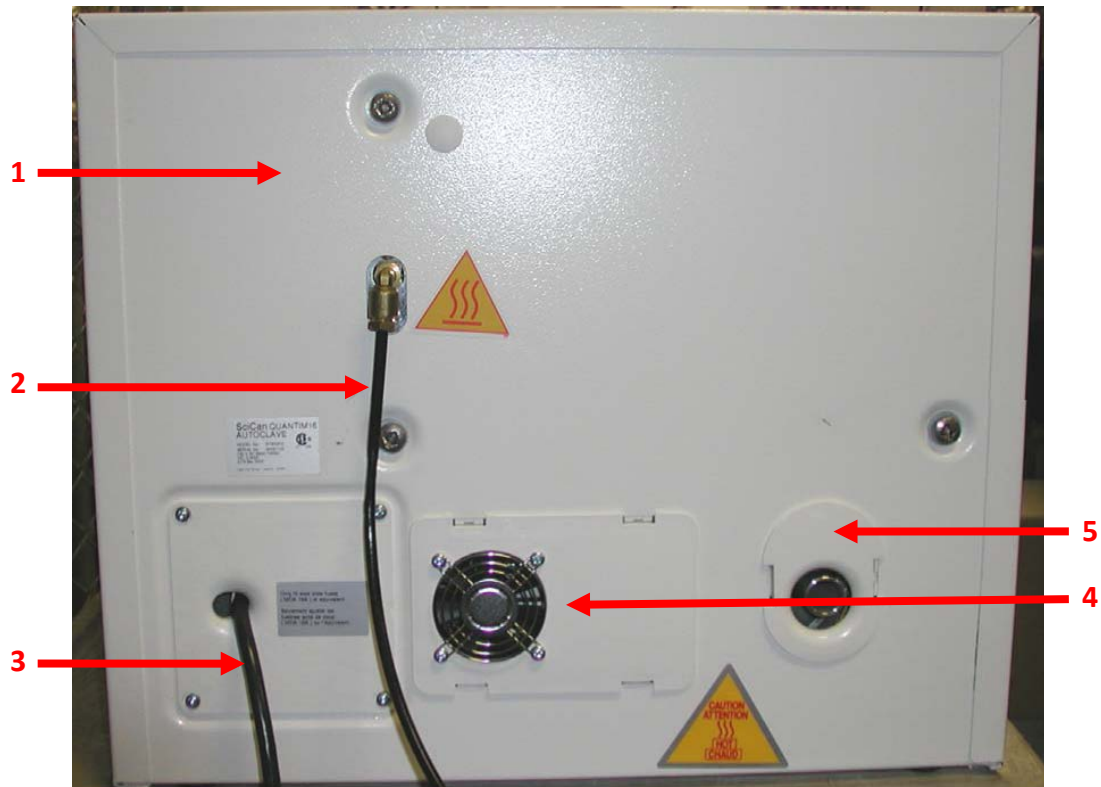


Item #	Part #	Description
1	319003	Cover w/Feet
2	279503	Waste Bottle Complete Quantim 16 (Bottle Not Shown)
3	319001	Power Cord
4	319014	Fan Assembly Rear
5	279469	Cover Circular

Quantim Right Side No Cover

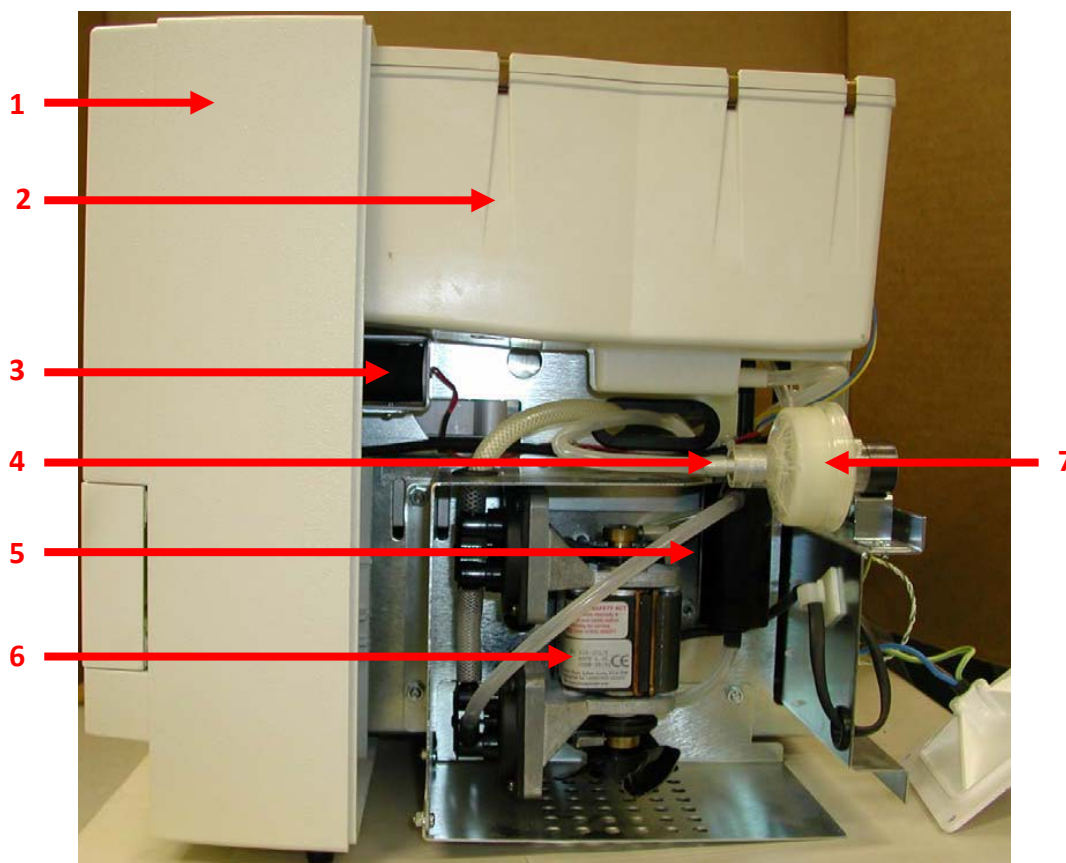


Item #	Part #	Description
1	279537	Front Bezel
2	309039	Reservoir Single Use (Quantim w/Waste Bottle)
2	279117	Reservoir Multi Use (Quantim w/No Waste Bottle)
3	279170	Door Opening Solenoid
4	299051	Air Filter Nozzle
5	289134	Silencer, Vacuum Pump
6	289118	Vacuum Pump
7	309077	Air Filter & Clip Replacement Kit

Quantim Front Right Side Behind Bezel

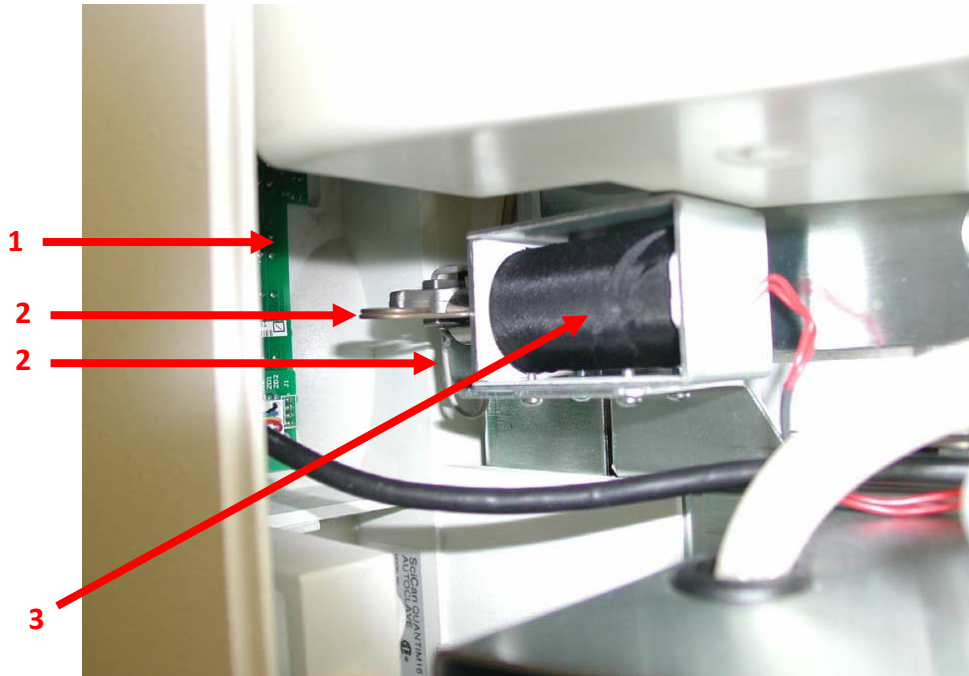


Item #	Part #	Description
1	279225	Display Module
2	279171	Door Solenoid Spring Kit (Pkg of 2)
3	279170	Door Opening Solenoid

Quantim Rear No Cover

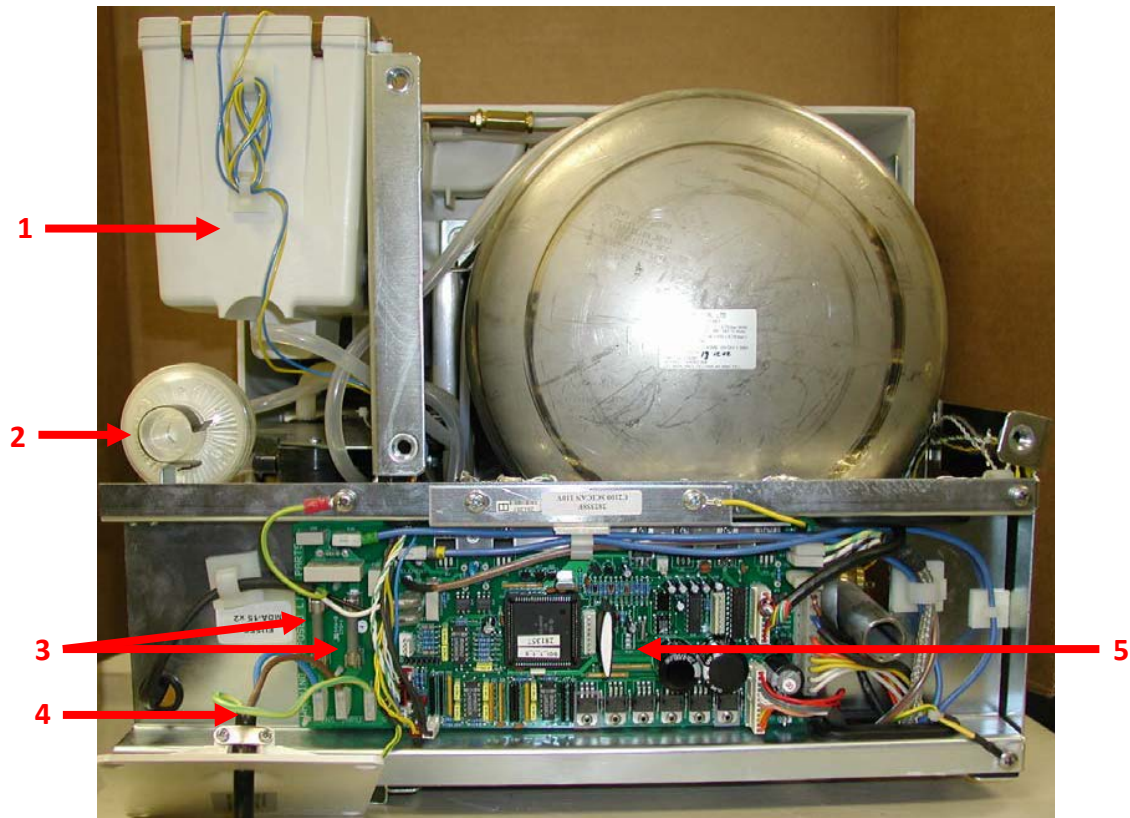


Item #	Part #	Description
1	309039	Reservoir Single Use (Quantim w/Waste Bottle)
1	279117	Reservoir Multi Use (Quantim w/No Waste Bottle)
2	309077	Air Filter & Clip Replacement Kit
3	319015	Fuse 15A
4	319001	Power Cord
5	319006	Control Module

Quantim Left Side Lower Front

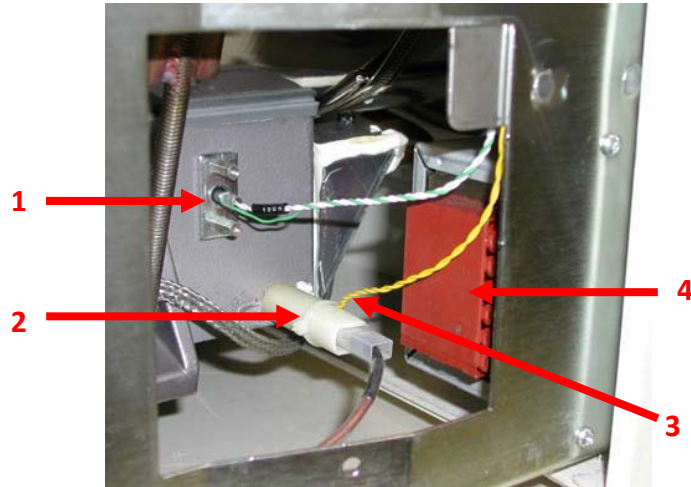

Item #	Part #	Description
1	289011	Temperature Probe Boiler
2	279155	Insulator Heating Element
3	279544	Boiler Thermistor
4	289136	Transformer


Quantim Left Side Lower Rear

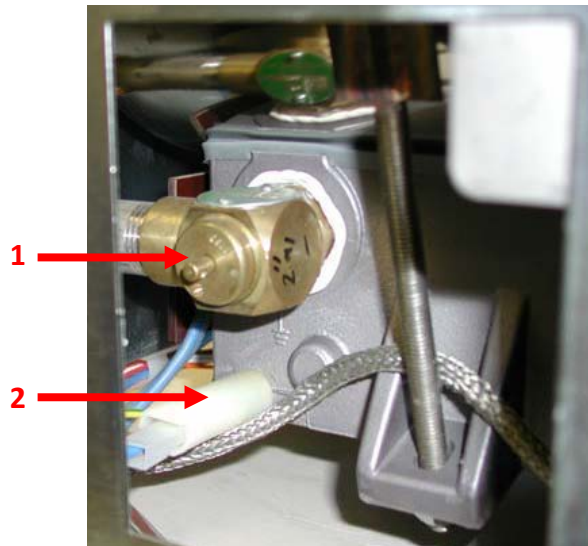

Item #	Part #	Description
1	279293	Pressure Relief Valve Kit***
2	279155	Insulator Heating Element


*****Note: Boiler must be removed to replace ASME valve. A new boiler to vessel seal (#319022) will need to be installed when reinstalling boiler.**

Quantim Left Side Upper Front

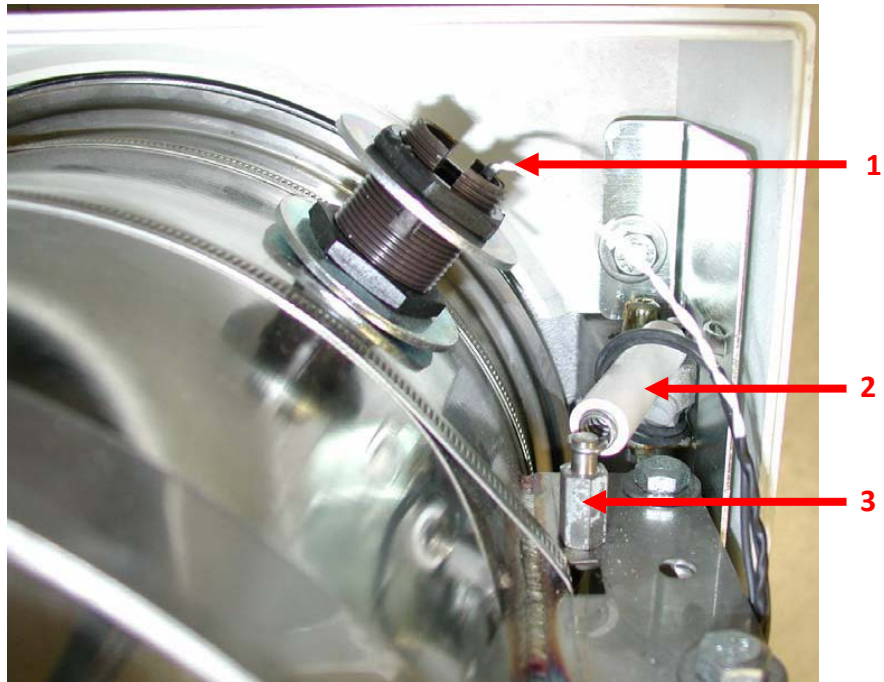

Item #	Part #	Description
1	289006	Temperature Probe Chamber
2	289015	Door Opening Spring
3	289028	Door Spring Pillars (Pkg of 2)

Quantim Bottom View

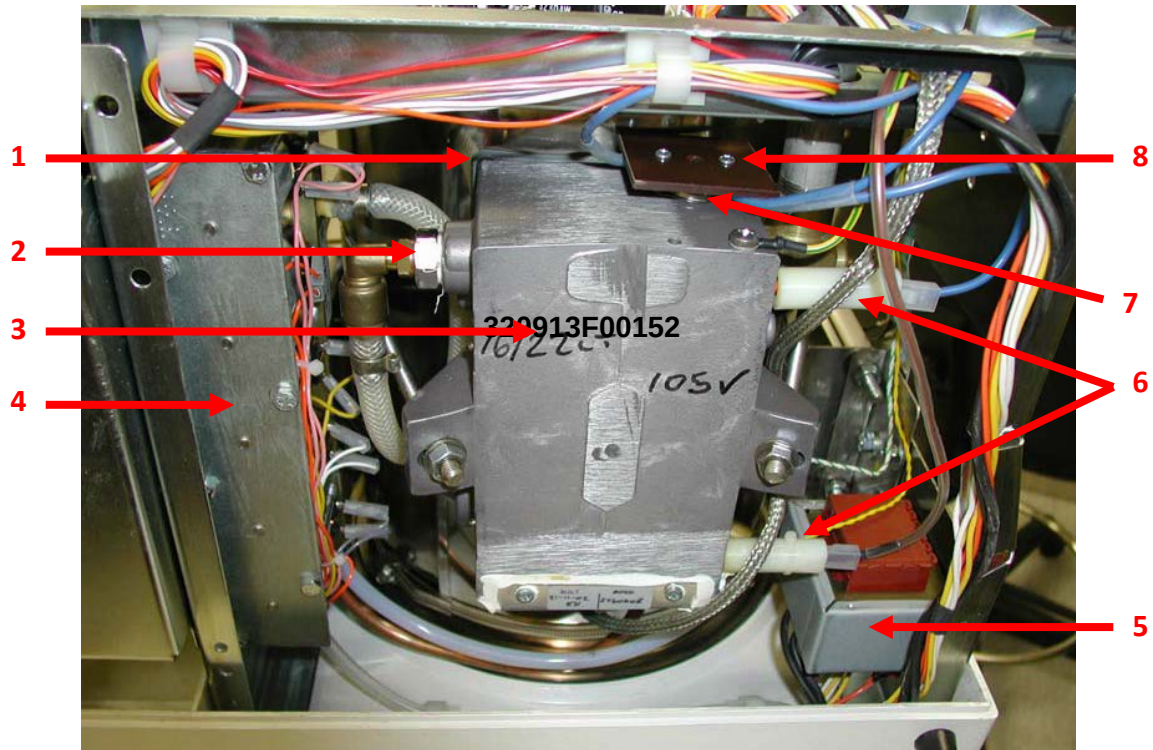


Item #	Part #	Description
1	319022	Seal, Boiler to Vessel (between boiler & vessel)
2	279176	Water Filter
3	319024	Boiler w/PRV
4	319019	Valve Block Kit
5	289136	Transformer
6	279155	Insulator, Heating Element
7	279266	Thermal Cutout
8	279203	Thermal Cutout Mounting Kit
Not Visible	289117	Heater Pad

Hydrim C51W w/Push Buttons Front View Door Closed




Item #	Part #	Description
1	01-107972S	Cover, Top (no vents)
1	01-110205S	Cover, Top (with vents)
1	01-112016S	Cover, Top (serial #210110C00001 & higher)
2	01-107974S	Door Facia
3	01-107935S	Chemical Door w/Hinge (HD01BJ0001 or higher)
3	01-107933S	Chemical Door (HD01Blxxxx or lower)
4	01-108917S	Facia Control Panel
4	01-109147S	Facia Control Panel w/Dryer (HD01FK0001 & higher)
5	01-108790S	Door Actuator Kit
Not Visible	01-108924S	Screw Kit (Cover Screws)
Not Visible	01-107975S	Adjustable Feet (Pkg of 4)
Not Visible	01-107789S	Drain Hose
Not Visible	01-107788S	Inlet Hose (Hot or Cold)

Hydrim C51W w/Push Buttons Front View Door Open

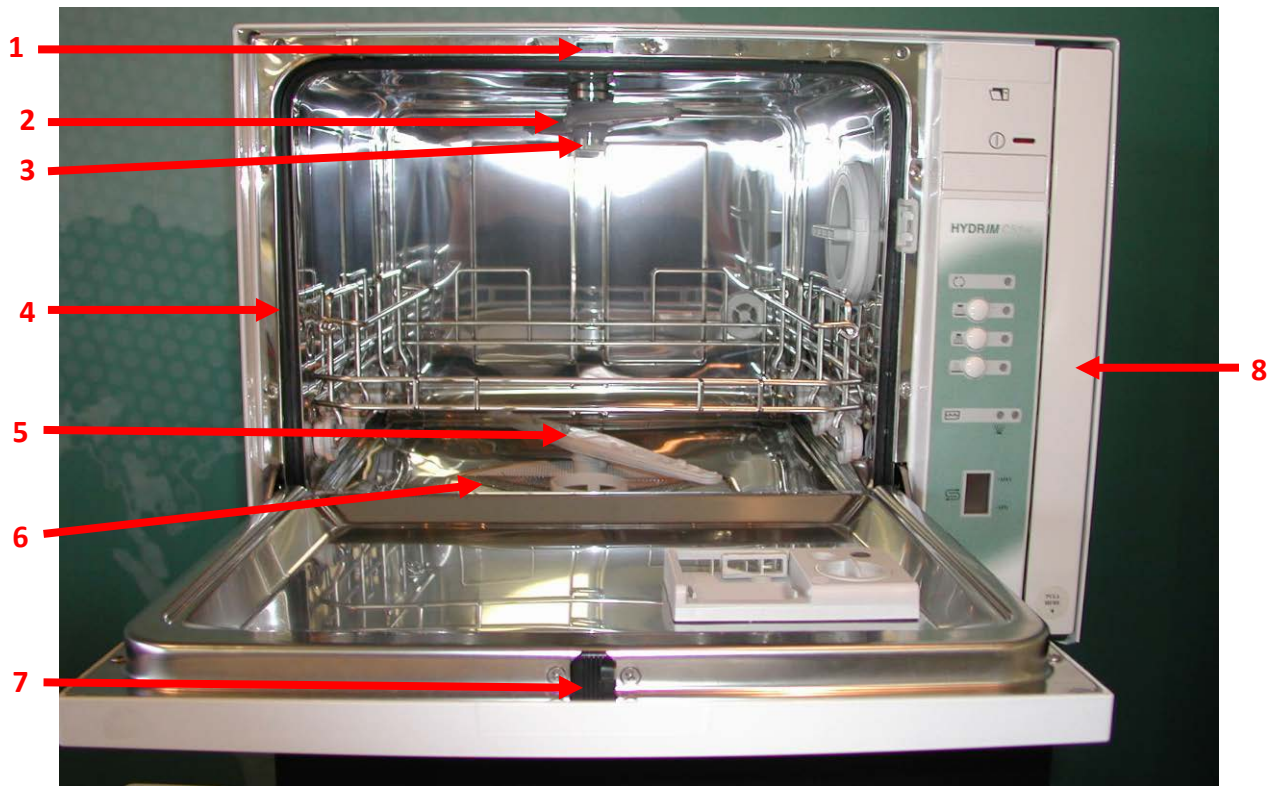


Item #	Part #	Description
1	01-107811S	Door Spring Kit
2	01-107805S	Wash Arm Upper
3	01-108154S	Plug Upper Arm
4	01-107786S	Seal Door
5	01-107804S	Wash Arm Lower
6	01-107806S	Drain Screen
7	01-107812S	Door Latch
8	01-107935S	Chemical Door w/Hinge (HD01BJ0001 or higher)
8	01-107933S	Chemical Door (HD01BI9999 or lower)
Not Visible	01-108090S	Seal Kit Door Bottom
Not Visible	01-107807S	Drain Filter


Hydrim C51W w/Push Buttons Right Side

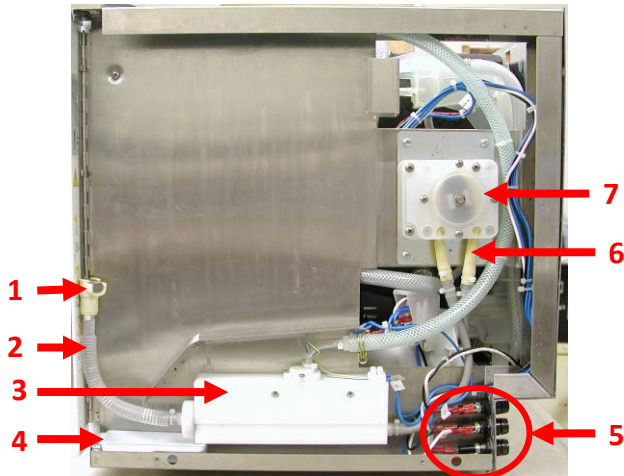


HD01DF9999 & Older

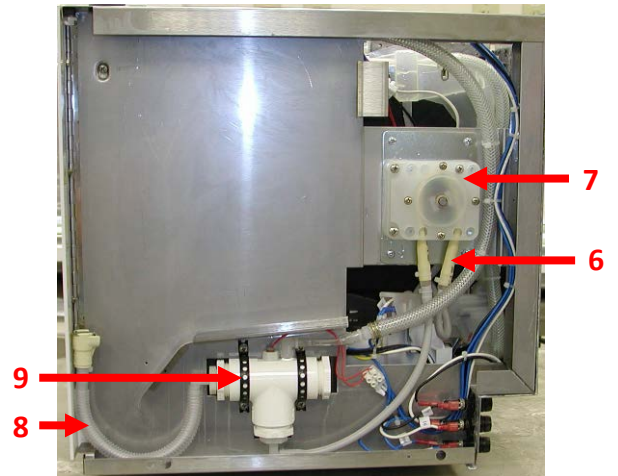

HD01DG0001 to HD01FJ9999

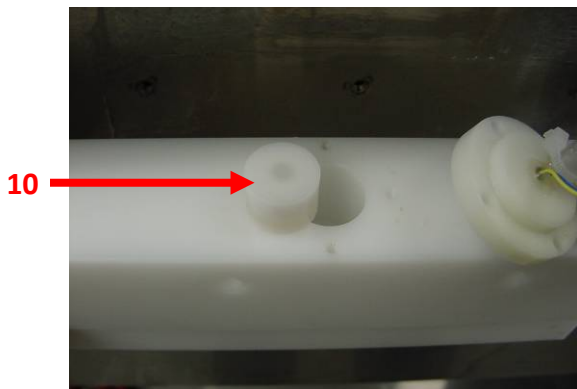



Old Style Dosing Reservoir

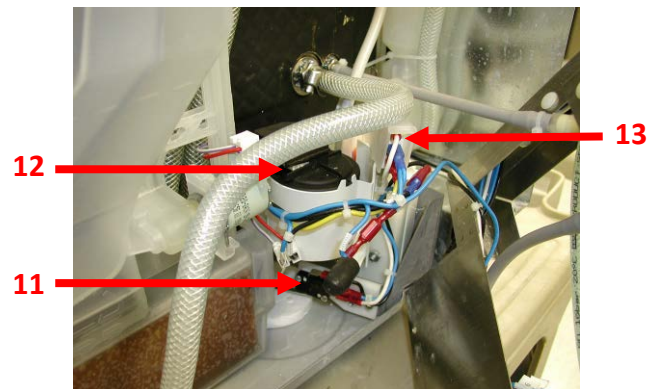

View behind Right Side Panel


Item #	Part #	Description
1	01-108030S	Quick Connect Female
2	01-108031S	Tube Chemical (HD01DF9999 & lower)
2	01-108699S	Tube Chemical (HD01DG0001 & higher)
3	01-108122S	Drip Tray
4	01-109058S	Dosing Reservoir Upgrade Kit (Old Style Reservoir not available)
5	01-107790S	Dosing Pump
6	01-108700S	Fuse Holders (Pkg. 3)
Not Visible	01-107800S	Fuse 10 Amp (Pkg. 10)
Not Visible	01-107799S	Fuse 2 Amp (Pkg. 5)
7	01-108699S	Tube Chemical (HD01DG0001 & higher)
8	01-108253S	Dosing Reservoir New Style
9	01-109909S	Tube Dosing Pump Internal
10	01-108351S	Float Dosing Reservoir
11	01-107938S	Switch Pressure Assembly
12	01-107803S	Switch Overflow Chamber
13	01-107937S	Switch Overflow Bottom

Hydrim C51W w/Push Buttons
Right Side w/Dryer
Inside Chamber w/Dryer
Serial #HD01FK0001 & Newer

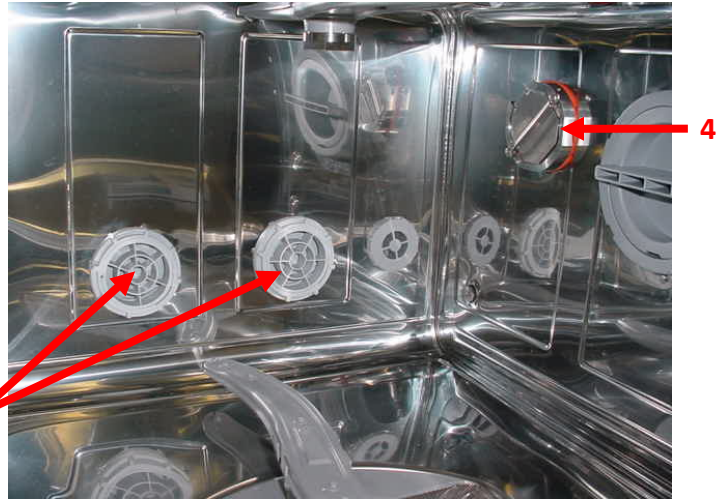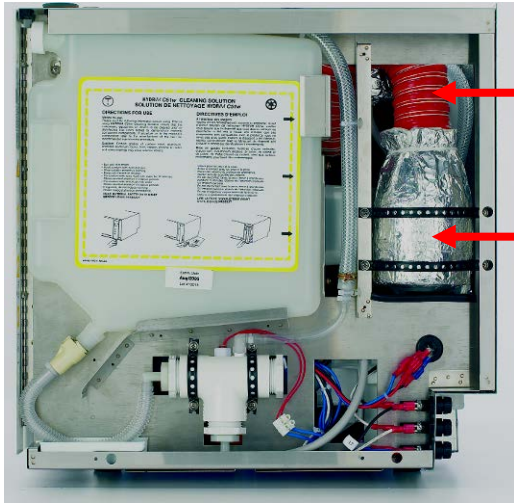

Item #	Part #	Description
1	01-109143S	Dryer Tubing
2	01-109142S	Dryer Assembly
3	01-109145S	Vent Assembly Dryer
4	01-109144S	Dryer Fitting

Hydrim C51W w/Push Buttons Bottom View

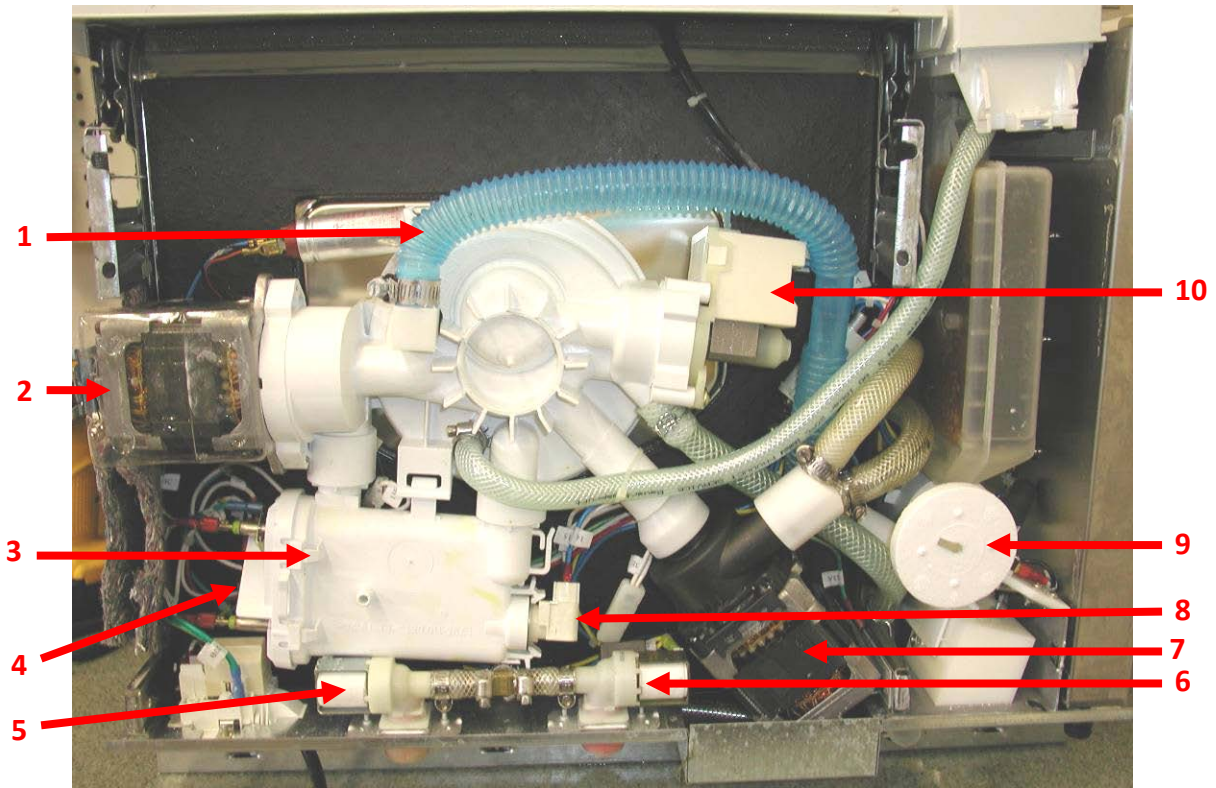

Item #	Part #	Description
1	01-108701S	Tubing (blue) Chamber
2	01-107794S	Recirculation Pump Lower Wash Arm
3	01-107808S	Water Heater
4	01-107810S	Pressure Switch, Water Heater
5	01-107815S	Valve Hot Water
6	01-107801S	Valve Cold Water
7	01-107795S	Recirculation Pump Upper Wash Arm
8	01-107809S	Thermostat Water Heater (NC Sensor)
9	01-111619S	Styrofoam Float Overflow Bottom
10	01-107797S	Waste Pump

Note: See next page for upper and lower pump impellers and upper pump Y fittings with stainless steel barbs.

Hydrim C51w Spare Parts Update

06-TSB-230

Please be advised of the new Hydrim C51w spare parts.

The Impellers (items # 4&5) have been set-up separately due to cost concerns of changing circulation pumps. The Y-fittings have been upgraded and are now made of stainless steel. The part numbers for the items have remained the same.



1. Stainless Steel Fitting For Y's
p/n 01-109590S (Package of 2)

2. Y-Fitting, Upper Arm Pump
p/n 01-109303S

3. Y- Fitting, Upper Arm
p/n 01-108702S



4. Impeller - Upper
p/n 01-109447S



5. Impeller - lower
p/n 01-109448S

Hydrim C51W Bur Screen Part #01-111828S



Hydrim C51W w/Touchscreen Accessories

1. Baskets only


Part Name	Order No.	Part Dimensions
Statim 2000 basket (2 included with Hydrim)	01-107240	275 mm x 178 mm x 32 mm 10 3/4 x 7 x 1 5/8"
Statim 5000 basket (Optional)	01-107241	373 mm x 178 mm x 42 mm 13 1/2 x 7 x 1 5/8"



2. Basket with hinged lid (optional)

Order No.:
01-108294


Dimensions: 216 mm x 140 mm x 54 mm
8 1/2 x 5 1/2 x 2 1/8"

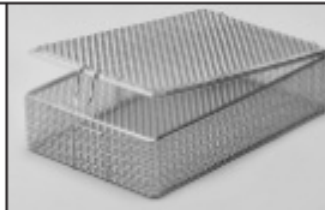

5. Large basket (optional)

Basket Order No.:
01-108210

Dimensions:
350 mm x 368 mm x 64 mm
14 1/2 x 13 3/4 x 2 1/2"

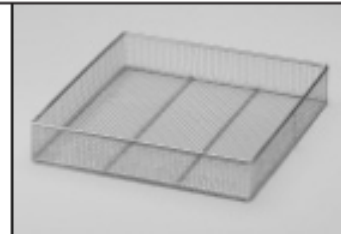

3. Four cassette rack (optional)

Order No.:
01-109153



Support Rack for Large Basket Order No.:
01-108214

Dimensions: 350mm x 350mm x 48mm
13 3/4 x 13 3/4 x 1 7/8"

4. Hygiene basket (optional)

Order No.:
01-108298

Dimensions: 203mm x 114 mm x 32 mm
8 x 4 1/2 x 1 1/4"




6. Rack for two baskets and two cassettes (optional)

Order No.:
01-109768S



Hydrim C51W w/Touchscreen

Front View

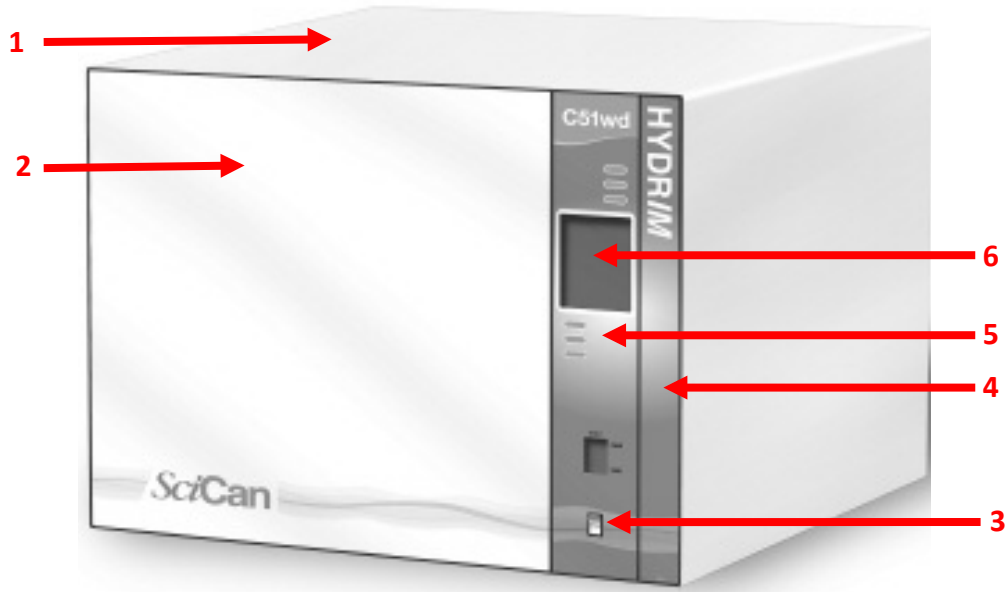


Item #	Part #	Description
1	01-112450S	Cover, Top
2	01-110201S	Door Facia w/Label
3	01-100573S	Power Switch
4	01-110200S	Detergent Door w/Label
5	01-110202S	Facia Controller w/Label
6	01-109785S	Touch Display
Not Shown	01-108924S	Screw Kit (Cover Screws)
Not Shown	01-107975S	Adjustable Feet (Pkg of 4)

Hydrim C51W w/Touchscreen Front View Door Open w/Trolley



Item #	Part #	Description
1	01-109894S	Trolley
2	01-107806S	Drain Filter Course
3	01-108309S	Kickplate
Not Shown	01-107807S	Drain Filter Fine (under Drain Filter Course)
Not Shown	01-108310S	Drain Mesh (inside Drain Filter Course)


Hydrim C51W w/Touchscreen Front View Door Open No Trolley

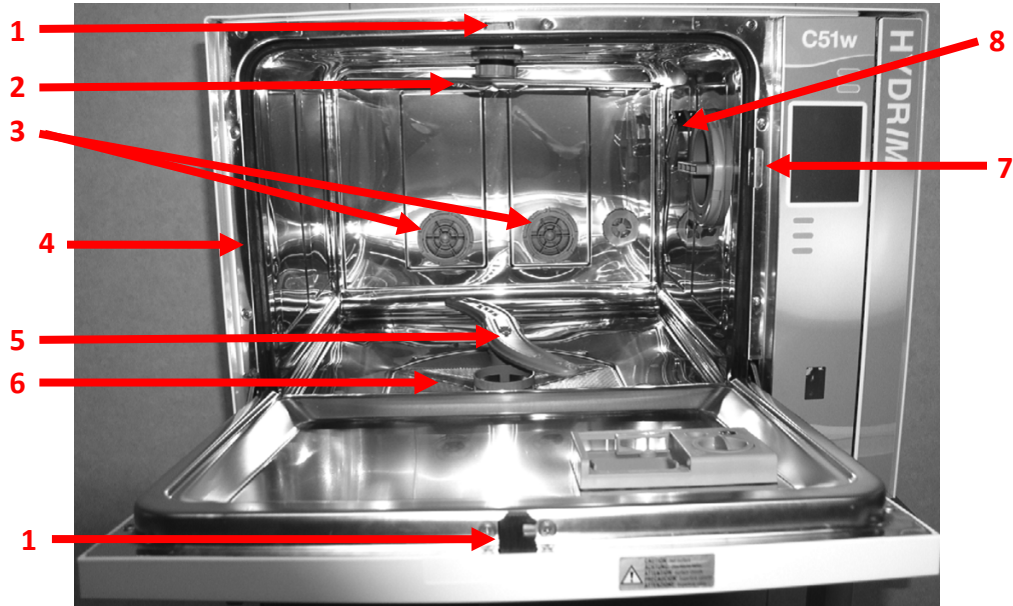

Item #	Part #	Description
1	01-111441S	Door Latch Assembly
2	01-109790S	Upper Wash Arm
3	01-109145S	Dryer Vent Assembly (Pkg of 2)
4	01-107786S	Door Seal
Not Visible	01-108090S	Door Seal Kit Bottom
5	01-107804S	Lower Wash Arm
6	01-107806S	Screen Drain
Not Visible	01-108310S	Mesh Drain Screen (small round mesh screen fits in center of Screen Drain)
Not Visible	01-107807S	Filter Drain (sits under Screen Drain)
Not Visible	01-111828S	Screen Assembly Sump
7	01-108795S	Plastic Trim Edge
8	01-109144S	Dryer Fitting


Hydrim C51W w/Touchscreen

Rear View

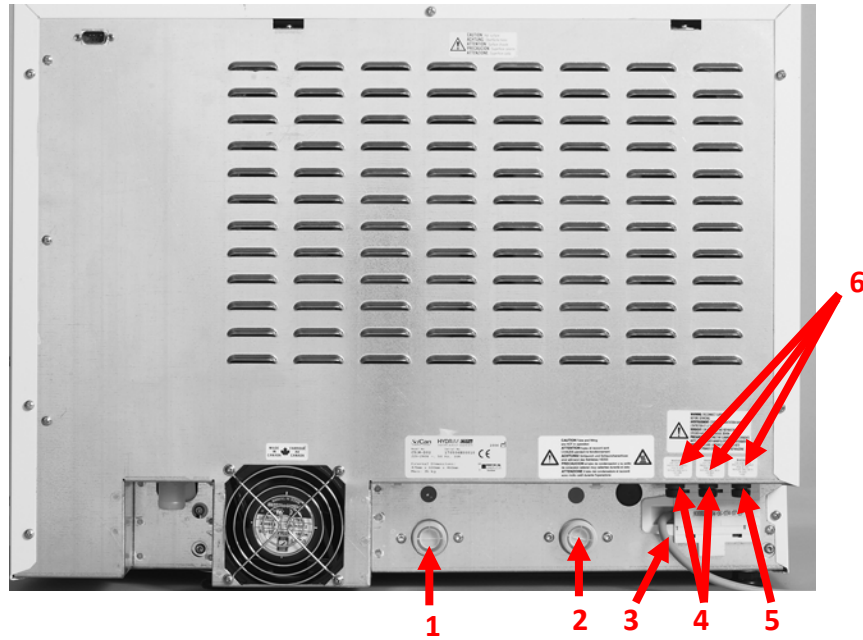


Item #	Part #	Description
1	01-107801S	Inlet Valve, Cold Water
2	01-107815S	Inlet Valve, Hot Water
3	01-109305S	Power Cord
4	01-107800S	Fuse 10A (Pkg of 10)
5	01-107799S	Fuse 2A (Pkg of 5)
6	01-108700S	Fuse Holder (Pkg of 3)
Not Shown	01-107789S	Drain Hose
Not Shown	01-107788S	Inlet Hose (Hot or Cold)

Hydrim C51W w/Touchscreen Right Side No Cover

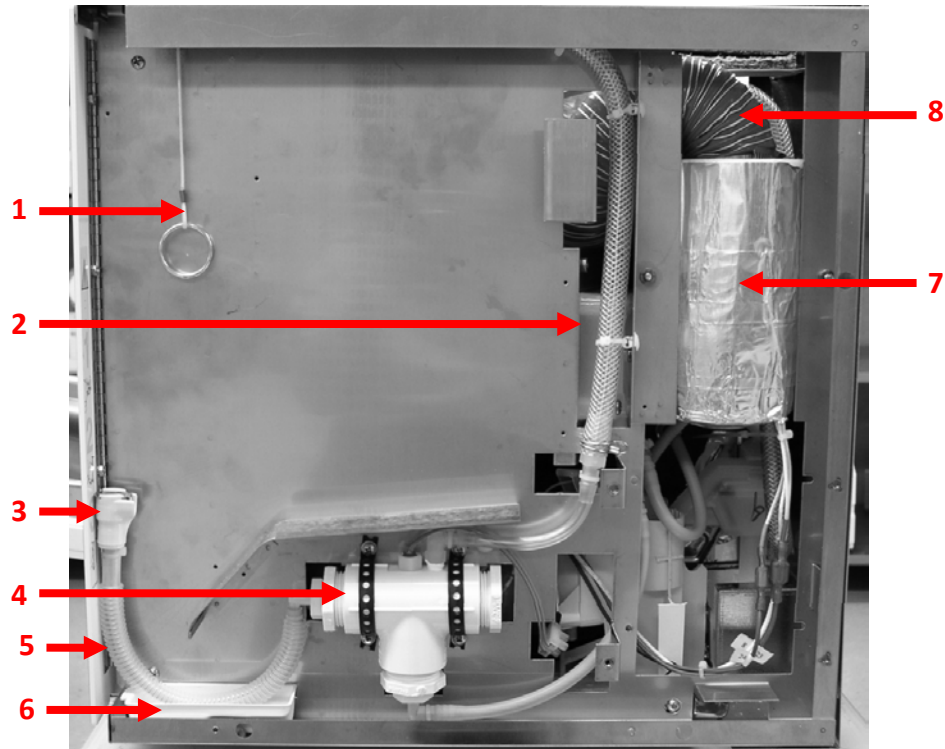


Item #	Part #	Description
1	01-109789S	Rope, Door Latch
2	01-107790S	Dosing Pump
2	01-111848S	Dosing Pump (Serial #210110C00001 & higher)
Not Visible	01-109909S	Dosing Pump Tubing, Internal (Black)
3	01-108030S	Quick Disconnect Female
4	01-108253S	Reservoir, Chemical
5	01-108699S	Tube, Chemical
6	01-108122S	Drip Tray
7	01-109835S	Dryer Assembly
8	01-109143S	Dryer Tubing

Hydrim C51W w/Touchscreen Right Side Behind Chemical Bracket

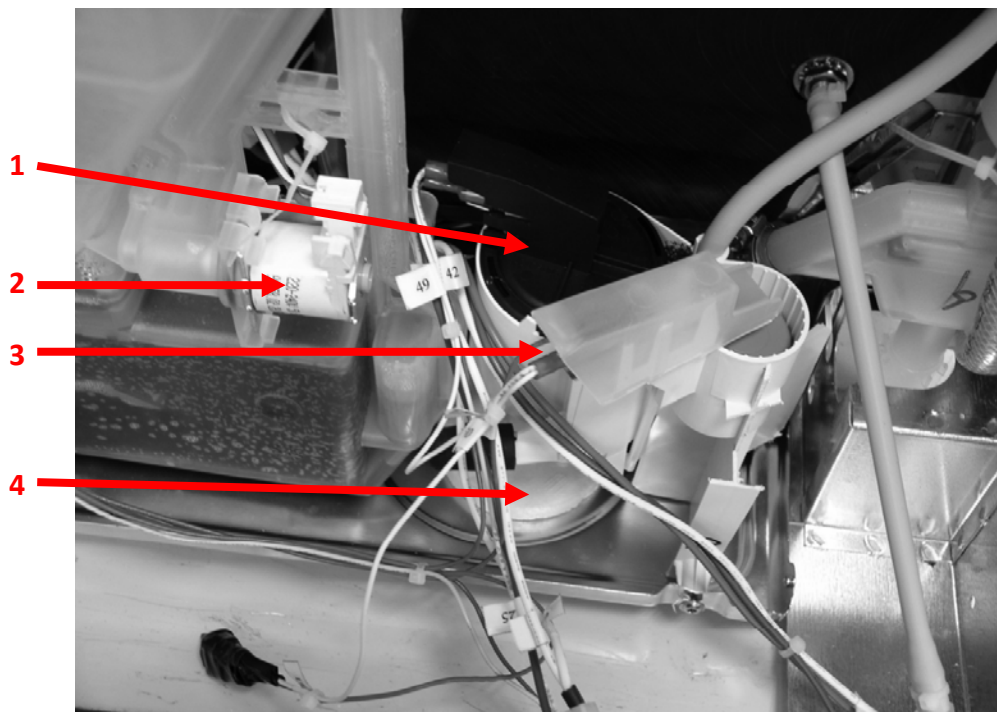


Item #	Part #	Description
1	01-107791S	Valve Salt Regeneration
2	01-107938S	Full Switch Assembly
3	01-107803S	Overflow Switch
4	01-111619S	Styrofoam Float (Overflow Bottom)

Hydrim C51W w/Touchscreen Left Side No Cover

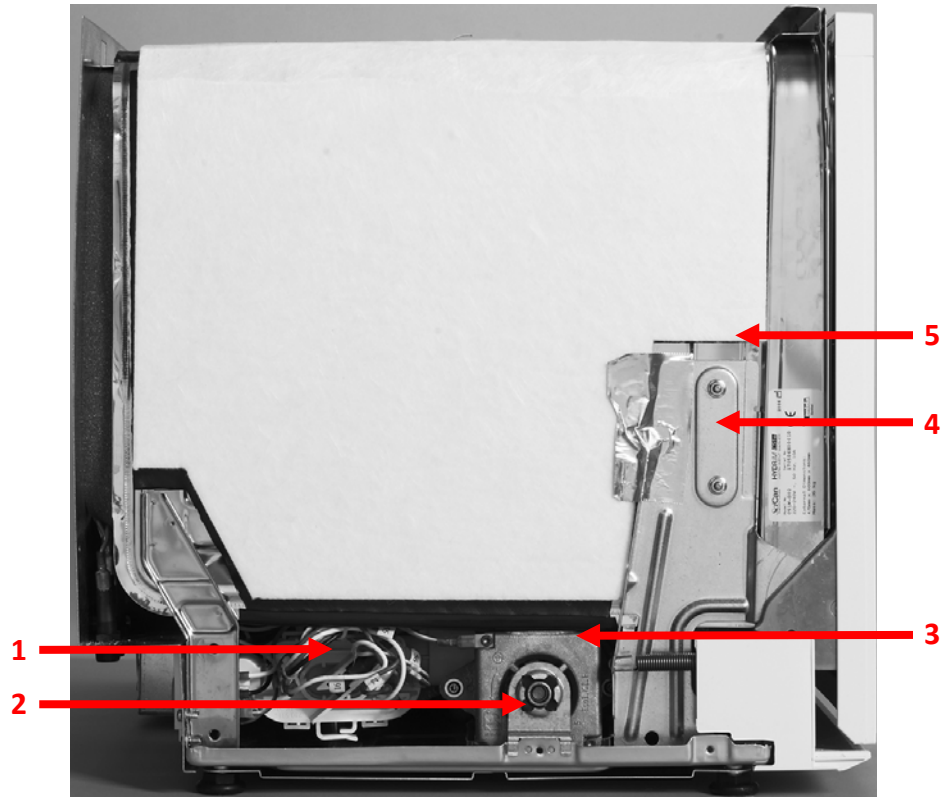


Item #	Part #	Description
1	01-107808S	Water Heater
2	01-108797S	Support Bracket Lower Pump
3	01-107794S	Recirculation Pump Lower Arm
4	16943.04	Frame Left Hand Side
Not Visible	16943.03	Frame Right Hand Side
5	16943.06	Joint Left Hand Side
Not Visible	16943.05	Joint Right Hand Side

Hydrim C51W w/Touchscreen Top No Cover

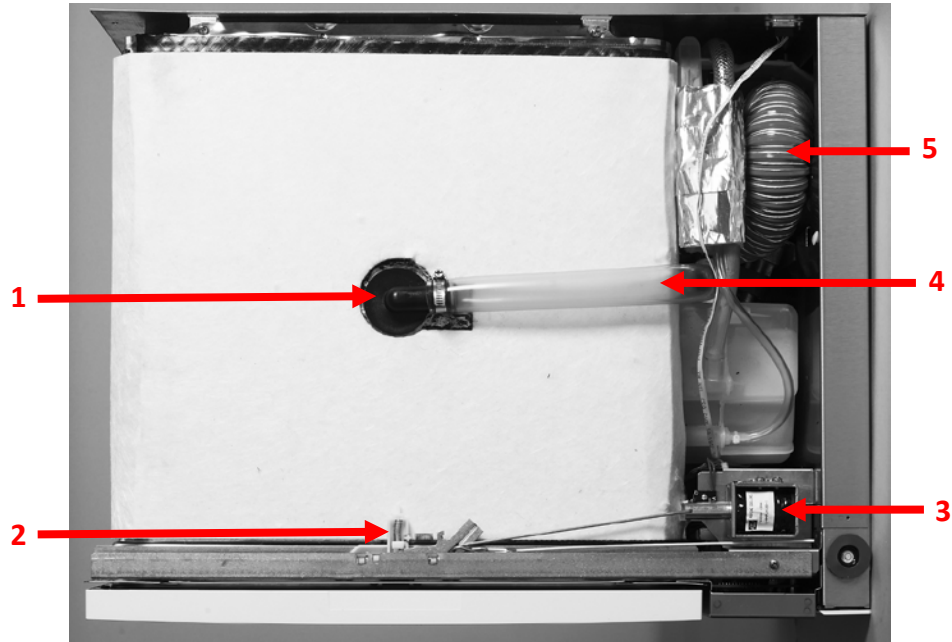


Item #	Part #	Description
1	01-109776S	Upper Wash Arm Fitting
2	01-111441S	Door Latch Assembly
3	01-109780S	Door Latch Solenoid
4	01-110204S	Tubing Upper Wash Arm Fitting
5	01-109143S	Dryer Tubing

Hydrim C51W w/Touchscreen Behind Controller Facia

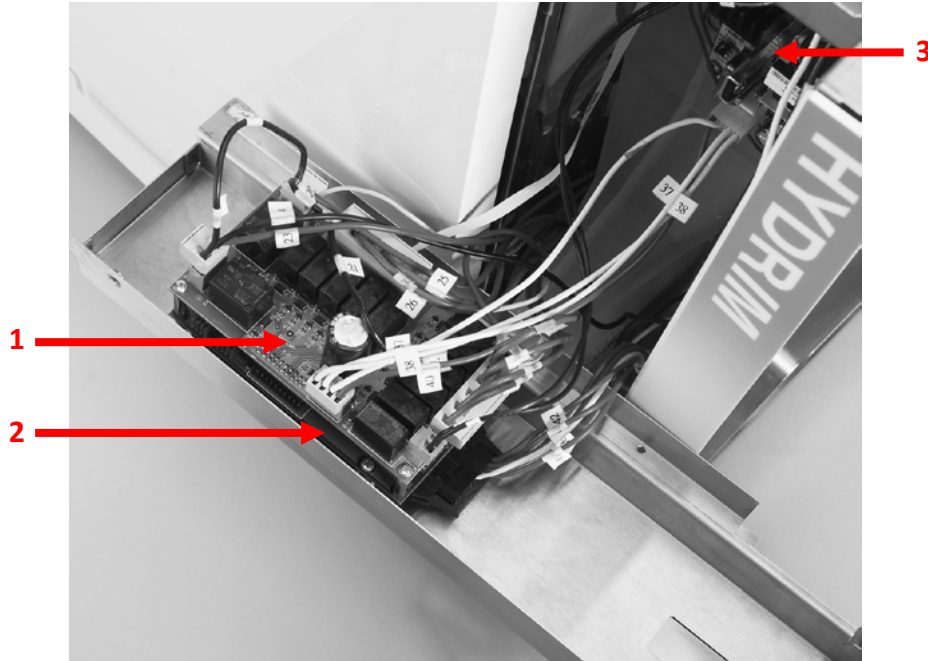

Item #	Part #	Description
1	01-110206S	Logic Board C51W
2	01-109783S	I/O Board
3	01-109782S	Power Supply Board
Not Visible	01-109785S	Touch Display

Hydrim C51W w/Touchscreen Bottom No Cover

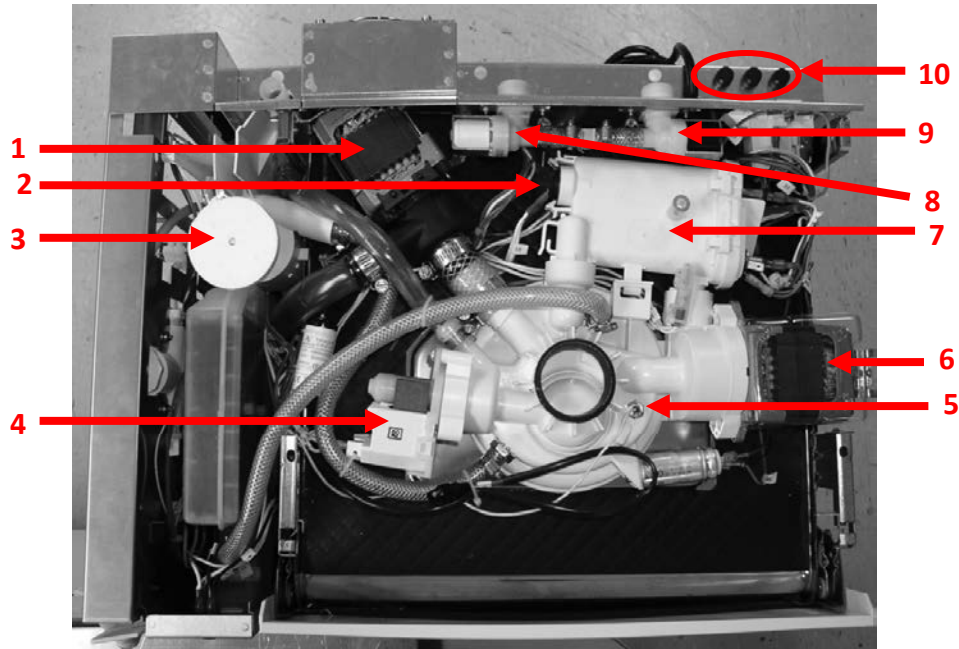

Item #	Part #	Description
1	01-110259S	Pump Upper Wash Arm without Fitting
2	01-107809S	NC Sensor (Water Heater)
3	01-111619S	Styrofoam Float Overflow Bottom
4	01-107797S	Waste Pump
5	01-110203S	Single Temperature Sensor C51W
6	01-107794S	Recirculation Pump Lower Wash Arm
7	01-107808S	Water Heater
8	01-107801S	Inlet Valve Cold Water
9	01-107815S	Inlet Valve Hot Water
10	01-108700S	Fuse Holder (Pkg of 3)

Hydrim C51W w/Touchscreen

Individual Parts



01-107786S
Seal Door, J



01-107788S
Inlet Hose N.A., J



01-107789S
Drain Hose, J



01-107790S
Dosing Pump, J



01-107791S
Valve Salt Regeneration, J



01-107794S
Pump Recirculation Lower Arm, J



01-110259S
Pump Upper Arm w/o
Fitting, J



01-107797S
Drain Pump N.A. 230V, 60Hz, J



01-107799S
2A Fuse, J



01-107800S
10A Fuse, J



01-107801S
Inlet Valve Cold Water, J



01-107803S
Switch Overflow Chamber, J



01-107804S
Lower Wash Arm, J



01-107806S
Screen Drain, J



01-107807S
Filter Drain, J



01-107808S
Water Heater, J



01-107810S
Switch Pressure Heater, J



01-107812S
Clip Door, J



01-107815S
Inlet Valve Hot Water, J



01-107938S
Switch, Pressure Assembly
(Chamber Full Switch Assembly)

Hydrim C51W w/Touchscreen

Individual Parts



01-107802S
Switch Full Chamber, J



01-107975S
Adjustable Feet, J



01-108030S
Quick Connect Female, J



01-108699S
Tube, chemical



01-108121S
Cap Quick Disconnect, J



01-108122S
Drip Tray, J



01-108253S
Reservoir



01-108309S
Kickplate, J



01-108310S
Mesh Drain, J



01-108351S
Float Dosing
Reservoir, J



01-108700S
Fuse Holder, J



01-108795S
Plastic Trim Edge, J



01-108797S
Support Bracket 1-st Pump, J



01-109143S
Dryer Tubing, J



01-109144S
Dryer Fitting, J



01-109145S
Dryer Vent Assy, J



01-109776S
Top Arm Fitting Hydrim, S



01-110204S
Tubing, Upper Arm Fitting, C51W, J

Hydrim C51W w/Touchscreen

Individual Parts



01-109780S
Door Solenoid C51WD, S



01-110202S
Fascia Controller w/
Label, C51W, J



01-109782S
Power Supply Board, S



01-110206S
Logic Board Hydrim, C51W, J



01-109783S
I/O Board Hydrim, S



01-109785S
Touch Display Hydrim, S



01-110203S
Single Temperature
Sensor C51W, J



01-109789S
Rope, Door Latch, C51WD, S



01-109790S
Upper Spray Arm
Hydrim, S



01-110201S
Fascia Door w. Label, C51W, J



01-110200S
Detergent Door w. Label
C51W, J



01-107809S
NC Sensor



01-110205S
Cover Top, C51W, J



01-107973S
Packaging Hydrim
C51W, S



01-109834S
Seals, Door Bottom Hydrim, S



01-109835S
Dryer Assy Hydrim, S

Hydrim C61 G4 Accessories

Baskets



STAT/M 2000 basket
01-107240



STAT/M 5000 basket
01-107241



Large basket
01-113545



Hygiene Basket
01-113546



Basket with hinged lid
01-113547

Racks (Trolley and baskets not included)



Six cassette rack
01-113251



Three cassette, two basket rack
01-113253



Two basket rack
01-113257

Rack for Vertical and Hinged Instruments

Requires support from a basket



Rack for vertical instruments
01-110410S



Rack for Hinged instruments
01-110409S



Ten tray, two basket rack
01-113254

Hydrim C61 G4 Front Right View




Item #	Part #	Description
1	01-113286S	Cover Stainless Top
2	01-113294S	Cover Stainless Door
3	01-113292S	Kick Plate Stainless
4	01-113290S	Cover Stainless Right Side
5	01-113314S	Decal Door
6	01-113261S	Cable USB
Not Shown	01-113288S	Cover Stainless Left Side

Hydrim C61 G4 Front View Door Open with Trolley & Cassette Rack



Item #	Part #	Description
1	01-113256	Trolley Kit
2	01-113333S	Wheel Set
3	01-113282S	Chemical Carrier
4	01-113393S	Hose Chemical Connection
Not Shown	01-113257	Trolley Center Support


Hydrim C61 G4 Front View Door Open no Trolley

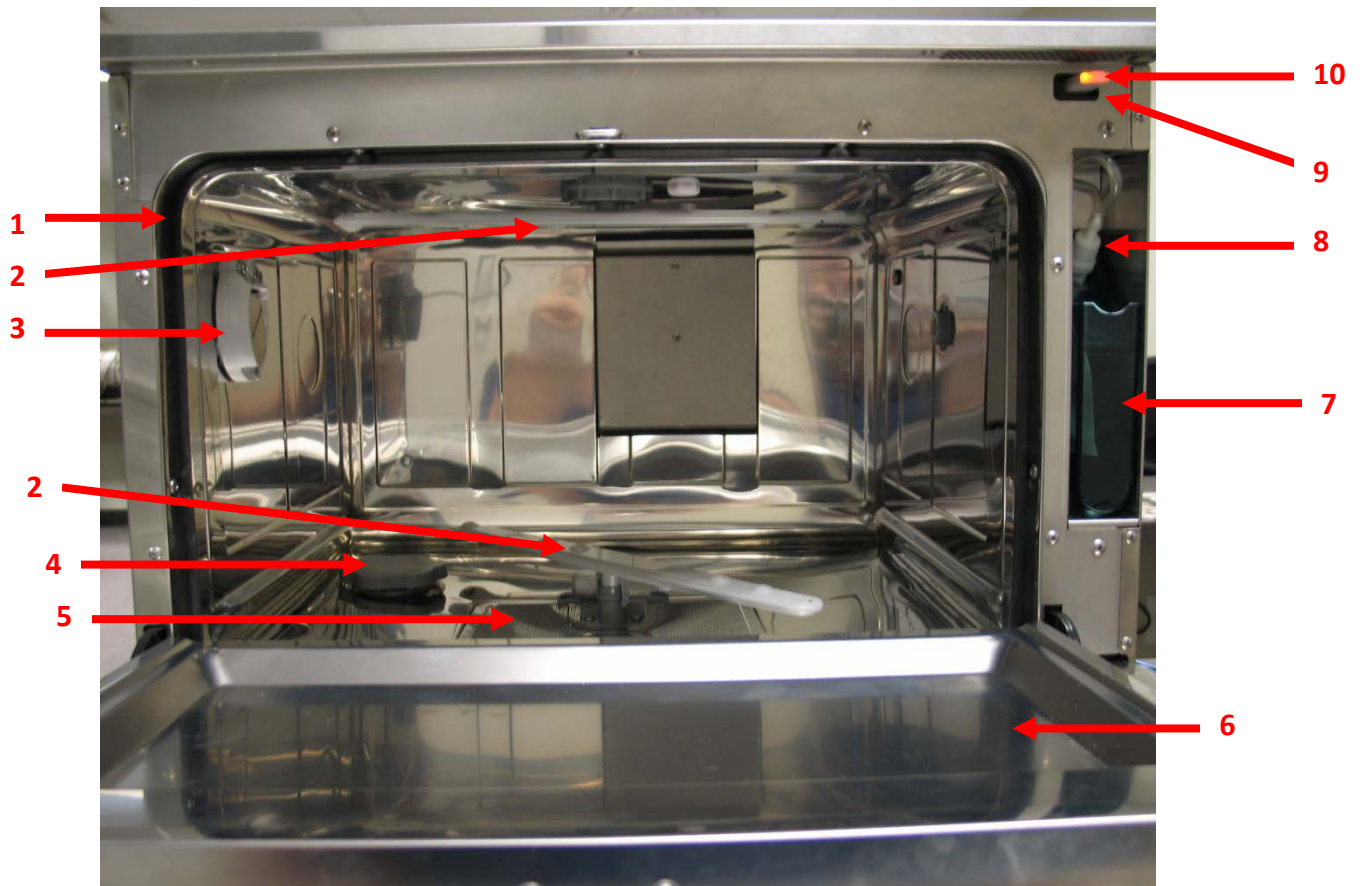

Item #	Part #	Description
1	01-107786S	Seal Door
2	01-113302S	Wash Arm
3	01-113356S	Deflector Assembly (Dryer Outlet Cover)
4	01-113321S	Cap Softener
5	01-113332S	Strainer Assembly
6	01-113299S	Inner Door Panel
7	01-113282S	Chemical Carrier
8	01-113393S	Hose & Connection Cap for Chemical
9	01-113261S	Cable USB
10	USB-STICK2	USB Stick
Not Shown	01-113387S	Fixed Screen, Sump
Not Shown	01-113271S	Temperature Sensor (Chamber)
Not Shown	01-113661S	Seal Door Bottom

Hydrim C61 G4


Front View Kick Plate Open

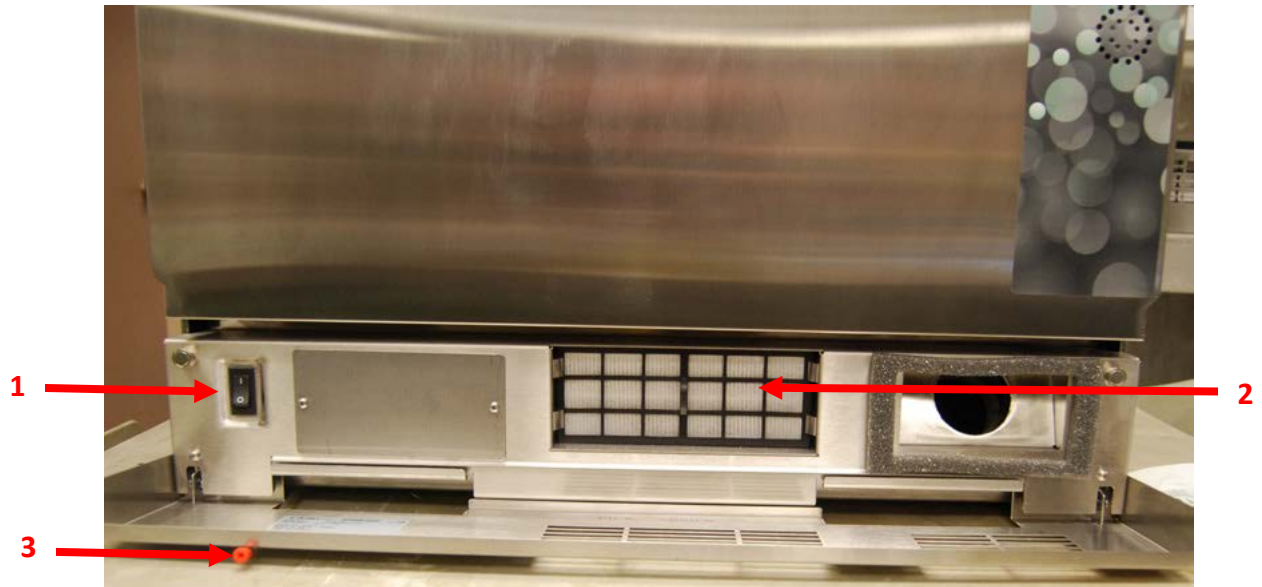

Item #	Part #	Description
1	01-112024S	Power Switch
2	01-113277S	Filter Air
3	01-104343S	Plug for Drain Tubing
Not Shown	01-113262S	Gasket Air Filter
Not Shown	01-113327S	Switch w/ Roller for Air Filter

Hydrim C61 G4


Front View No Door Cover

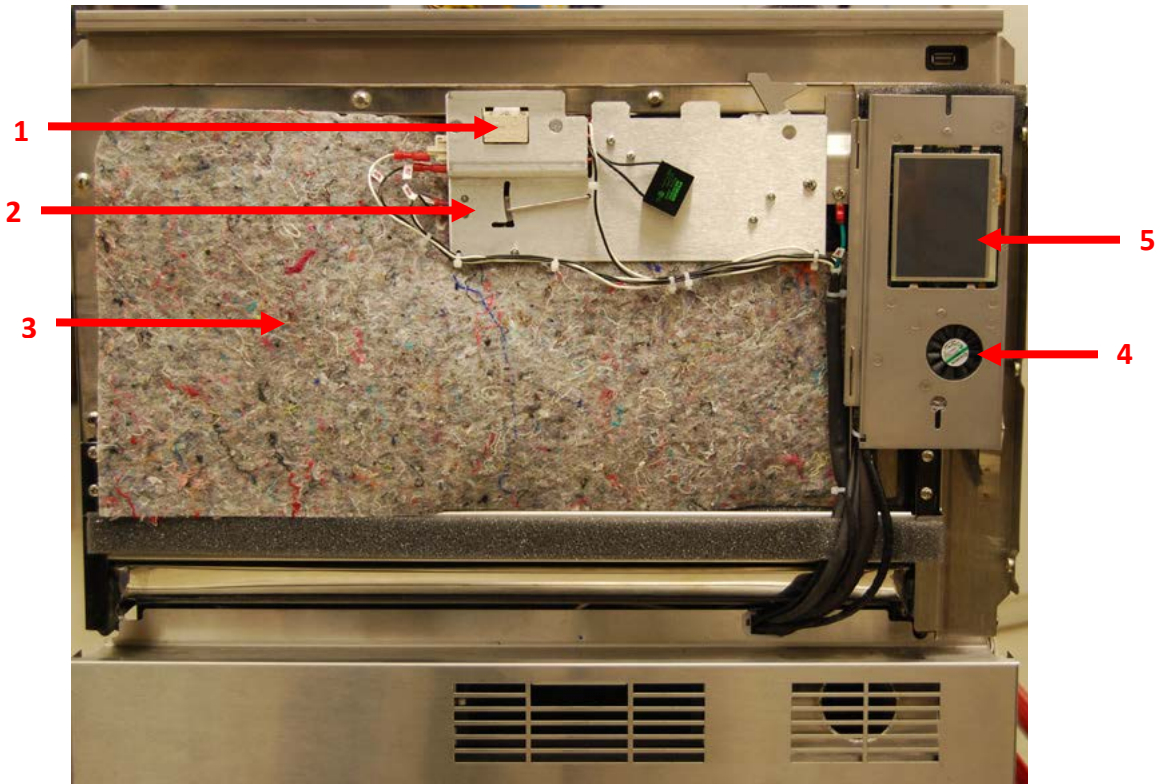

Item #	Part #	Description
1	01-113322S	Door Latch
2	01-113951S	Door Latch Kit (includes plate)
3	01-113263S	Insulation Door
4	01-113284S	Blower 5V LCD
5	01-113311S	LCD Assembly
Not Shown	01-113605S	Door Latch Microswitch (behind latch plate)
Not Shown	01-113319S	Door Latch Solenoid (behind latch plate)
Not Shown	01-113682S	Speaker Assembly (behind LCD plate)
Not Shown	01-113391S	LCD Controller

Hydrim C61 G4


Left Side No Cover

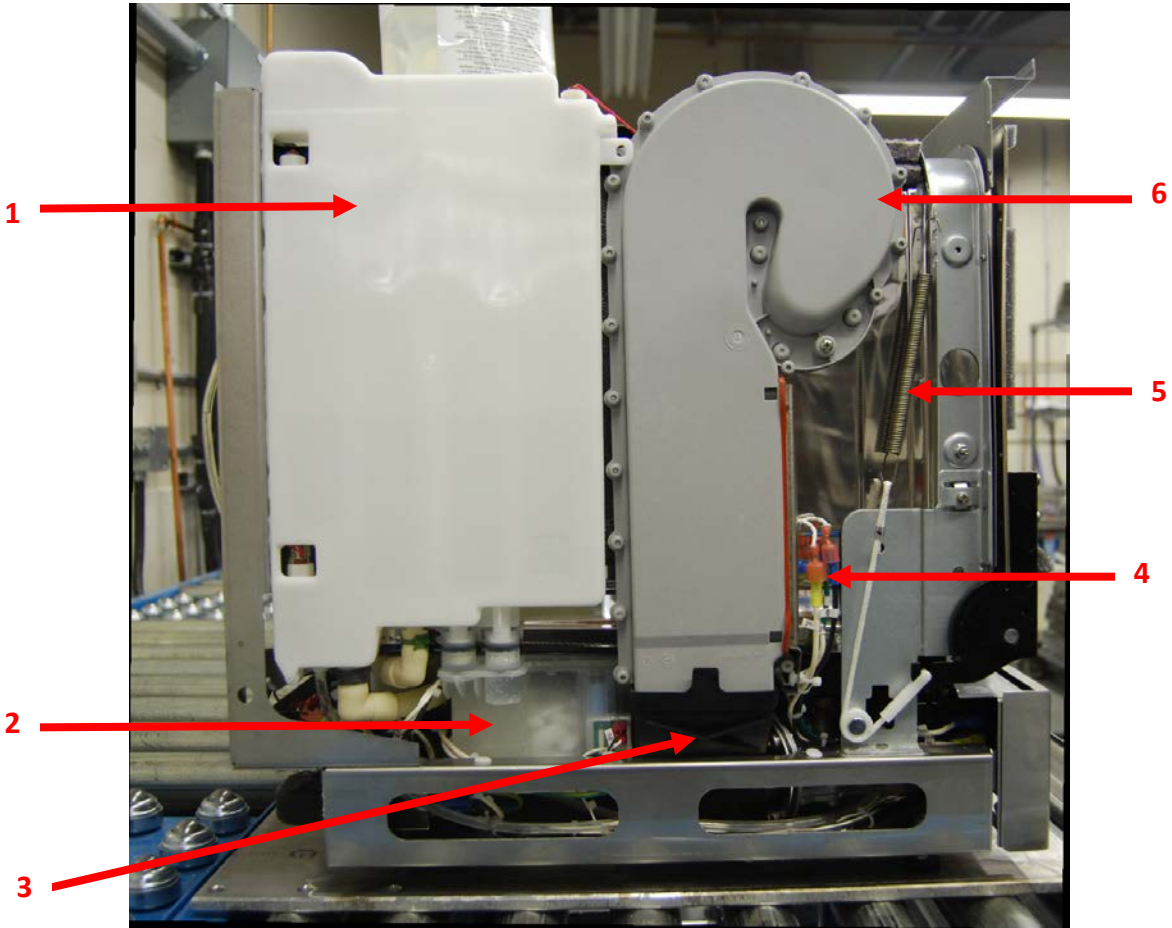

Item #	Part #	Description
1	01-113318S	Backflow Preventer (Air Gap)
2	01-113325S	Softener Assembly
3	01-113313S	Duct, Elbow (Air Dryer duct)
4	01-113273S	Heater Air
5	01-113298S	Door Spring Kit
6	01-113324S	Duct, Vertical (complete with heater)

Hydrim C61 G4


Right Side No Cover

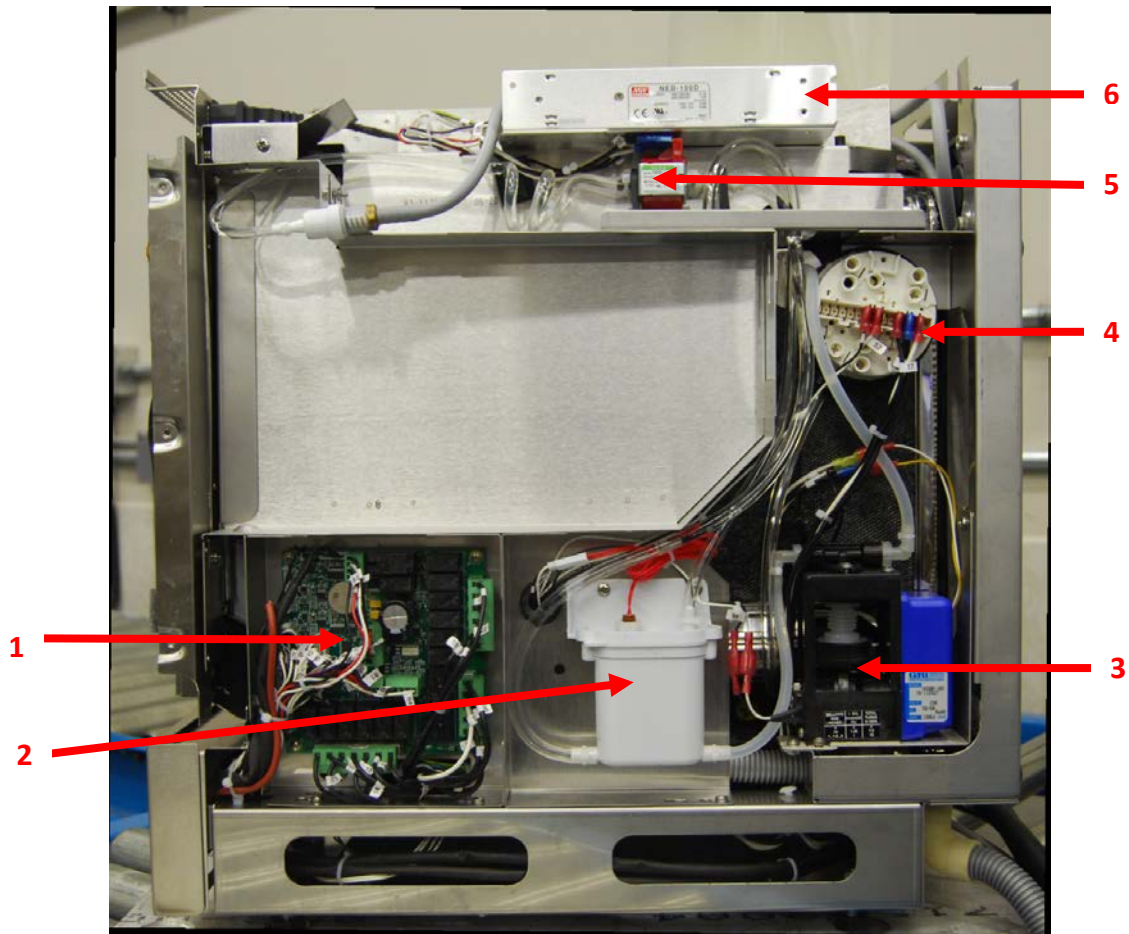

Item #	Part #	Description
1	01-113310S	IO PCB
2	01-113323S	Reservoir Assembly (for chemical)
3	01-113306S	Pump Bellows (includes microswitch)
4	01-113265S	2-Level Pressure Switch
5	01-113307S	Pump Chemical (fills dosing reservoir)
6	01-113266S	Power Supply 5V/24V

Hydrim C61 G4


Rear View

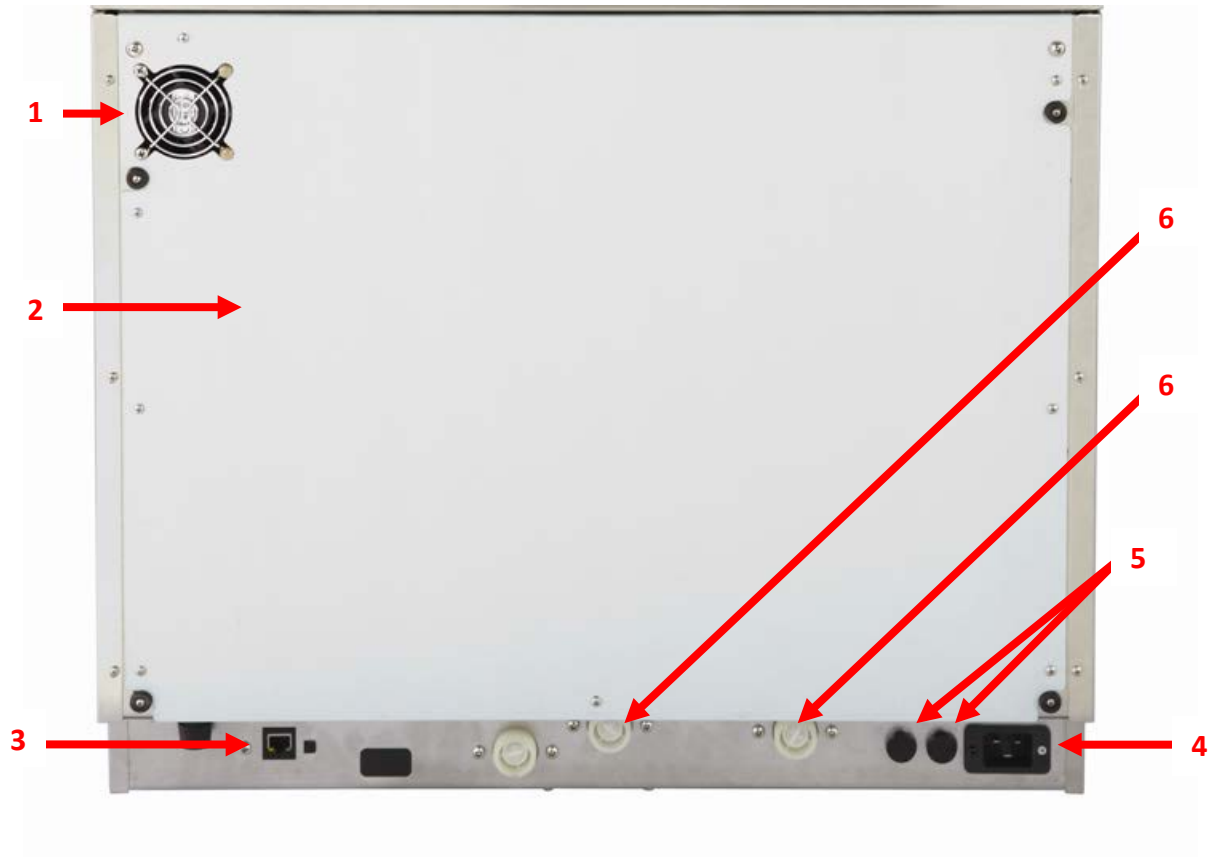

Item #	Part #	Description
1	01-113329S	Fan 24V Power Source
2	01-113297S	Cover Rear
3	01-113259S	Cable Ethernet
4	01-110505S	Line Filter
5	01-112545S	Fuse Holder (Qty. 2)
6	01-113330S	Valve Inlet (hot & cold water)
Not Shown	01-103472S	Fuse 15A (Qty. 2)

Hydrim C61 G4


Rear View No Cover

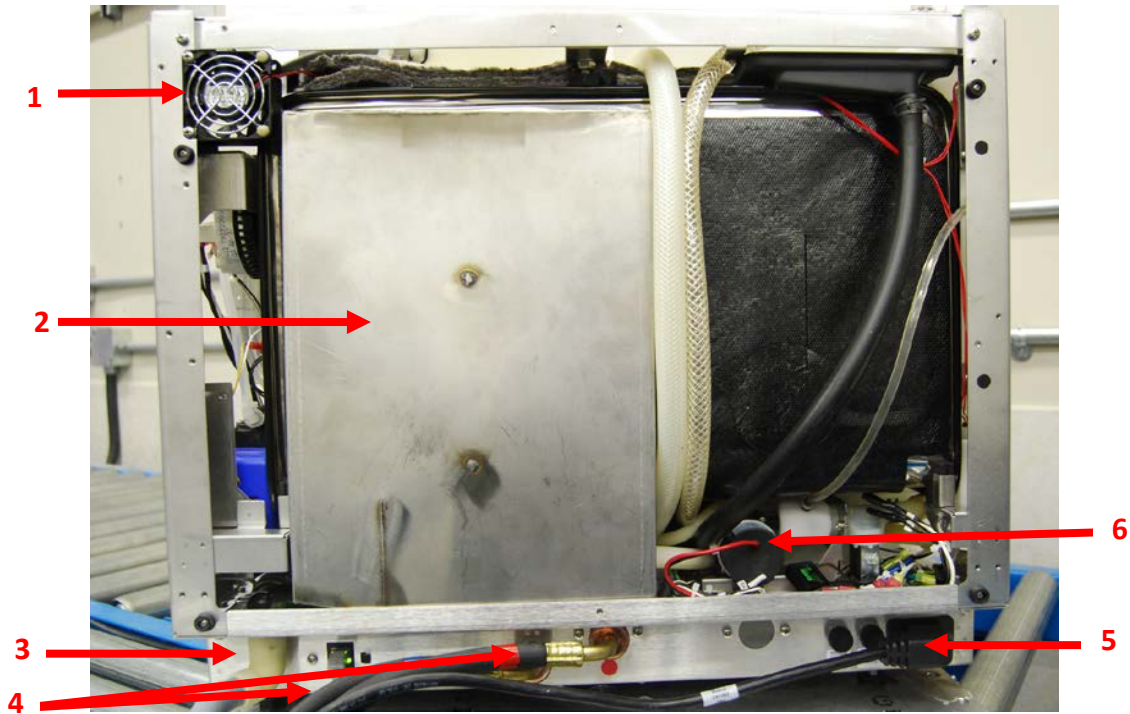

Item #	Part #	Description
1	01-113329S	Fan 24V Power Source
2	01-113335S	Duct Exhaust Assembly
3	01-107789S	Drain Hose
4	01-107788S	Water Inlet Hose
5	01-110281S	Power Cord
6	01-113283S	Pump 24V Air Gap

Hydrim C61 G4


Bottom View No Cover

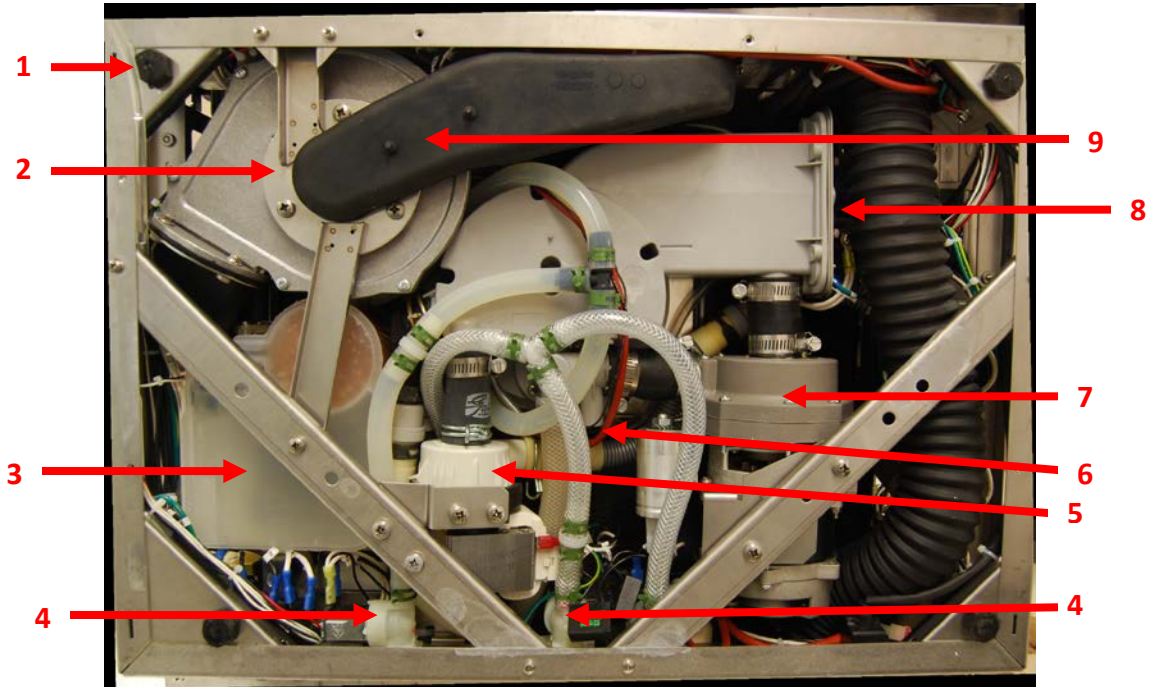

Item #	Part #	Description
1	01-113301S	Feet, Adjustable
2	01-113285S	Blower 24V (Dryer Motor)
3	01-113325S	Softener Assembly
4	01-113330S	Valve Inlet (hot & cold water)
5	01-113303S	Pump Drain
6	01-113681S	Proximity Sensor (lower RPM sensor)
7	01-113304S	Pump Circulation 60Hz
8	01-113272S	Heater Sump Kit (water heater)
9	01-113312S	Duct Inlet (air dryer duct)
Not Shown	01-113296S	Cover Bottom
Not Shown	01-113601S	Sump
Not Shown	01-113328S	Seal Sump
Not Shown	01-113320S	Gasket-Nut Softener


Hydrim C61 G4

Top View No Cover

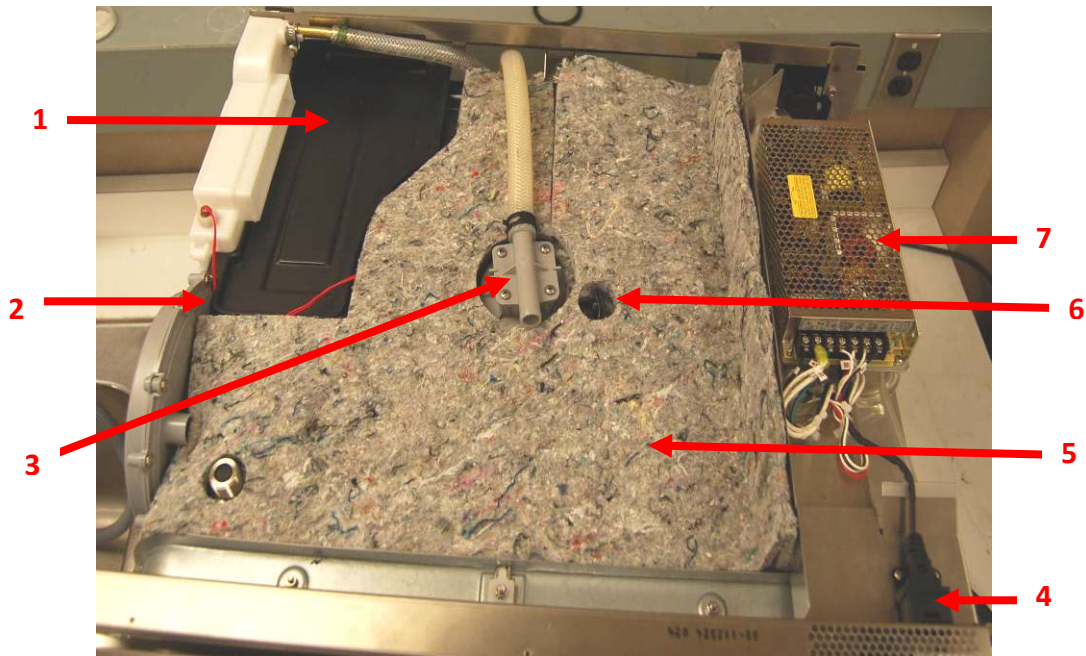


Item #	Part #	Description
1	01-113602S	Overflow Tray
2	01-113358S	Thermistor (for drying temperature)
3	01-113600S	Top Arm Fitting
4	01-113261S	Cable USB
5	01-113264S	Insulation Top
6	01-113681S	Proximity Sensor
7	01-113266S	Power Supply 5V/24V

Hydrim L110W Solid Door Front Door Closed

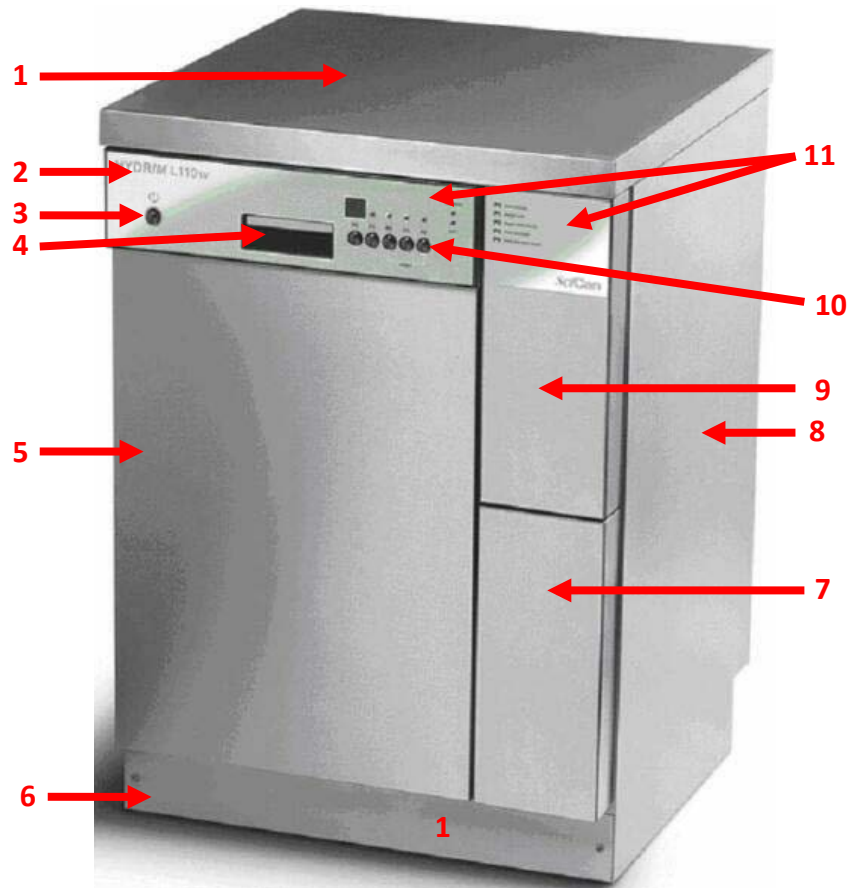


Item #	Part #	Description
1	50750.10	Cover, Top
2	13476.25	Front Facia Complete with Decal
3	13459.00	Power Switch
4	16945.00	Handle Door Lock
5	16979.01	Cover, Front Door Complete
6	16648.01	Kickplate Front
7	16662.06	Door, Cleaning Solution
8	16695.00	Cover, Right Side
9	16663.06	Cover, Slave Controller (no decal)
10	13471.02	Button, Program Silver
11	13790.06	Decal, Front (set of 2)
Not Shown	16696.00	Cover, Left Side
Not Shown	16656.00	Cover, Rear
Not Shown	13633.00	Screw Kit (cover screws)

Hydrim L110W Solid Door Front Door Open

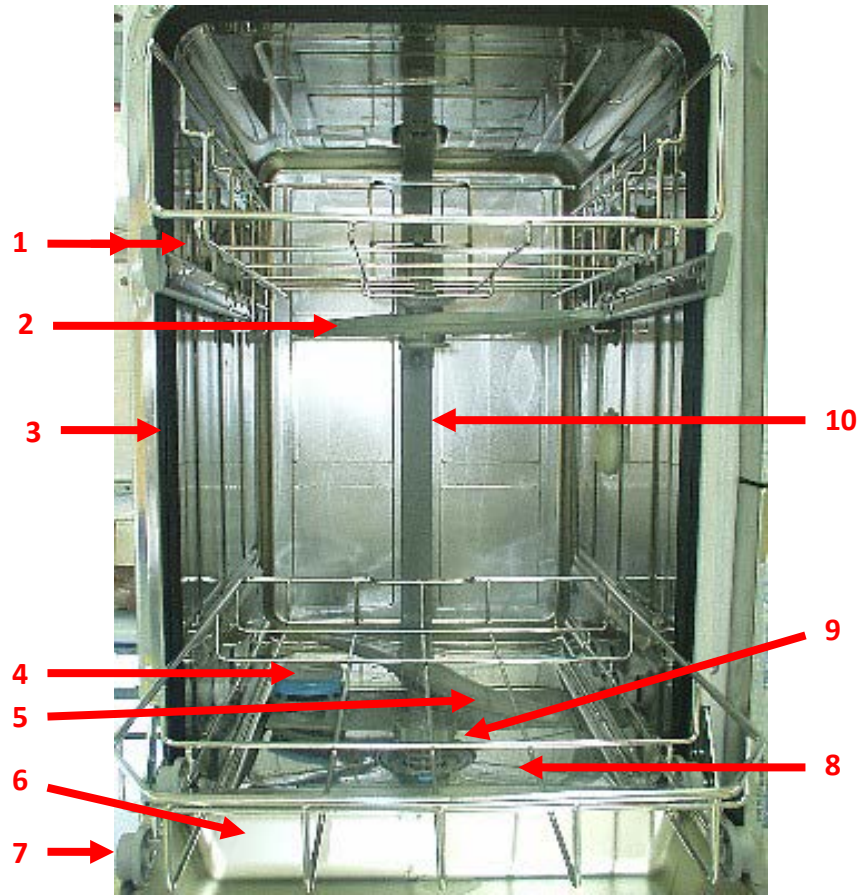


Item #	Part #	Description
1	13422.01	Wheel for Upper Trolley
2	13492.00	Wash Arm Upper
3	13456.00	Seal Peripheral Door Seal
4	13638.00	Lid for Salt Container
5	13481.00	Wash Arm Lower
6	13611.06	Door Inner Stainless Steel
7	13422.00	Wheel for Lower Trolley
8	13466.00	Filter Bottom Flat
9	13465.00	Filter Bottom Round
10	13629.00	Pipe Water Feed to Wash Arm

Hydrim L110W Solid Door
Left Side No Cover

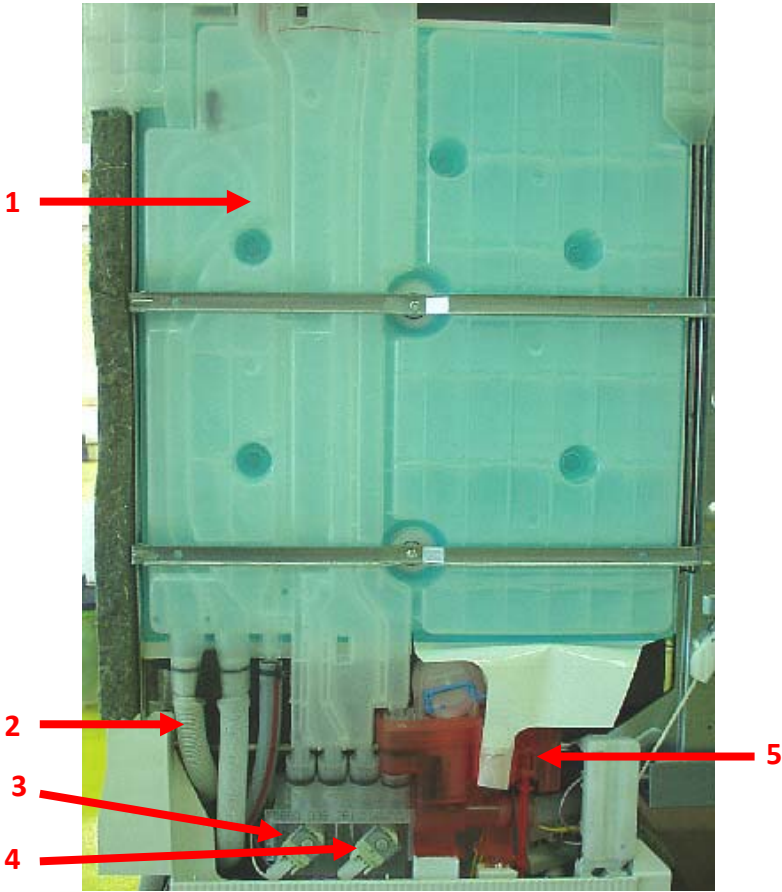


Item #	Part #	Description
1	13462.00	Side Bag
2	13455.00	Drain Tubing
3	13615.00	Solenoid Valve Regeneration
4	13614.00	Solenoid Valve Drain
5	13447.01	Emitter Housing Red
Not Shown	13447.00	Emitter Housing Clear

Hydrim L110W Solid Door Right Side No Cover

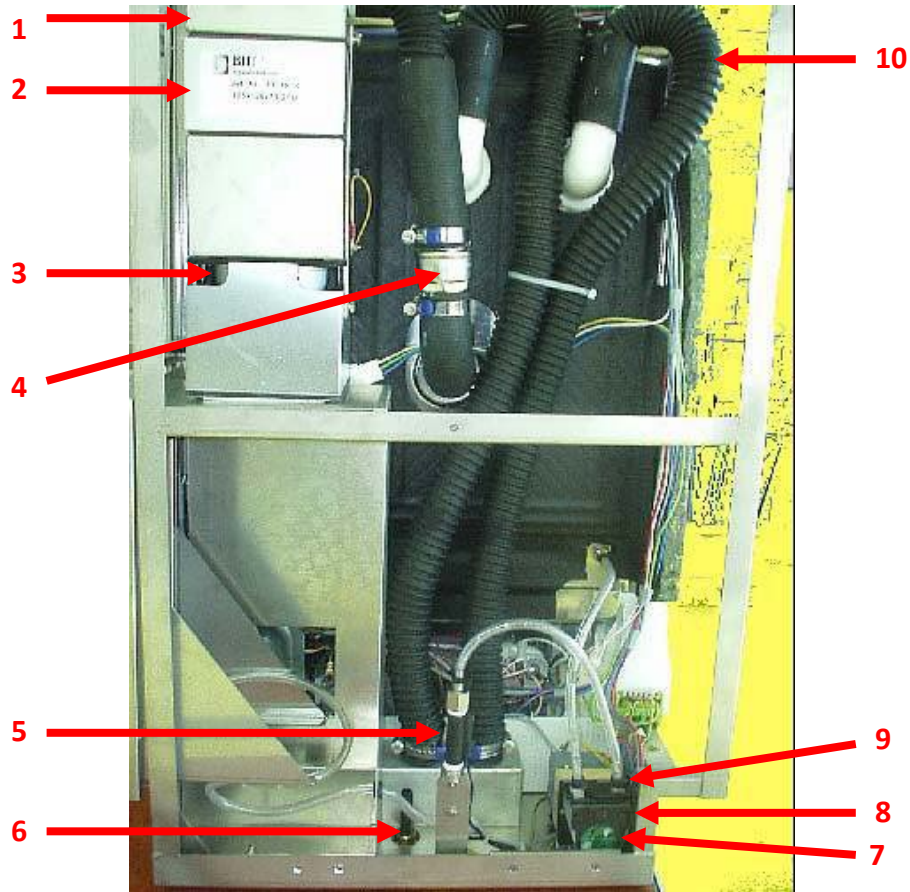


Item #	Part #	Description
1	16590.10	Dryer Unit Complete
2	13319.10	Dryer Air Filter
3	16590.10	Dryer Air Turbine Kit (to replace turbine #18466.00)
3	13220.06	Dryer Air Turbine (for Hydrim with kit #16590.10 installed)
4	13416.01	Check Valve Dryer 1"
5	01-111568S	Flow Switch w/Bracket
6	13599.00	Foot Front
7	21970.00	Dosing Pump Tubing
8	13634.00	Dosing Pump
9	21624.02	Dosing Pump Clamp External
10	21027.00	Tubing, Exhaust Air

Hydrim L110W Solid Door Inlet Solenoids & Controllers

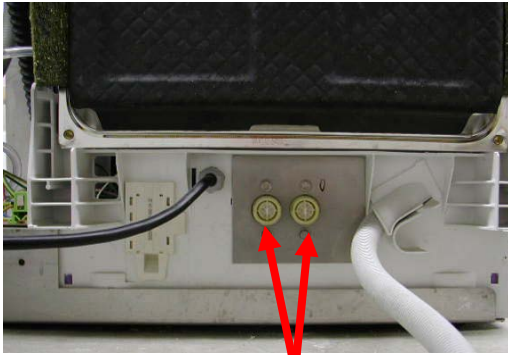


1

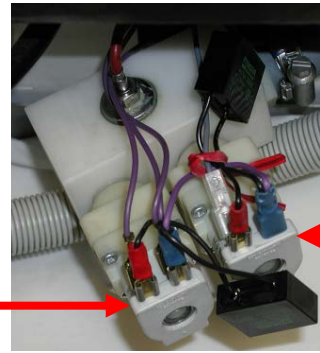

1


Item #	Part #	Description
1	N/A	Water Inlet Solenoid Valves***
1	01-109707S	Upgrade Kit
1	01-109863S	Valve, Harness & Temp. Sensor
***Note: If valves are white order Upgrade Kit If valves are blue order Valve, Harness & Temp. Sensor		

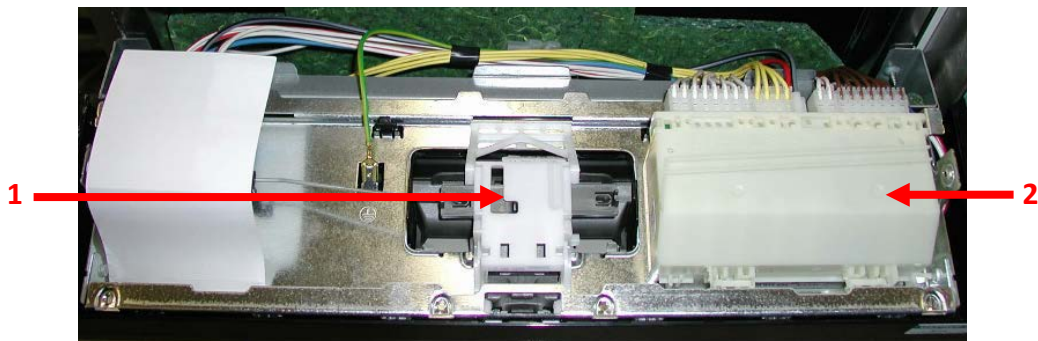

1

2

Item #	Part #	Description
1	16945.00	Handle Door Latch
2	01-113983S	Controller Master w/2 RC Suppressors

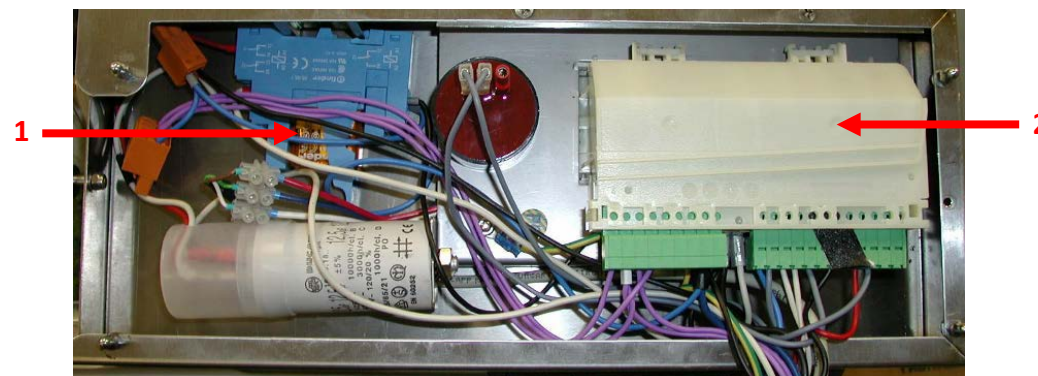


1

2


Item #	Part #	Description
1	23126.01	Relay Dryer
2	13475.04	Controller Slave

Hydrim L110W Solid Door Preventative Maintenance Kit

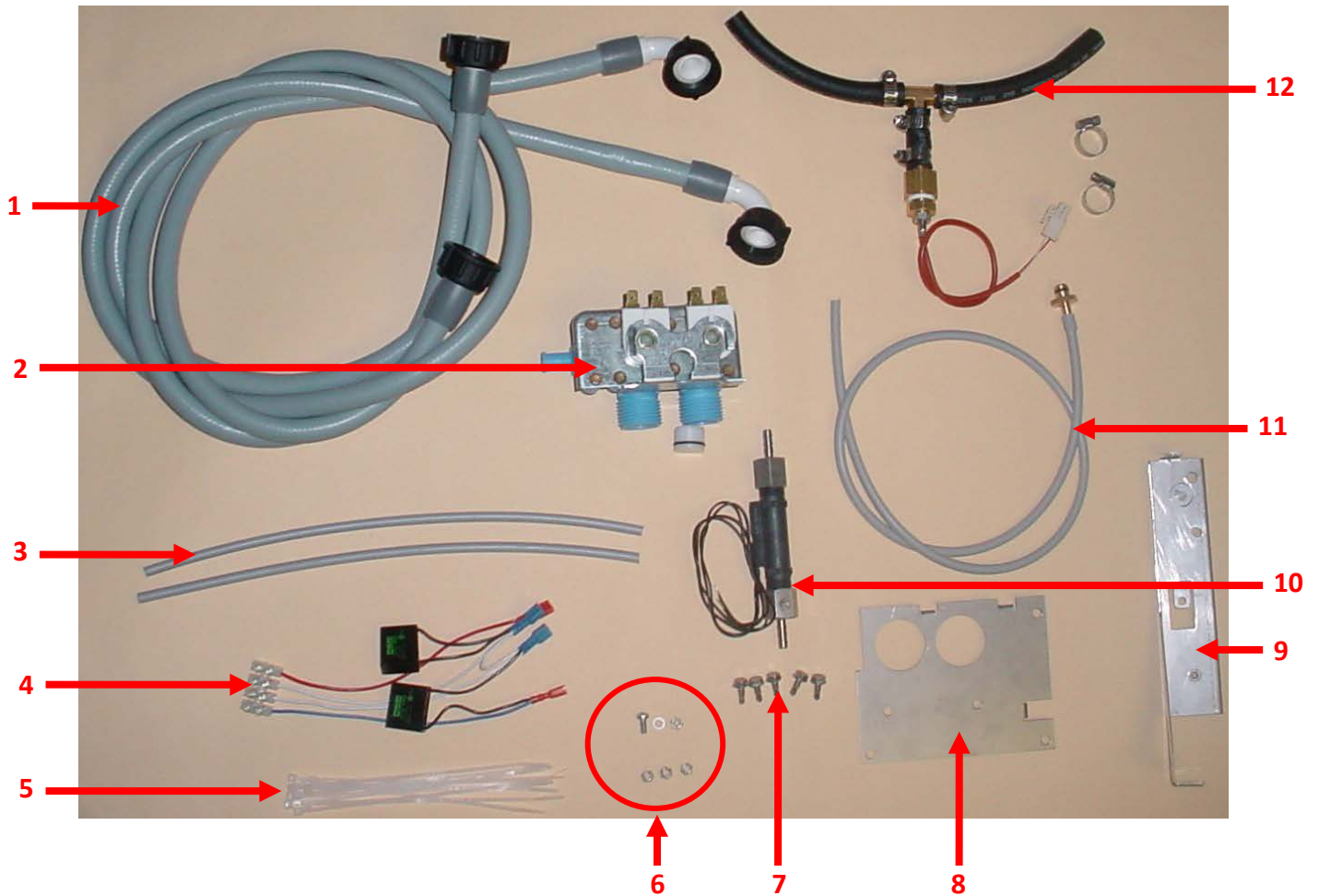

Item #	Description
1	Inlet tubing's for hot and cold water (01-109861S)
2	Water inlet valve block
3	Tubing's from flow switch to dosing pump and dosing pump to chamber
4	Wiring harness for new valve block
5	Tie wraps for wiring harness
6	Hardware to mount flow switch
7	Screws to mount valve block plate
8	Plate for water inlet valve block
9	Flow switch mounting bracket
10	Flow switch
11	Vent tubing for top of sidebag
12	Tubing assembly with sensor from inlet valve block to sidebag

Hydrim L110W w/Glass Door Front View

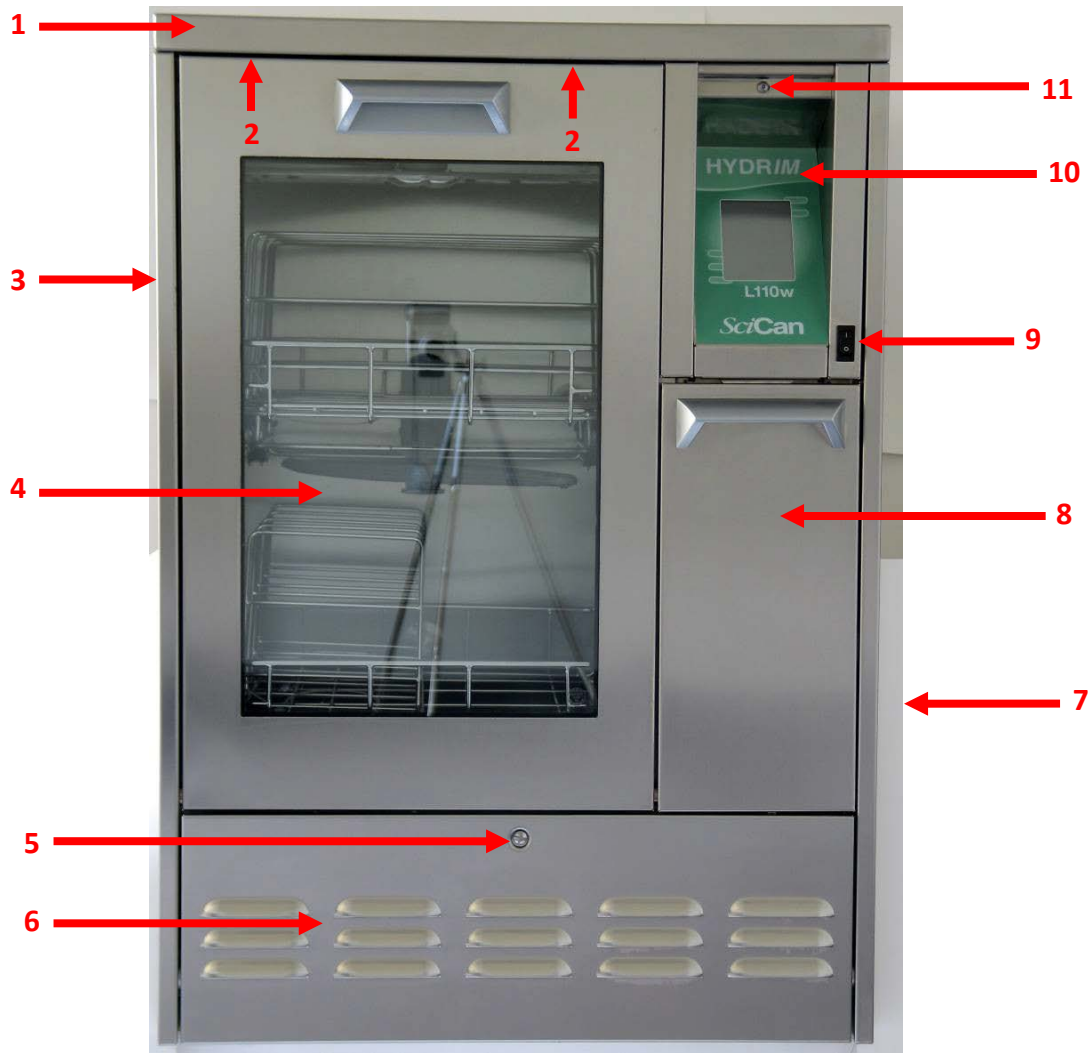


Item #	Part #	Description
1	01-111469S	Cover Top
2	01-111483S	Screw 2.5mm Hex Head M4x8 Top Cover (Not Visible)
Not Shown	01-111478S	Nut Top Cover for 2.5MM Screw
3	01-111466S	Cover Left Side
4	01-111578S	Door Complete (light weight)
5	01-111485S	Screw Kickplate M5x12 Slot Round Head
6	01-111476S	Kickplate Front
Not Shown	01-112593S	Foam for Kickplate (long piece & 2 squares)
7	01-111468S	Cover Right Side
8	01-111470S	Door Cleaning Solution
9	01-112024S	Rocker (Power) Switch
10	01-110219S	Decal Kit
11	01-111484S	Screw 3mm Hex Head M5x16 Back Cover/Controller Door
Not Shown	01-111467S	Cover Rear
Not Shown	01-111477S	Kickplate Rear

Hydrim L110W w/Glass Door Chamber with Trolleys




Item #	Part #	Description
1	01-111491S	Trolley Upper
2	01-109963S	5 Cassette Rack Full Size
3	01-111490S	Trolley Lower
4	01-112438S	Lower Basket Rail (Pkg of 2)

Hydrim L110W w/Glass Door Chamber

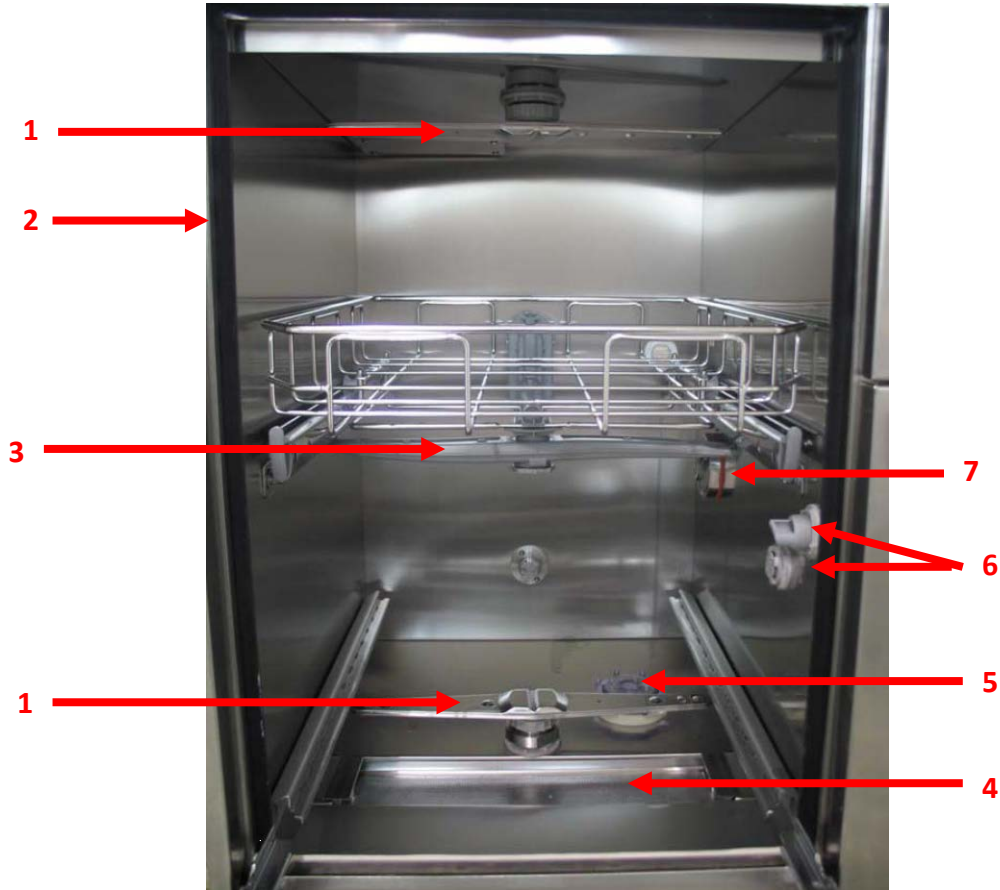


Item #	Part #	Description
1	01-109790S	Wash Arm Upper & Lower Metal
2	01-111559S	Door Seal Kit
3	01-111495S	Wash Arm Middle Complete
4	01-111589S	Mesh Filter Wash Chamber
5	13448.22	Salt Cap
Not Shown	01-112594S	Salt for Water Softener
6	01-111705S	Plugs for Rinse Aid Valve (Valve no longer available)
7	01-109144S	Dryer Fitting

Hydrim L110W w/Glass Door Under Mesh Filter

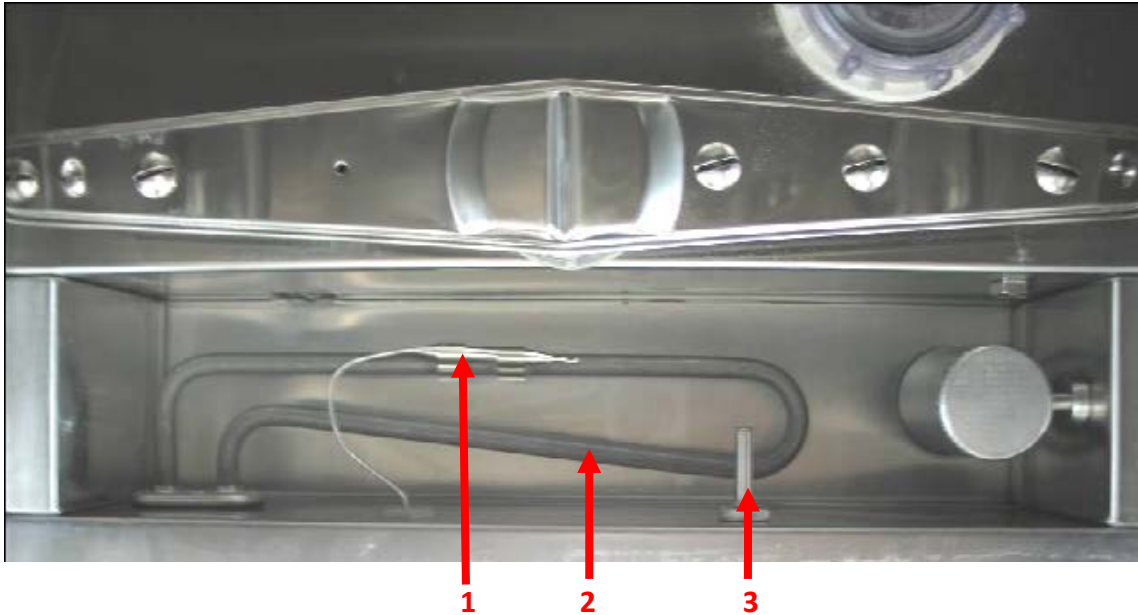


Item #	Part #	Description
1	01-111702S	Temperature Sensor Chamber Heater
2	01-111475S	Heater Chamber
3	01-111489S	Temperature Sensor Chamber PT-100

Inside Detergent (Cleaning Solution) Door

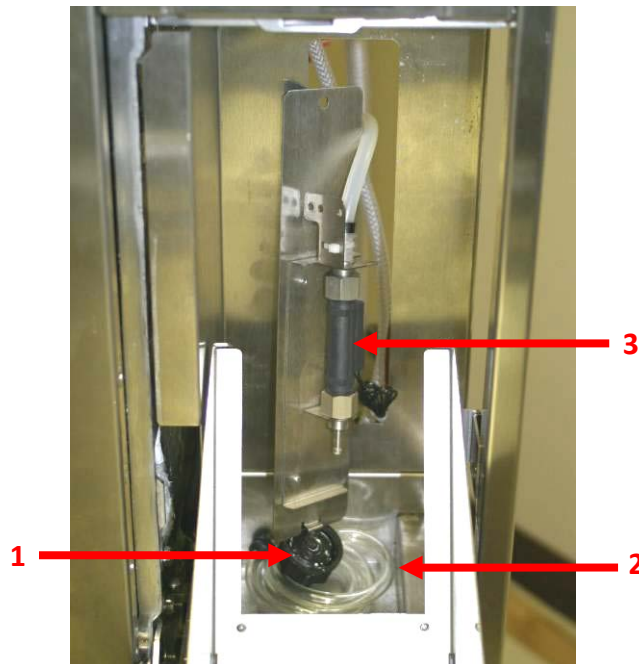


Item #	Part #	Description
1	13661.00	Cap Only for Detergent Tubing
2	01-111475S	Tubing Coiled w/ Cap for Detergent
3	01-111474S	Flow Switch

Hydrim L110W w/Glass Door Control Boards

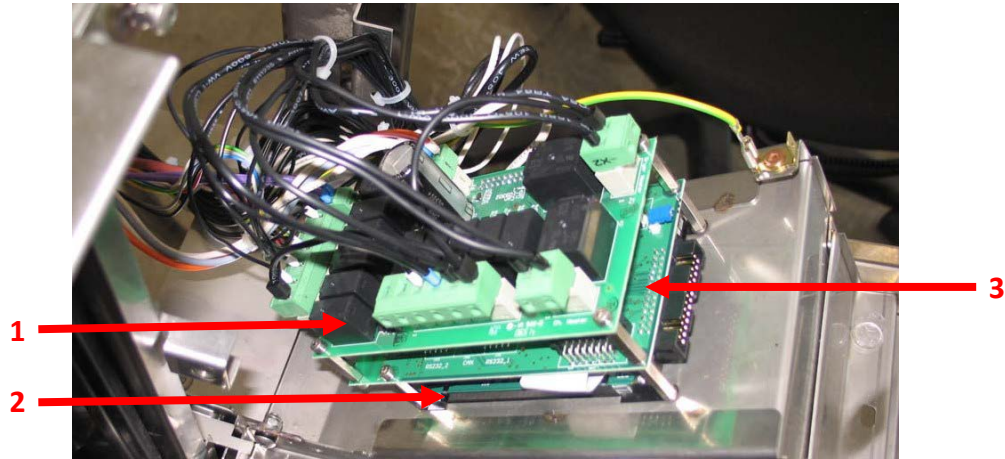


Item #	Part #	Description
1	01-109783S	IO Board
2	01-109785S	Touch Display
3	01-110217S	Logic Board

Hydrim L110W w/Glass Door Behind Front Kickplate Left Side

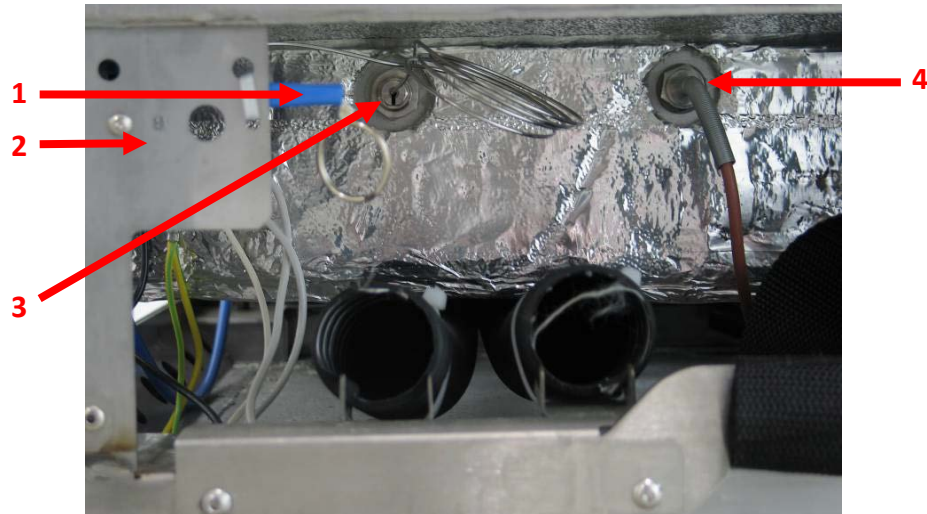


Item #	Part #	Description
1	01-112080S	Door Latch Cord Assembly
2	01-111482S	Switch Safety (behind plate)
3	01-111702S	Temperature Sensor Chamber Heater
4	01-111489S	Temperature Sensor Chamber PT-100
Not Shown	01-113125S	Black Wire from Safety Switch to Heating Element

Behind Front Kickplate Right Side

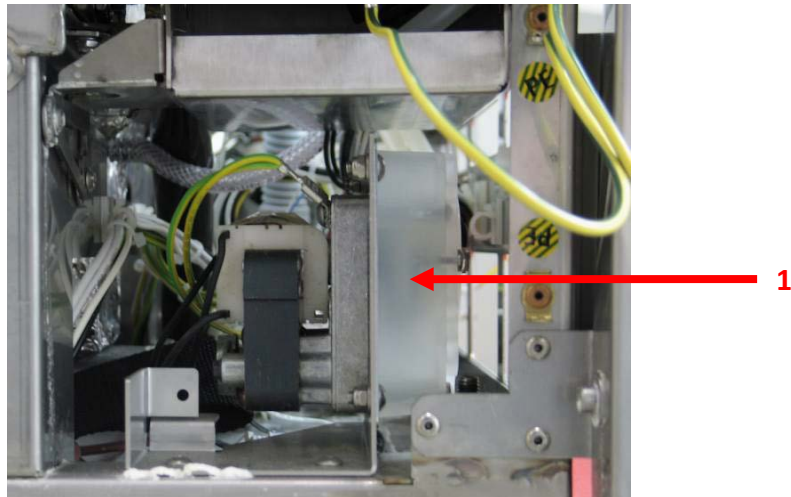


Item #	Part #	Description
1	01-111471S	Dosing Pump
Not Shown	01-109909S	Tube Dosing Pump Internal

Hydrim L110W w/Glass Door Upper Right Side

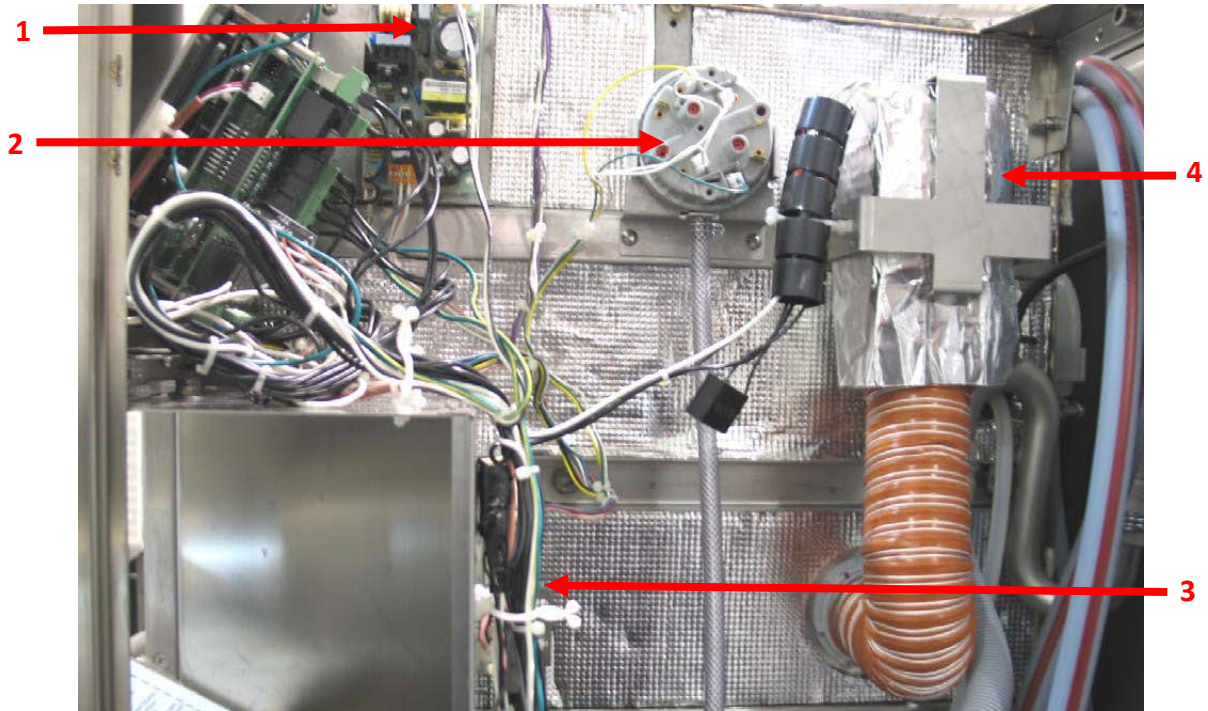


Item #	Part #	Description
1	01-109782S	Power Supply Board
2	01-111408S	Full & Overflow Switch
3	01-111474S	Flow Switch
4	01-109142S	Dryer Assembly

Hydrim L110W w/Glass Door Lower Right Side

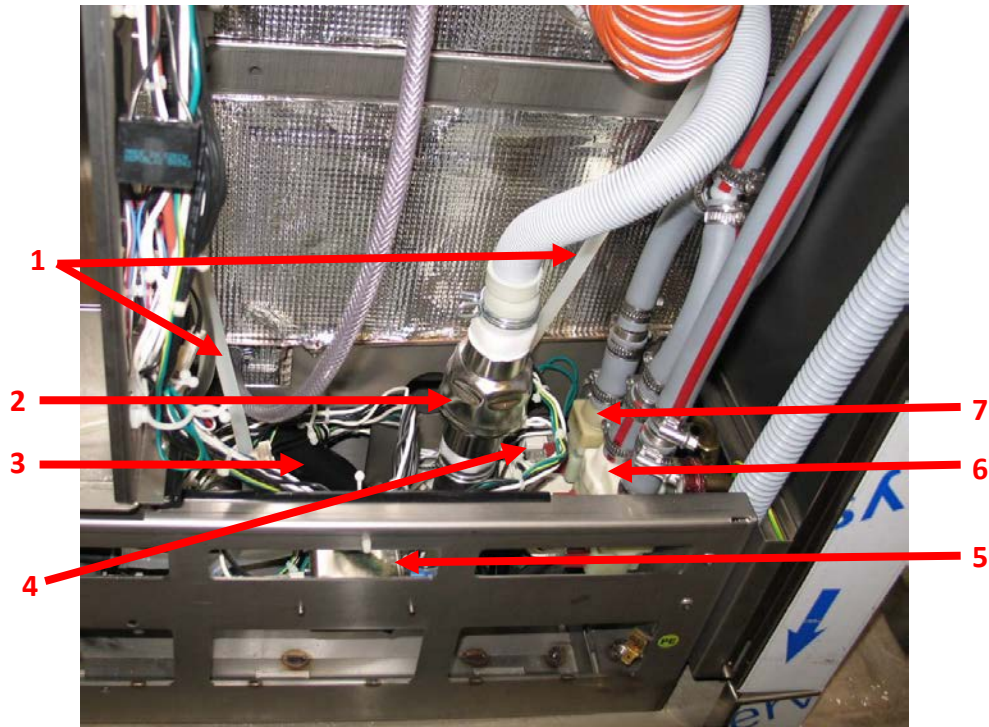


Item #	Part #	Description
1	01-112182S	PVC Tubes Dosing System
2	01-112026S	Check Valve Drain
3	01-111580S	Door Tension Spring (light door no cutouts)
3	18156.00	Door Tension Spring (heavy door with half-moon cutouts)
3	01-112729S	Dual Spring Retro-fit Kit
3	01-112730S	Dual Springs (replacement springs only)
4	01-111412S	Drain Pump
5	01-111473S	EMI Filter Kit
6	01-111494S	Valve Single Hot Water
7	01-111493S	Valve Double Cold Water

Hydrim L110W w/Glass Door Rear View No Cover

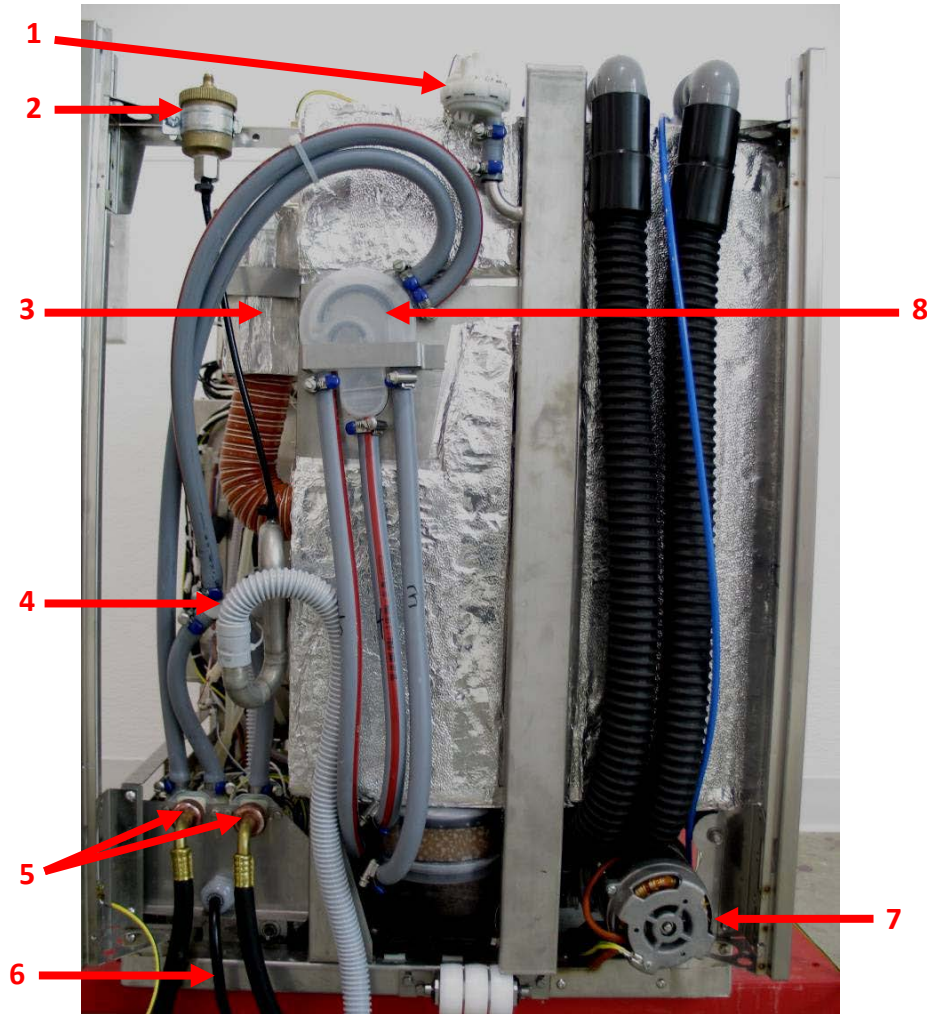

Item #	Part #	Description
1	01-111409S	Switch Pressure
2	01-111585S	Breather
3	01-109142S	Dryer Assembly
4	01-111492S	Drain Tubing
5	01-107788S	Water Inlet Tubing
6	01-111480S	Power Cord
7	01-111465S	Recirculation Pump
8	01-112873S	Air Break

Hydrim L110W w/Glass Door Top View No Cover

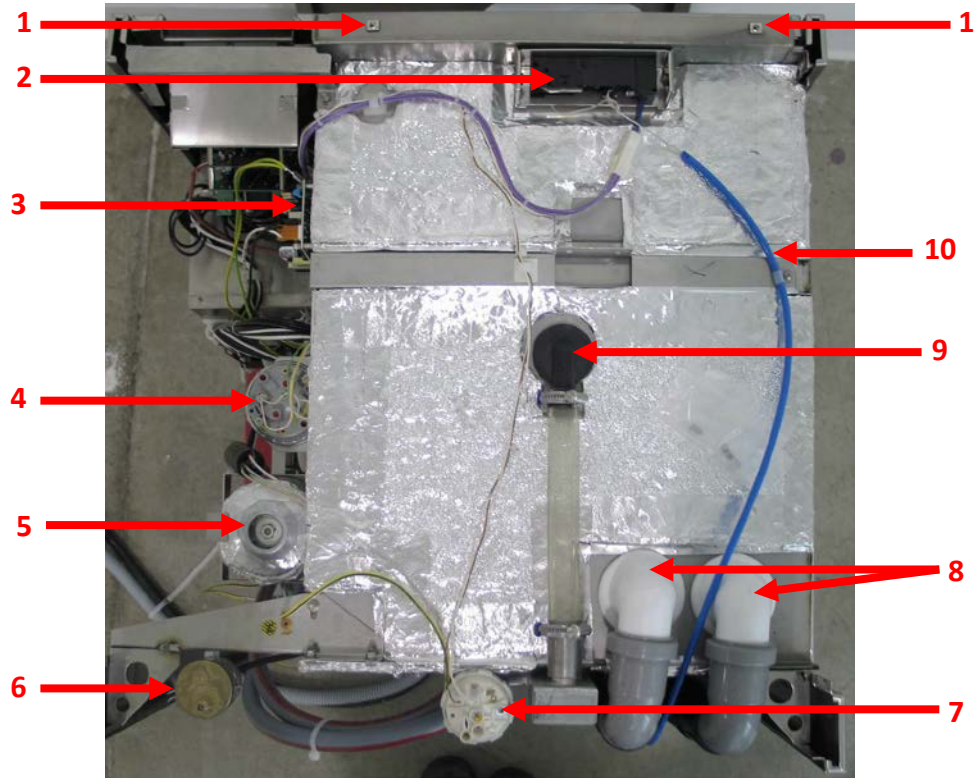

Item #	Part #	Description
1	01-111478S	Nut for Top Cover 2.5mm Screw
2	01-111783S	Door Latch Assembly
3	01-109782S	Power Supply Board
4	01-111408S	Switch Full/Overflow
5	01-109142S	Dryer Assembly
6	01-111585S	Breather
7	01-111409S	Switch Pressure
8	01-113814S	Air Vent Adapter Fittings (Qty 2)
9	01-109776S	Fitting Top Arm
10	01-112080S	Door Latch Cord Assembly


Hydrim L110W G4

Accessories


The following are included with the HYDRIM:




Rack for five 8" x 11" cassettes, for upper or lower trolley. Full size (1/1).
Part-number
01-109963S

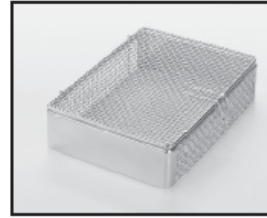

Basket with lid.
Quarter size (1/4).
Part-number
01-109966S

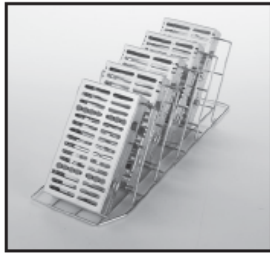

Rack for five medium size cassettes, for upper or lower trolley. Half-size (1/2).
Part-number
01-109964S

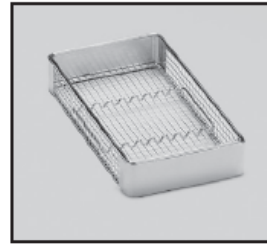


Hygiene basket for loose instruments.
Quarter size (1/4).
Part-number
01-109967S

The following are optional:




Rack for four 11" x 14" cassettes, for upper or lower trolley. Full size (1/1).
Part-number
01-109965S

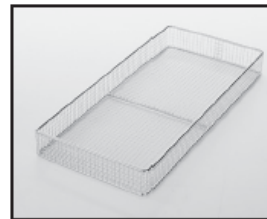


STATIM 5000 basket.
Part-number
01-107241




Hinged instrument rack.
Part-number
01-110409S

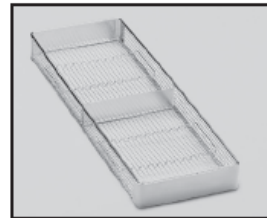

Long Basket.
Part-number
01-108232

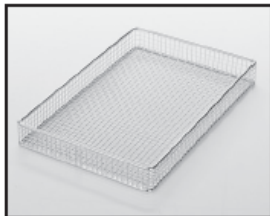

STATIM 2000 basket.
Part-number
01-107240

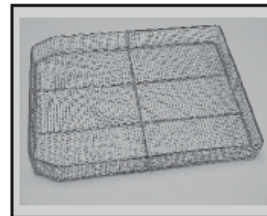

Full Size Basket.
Part-number
01-111598

Hydrim L110W G4

Front View



Item #	Part #	Description
1	01-112398S	USB Cable
2	01-113856S	LCD Assembly
3	01-113833S	Decal
4	01-112024S	Power Switch Rocker

L110W G4 Covers (Serial #400114I00013 & Below)



Item #	Part #	Description
1	01-111469S	Cover Top
2	01-111466S	Cover Left Side
3	01-113836S	Door Wash Chamber
4	01-111476S	Kickplate Front
5	01-111485S	Screw Kickplate
6	01-113835S	Door Chemical
7	01-111468S	Cover Right Side
8	01-111484S	Screw Service Door & Back Cover
Not Shown	01-111467S	Cover Rear
Not Shown	01-111477S	Kickplate Rear

L110W G4 Covers (Serial #400114I00014 & Above)



Item #	Part #	Description
1	01-114232S	Cover Top (Shown in picture)
1	01-114233S	Cover Top Low (1 ³ / ₁₆ " lower)
2	01-114237S	Cover Left Side
3	01-113836S	Door Wash Chamber
4	01-114234S	Kickplate Front
5	01-111485S	Screw Kickplate
6	01-113835S	Door Chemical
7	01-114230S	Cover Right Side
Not Shown	01-114231S	Cover Rear
Not Shown	01-114235S	Kickplate Rear
Not Shown	01-114236S	Screw Kit (for covers & kickplates)

Hydrim L110W G4 Chamber




Item #	Part #	Description
1	01-111495S	Wash Arm Upper & Lower
2	01-111495S	Wash Arm Middle Complete
3	01-113790S	Seal Main Chamber
4	01-112438S	Rail (White Plastic) Lower Trolley
5	01-113843S	Filter Course
6	13448.22	Salt Cap
7	01-109144S	Dryer Fitting
Not Shown	01-113844S	Filter Fine (under course filter)
Not Shown	01-113789S	Lower Door Seal

01-109790S

Hydrim L110W G4

Front View No Kickplate


Left Side

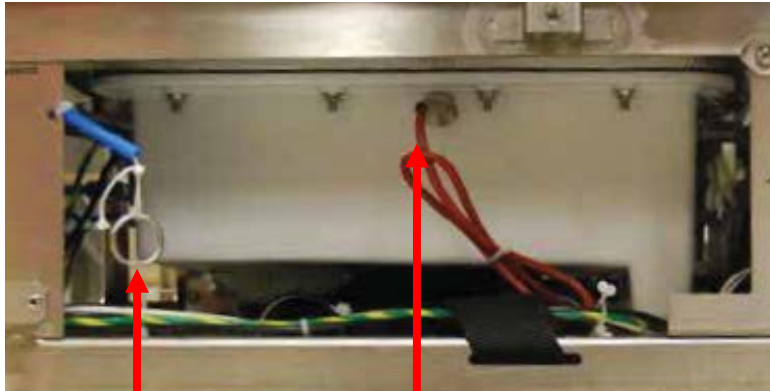

1

2

Right Side

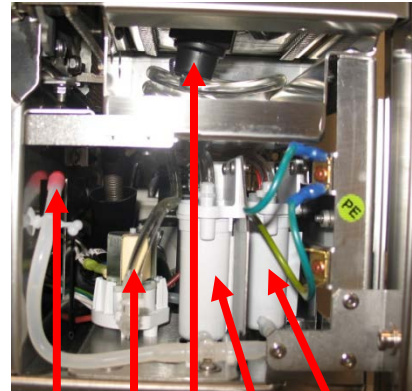

3

4

5


6

7

Item #	Part #	Description
1	01-112080S	Door Latch Cord Assembly
2	01-113271S	Temperature Sensor (Sump)
3	01-104343S	Plug Drain Tubing
4	01-113860S	Valve Dosing
5	01-111670S	Tubing Coiled w/ Cap for Detergent
6	01-113859S	Dosing Reservoir w/Plug
7	01-113858S	Dosing Reservoir w/Switch

Hydrim L110W G4


Left Side View No Cover

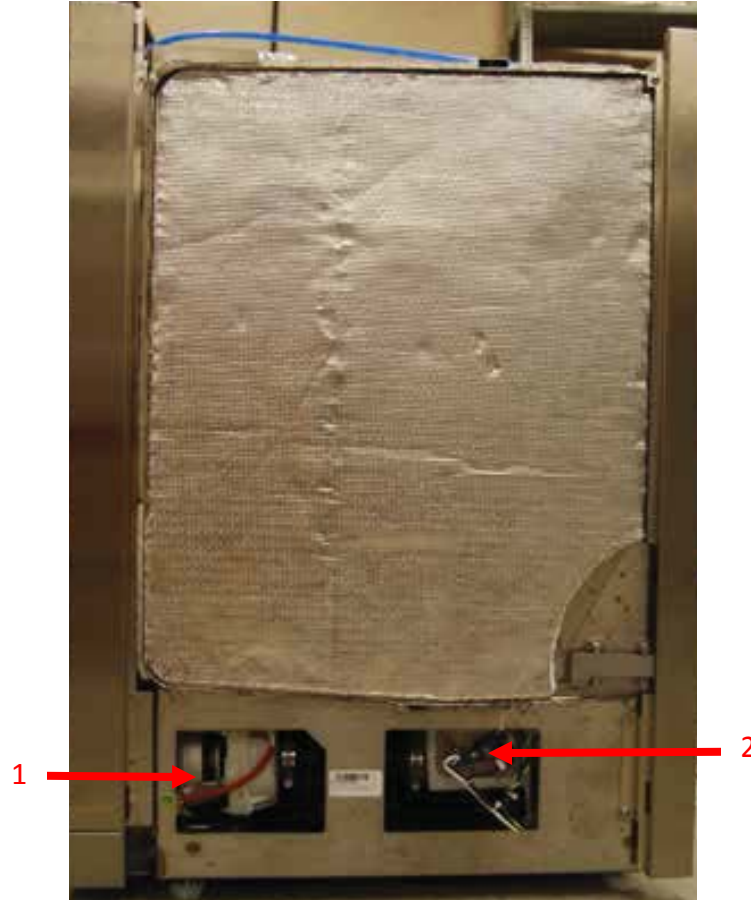

Item #	Part #	Description
1	01-111465S	Circulation Pump
2	01-113839S	Water Heater
Not Shown	01-113840S	Hinge Left

Hydrim L110W G4


Right Side View Upper No Cover

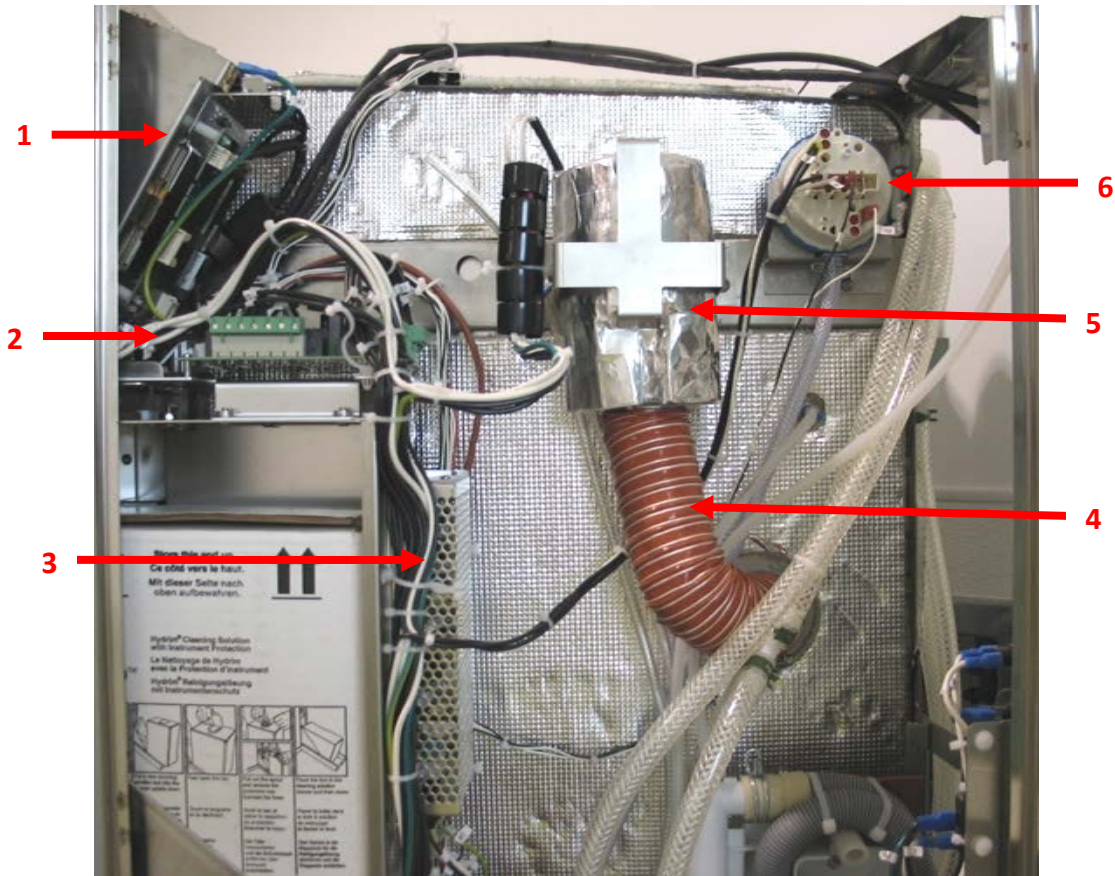

Item #	Part #	Description
1	01-113665S	Color LCD Controller
2	01-113310S	IO PCB
3	01-114108S	Power Supply 5V/24V Upgrade Kit
4	01-109143S	Dryer Tubing
5	01-109142S	Dryer Assembly
6	01-111408S	Chamber Level & Overflow Switch
Not Shown	01-113682S	Speaker Assembly

Hydrim L110W G4


Right Side View Lower No Cover

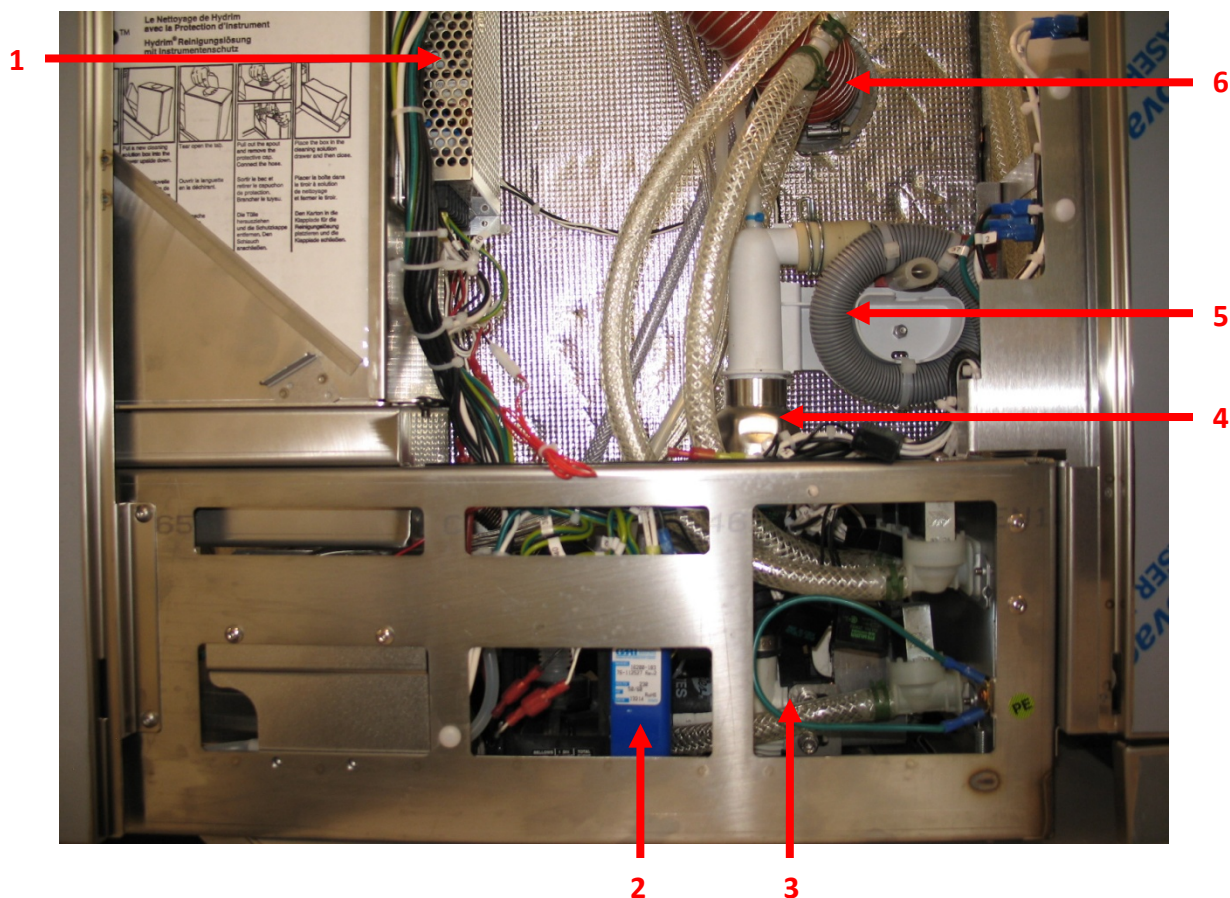

Item #	Part #	Description
1	01-114108S	Power Supply 5V/24V Upgrade Kit
2	01-113837S	Dosing Pump (includes microswitch)
3	01-111412S	Drain Pump
4	01-113867S	Check Valve 1"
5	01-113851S	Tubing Drain
6	01-109143S	Dryer Tubing
Not Shown	01-112730S	Dual Spring
Not Shown	01-113841S	Hinge Right


Hydrim L110W G4

Rear View No Cover

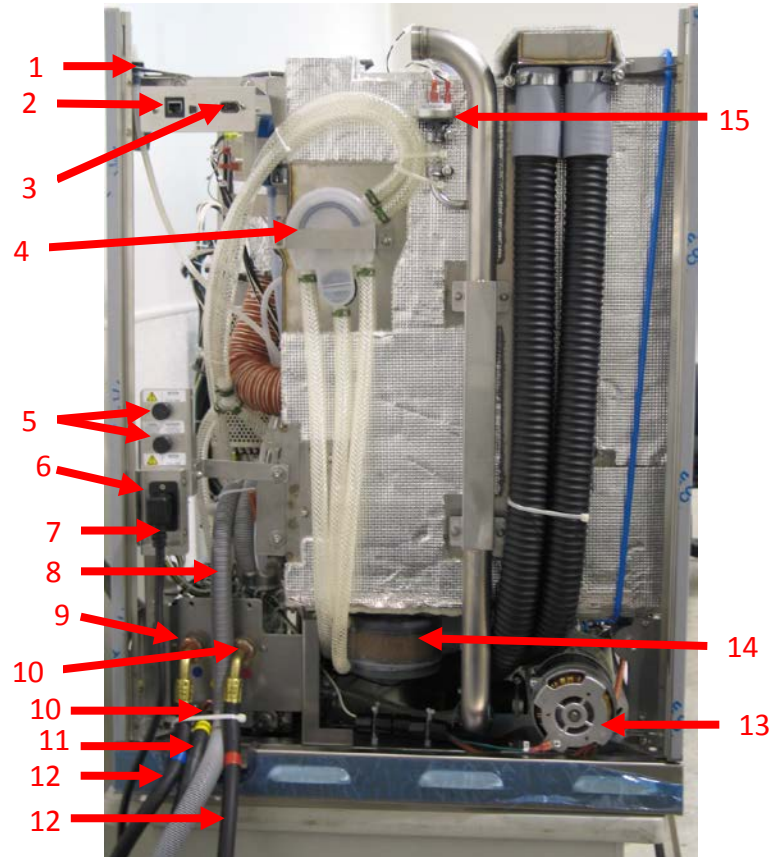

Item #	Part #	Description
1	01-113831S	Breather
2	01-113854S	Cable Ethernet
3	01-113260S	Cable RS232 Port
4	01-113830S	Air Break
5	01-112545S	Fuse Holder (Qty 2)
Not Shown	01-103472S	Fuses 15A (Qty 2)
6	01-110505S	EMI Filter
7	01-110281S	Power Cord
8	01-113851S	Tubing Drain
9	01-113331S	Cold Valve 1 in 2 out
10	01-113330S	Hot or RO Valve 1 in 1 out
11	01-113863S	Water Inlet Hose RO
12	01-107788S	Water Inlet Hose (Hot or Cold)
13	01-111465S	Circulation Pump
14	01-113857S	Water Softener
15	01-111409S	Chamber Pressure Switch

Hydrim L110W G4 Top View No Cover

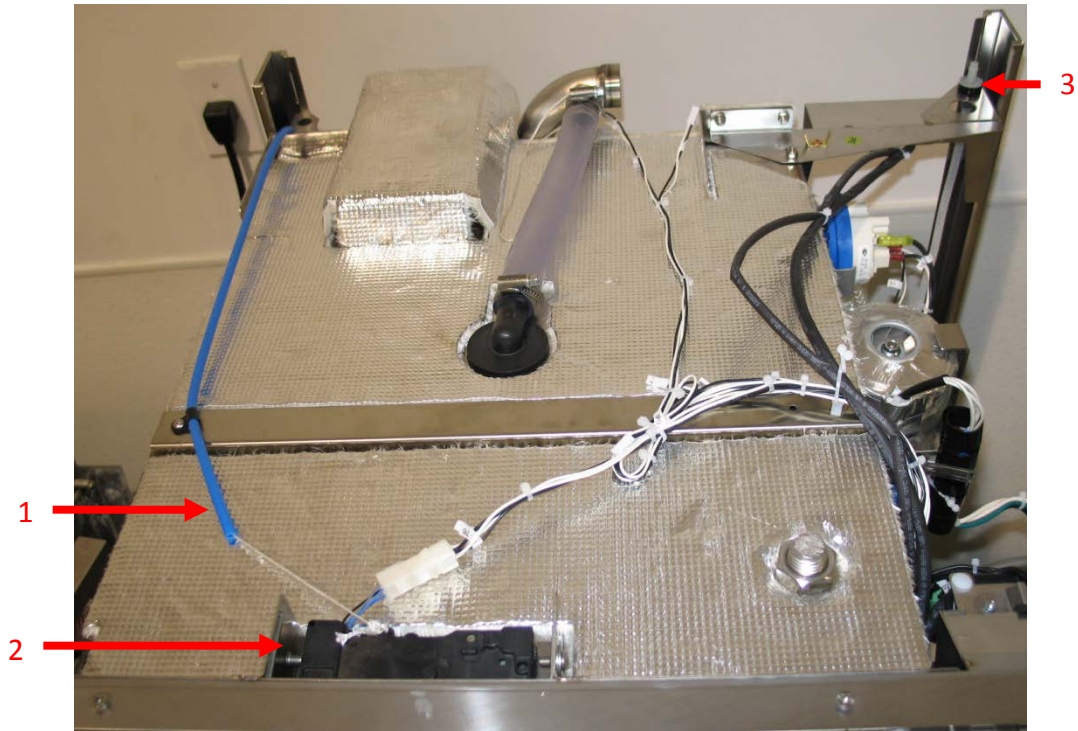


Item #	Part #	Description
1	01-112080S	Door Latch Cord Assembly
2	01-111783S	Door Latch Assembly
3	01-113831S	Breather

Statmatic Parts





Item #	Part #	Description
1	S30153	Cover, Top with Keypad
2	S30123	Keypad
3	S30141	Spray Can Black Plastic Adapter (no brass sliding lock ring)
4	S30143	Door Collett

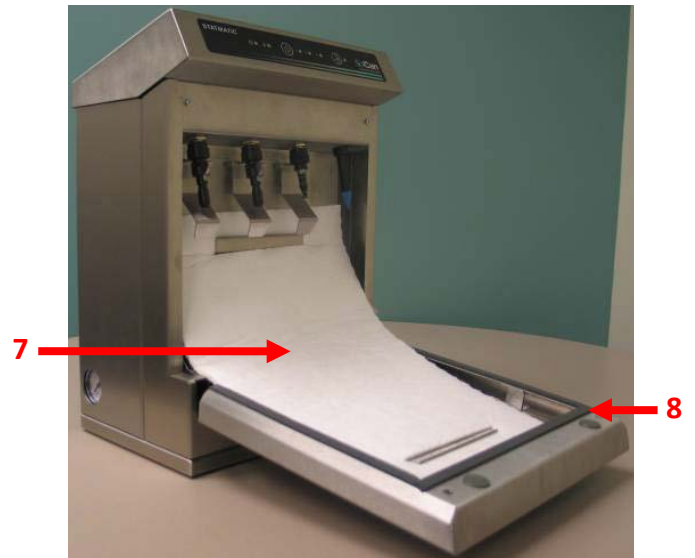


Item #	Part #	Description
5	S30113	Power Inlet Assembly
6	S30112	Regulator, Air
7	S30102	Absorption Pad (6 large pads, 2 bottom pads under large pad)
8	S30120	Seal Kit Front Door

Statmatic Parts

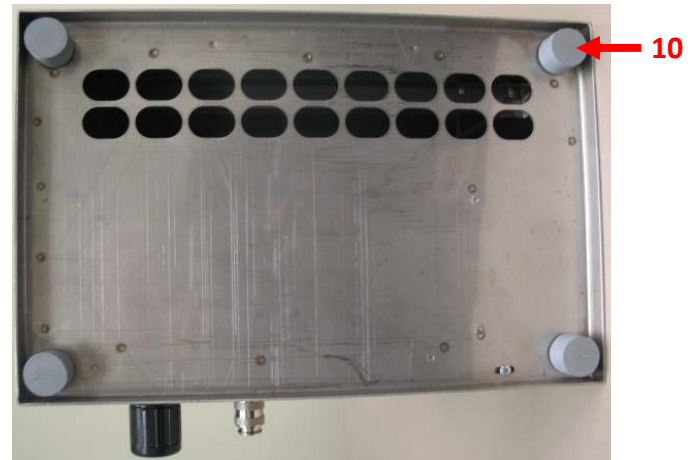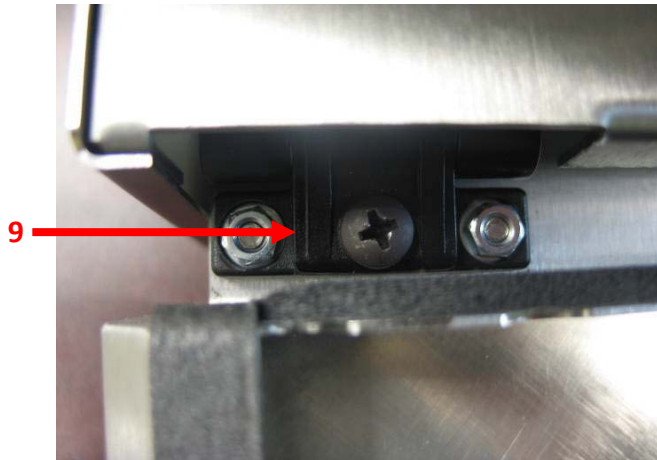


Item #	Part #	Description
9	S30103	Hinge (for front door, no nuts or screws)
10	S30108	Rubber Foot


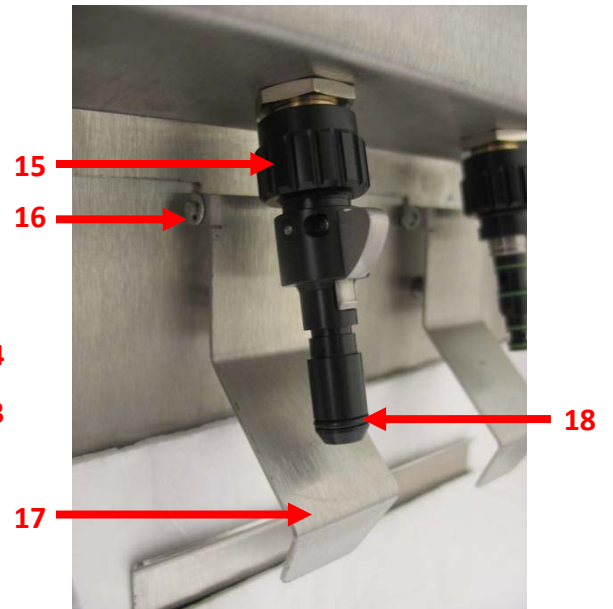


Item #	Part #	Description
11	S30101-CX	Coupling Multiflex/Connex (for SciCan & Kavo Highspeeds)
12	S30101-E	Coupling E-Type (for SciCan & Kavo low speed contra angles)
13	S30116	O-ring Set Multiflex/Connex Coupling
14	S30109	Chuck Adapter Complete
15	S30144	Clamping Nut
16	S30155	E-Clip Hinged Post
17	S30150	Recognition Flap
18	S30118	O-ring Intramatic (o-ring set for E-Type coupler)

Statmatic Parts

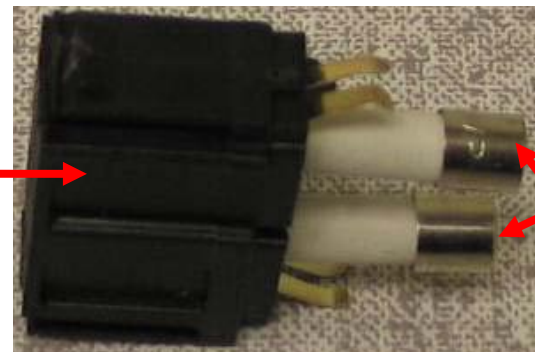


Item #	Part #	Description
19	S30115	Fuse Drawer
20	S30117	Fuse 1A

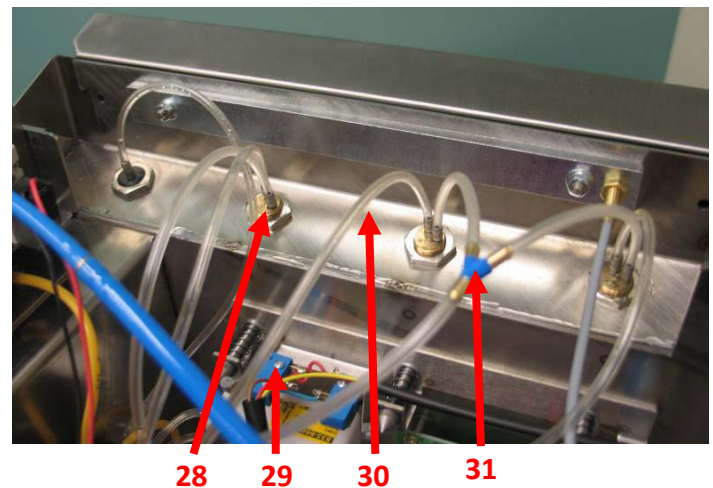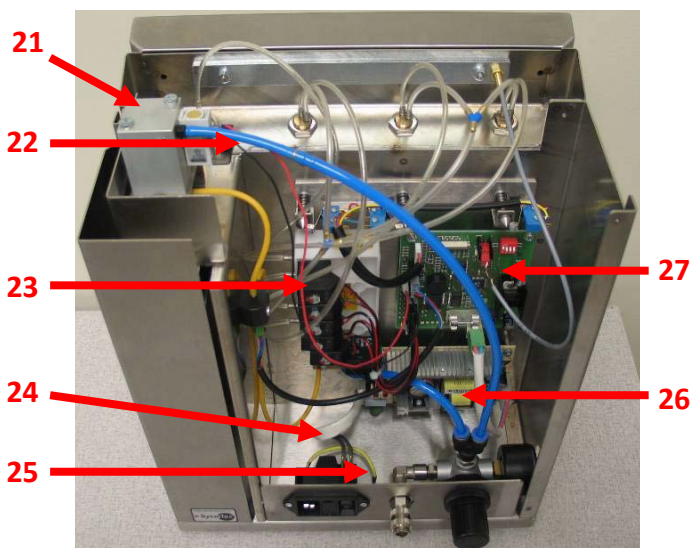



Item #	Part #	Description
21	S30107	Comp. Air Cylinder (air solenoid for solution bottle)
22	S30132	Tubing Blue (2" piece of blue air line tubing)
23	S30105	Solenoid Valve Block
24	S30130	Power Cable (from power inlet to power supply board)
25	S30129	Cable PE (ground wire for power inlet assembly)
26	S30119	Power Supply Board
27	S30121	Control Board
28	S30151	Coupler Receptacle (for 4, 5 & 6 hole configuration)
29	S30106	Microswitch Assembly (3 Switches with Wiring)
30	S30125	Tubing (clear 10" tubing)
31	S30110	Hose Set (complete tubing assembly including "Y" fittings)

Statmatic Parts

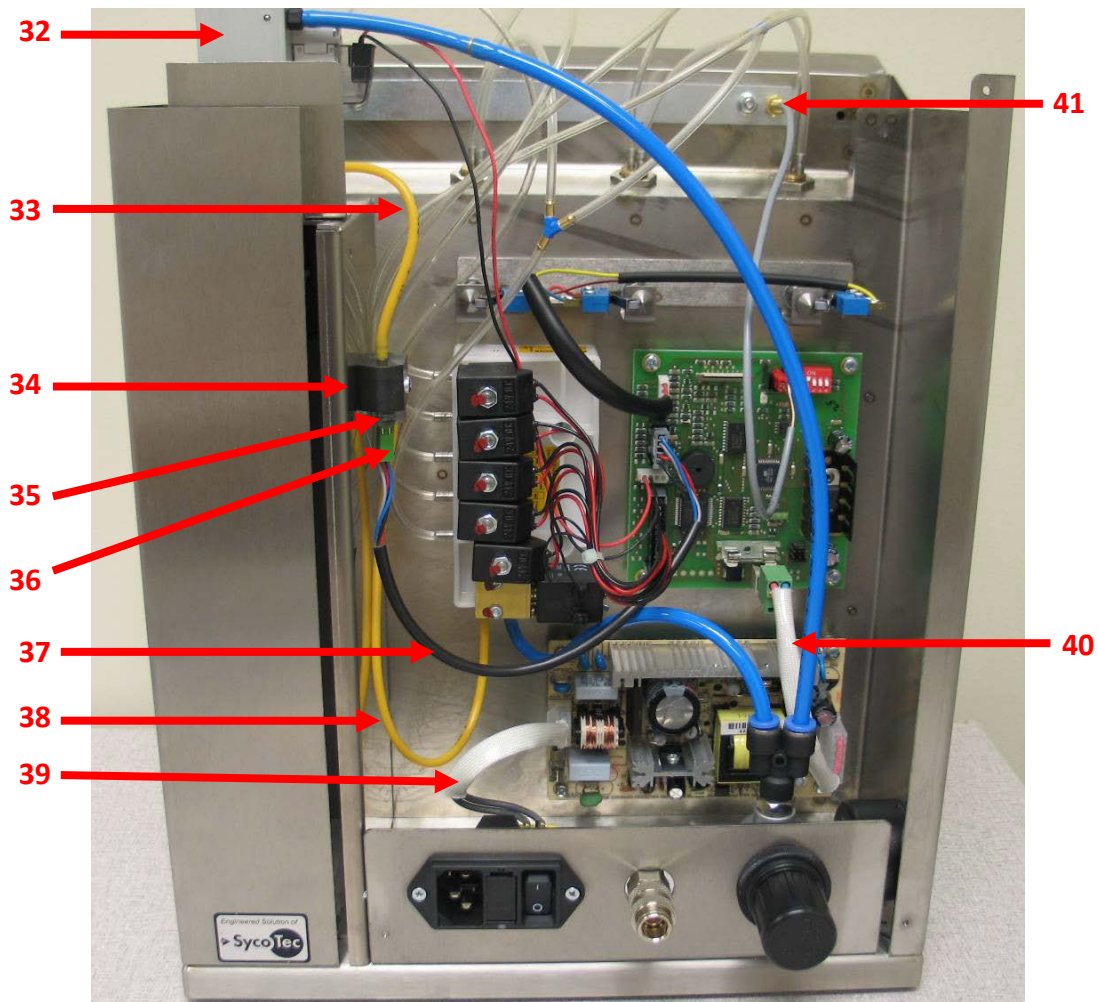

Item #	Part #	Description
32	S30107	Comp. Air Cylinder (air solenoid for solution bottle)
33	S30127	Tubing Labeled (4" yellow tubing labeled "oil")
34	S30145	Sensor Collet Complete (with oil tubes, optical sensor & wiring)
35	S30104	Fluid Sensor (optical fluid sensor)
36	S30137	Clamp Bugel (wire fastener that holds cable to fluid sensor)
37	S30138	Cable Photo Sensor (cable from fluid sensor to main PCB)
38	S30126	Tubing Labeled (10" yellow tubing labeled "oil")
39	S30130	Power Cable (from power inlet to power supply board)
40	S30131	Cable SNT/Elect. (cable from power supply board to main PCB)
41	S30133	Tubing Switch (door magnet sensor and cable)

Statmatic Parts

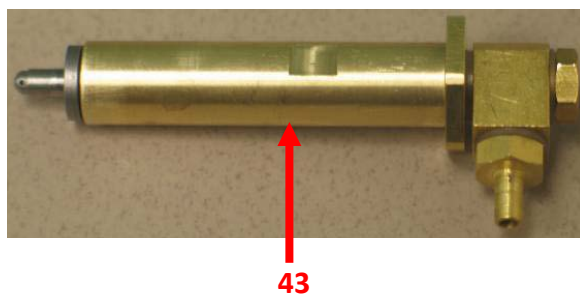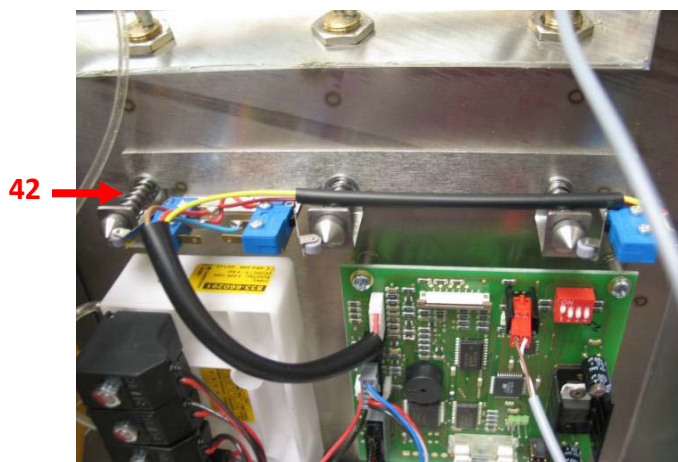

Item #	Part #	Description
42	S30124	Spring (for microswitch activating)
43	S30142	Guide Set (plunger assembly for solution bottle)

Statmatic Couplers



S30101-BA
BienAir



S30101-CX
Multiflex/Connex



S30101-E
E-type



S30101-KVH
Kavo Intra head



S30101-MW
Midwest Slow speed



S30101-NSK
Coupling NSK



S30101-ST1I
Sirona T1 Slow speed
Attachments




S30101-ST1T
Sirona T1 High speed




S30101-W&H
Coupling W&H

Aquastat Boiler

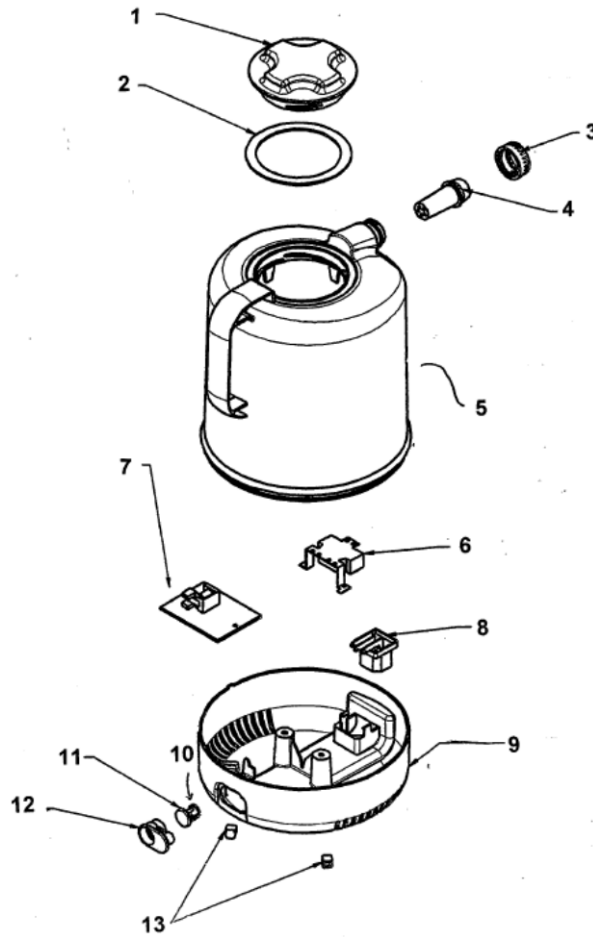


Item #	Part #	Description
1	P492-2A	Boiling Chamber Lid (No Gasket)
2	P167-72	Gasket for Boiling Chamber Lid
3	KP05EC001	Baffle Nut
4	KP03WS010	Baffle without O-ring
Not Shown	KP32GA162	Baffle Gasket
Not Shown	KP32GA163	Baffle O-ring
5	P186-1X	Boiler Vessel with Heater Only
6	KP28TS091	Thermostat
Not Shown	KP28TS098	Thermostat (round)
7	KP64CO001U	Control Board
8	P338-26	Strix Connector Boiler
9	KP44WP005	Boiler Base Bottom with Grey Feet
10	P128-134	Spring
11	P81-174	Button
12	P118-96	Retainer
13	P19-283	Grey Feet

Aquastat Chassis

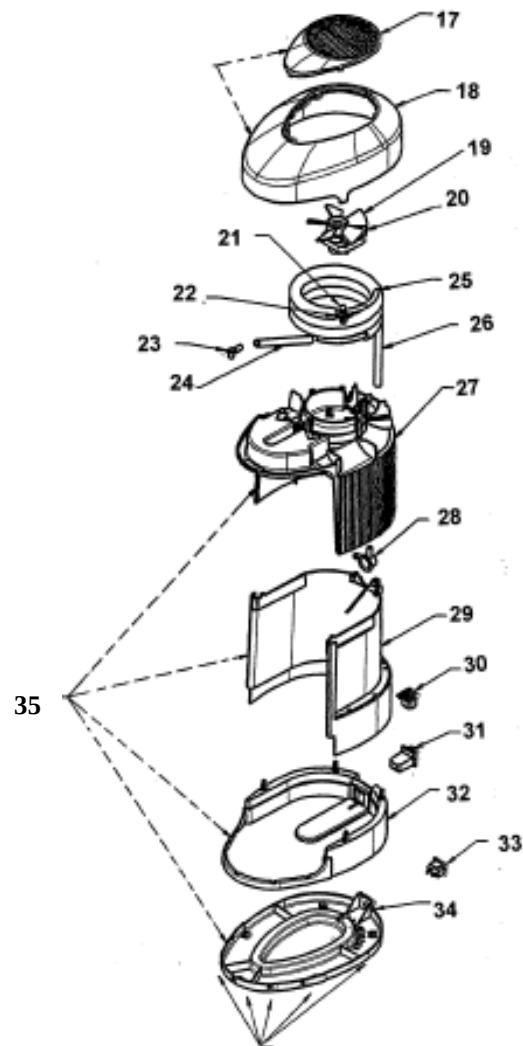

Item #	Part #	Description
17	P8-34	Vent Piece for Top Cover
18	P72-8	Cover Top without Vent Piece
19	KP76PL001	Fan Blade Clear
20	KP63MT019	Motor 110V
21	P450-42	Clip Thermostat
22	P185-97	Thermostat Condensing Coil
23	P3-192	Elbow
24	P26-93	Hose without Vent Hole
25	P84-126	Condensing Coil
26	P26-93D	Hose with Vent Hole
27		Item #27, 29, 32, 34 See 35 Chassis Kit
28	KP03WS015	Spout
30	P338-26	Strix Connector Boiler
30	P314-26	Otter Connector Boiler
31	P16-78	Strix Connector Chassis
31	P13-78	Otter Connector Chassis
33	P118-14	Power Socket Main Frame
35	P35-197	Chassis Kit Strix (oval opening)

Aquastat Reservoir

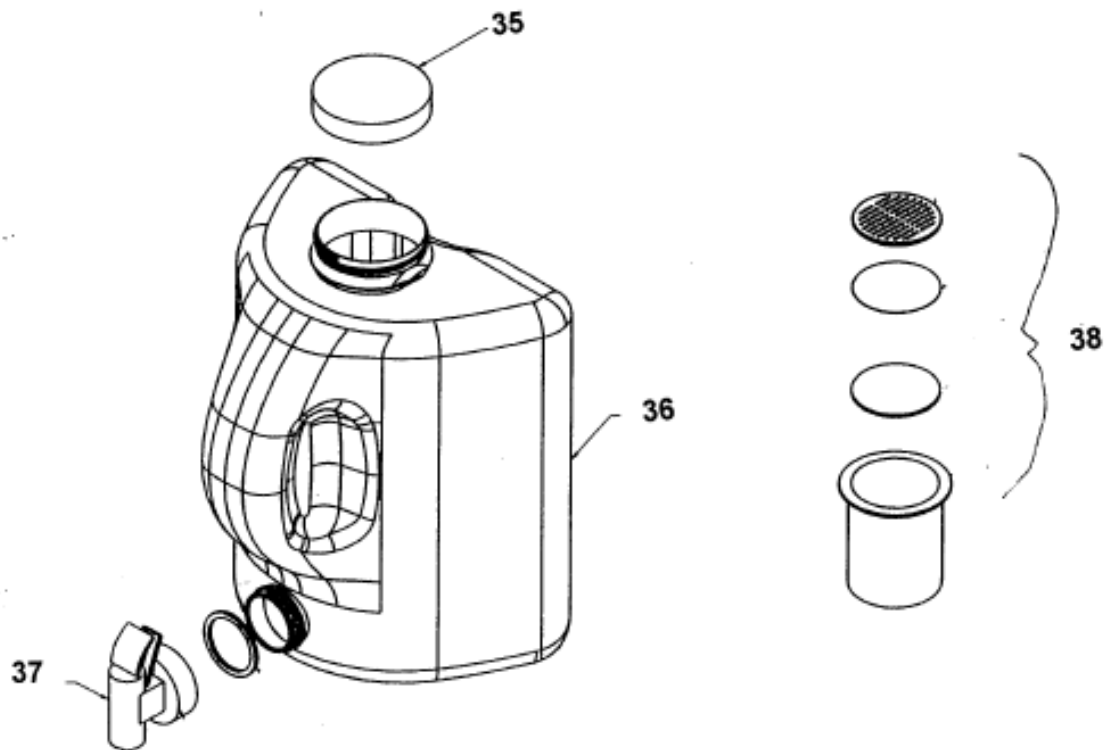

Item #	Part #	Description
35	P160-89	Reservoir Cap
36	W50211	Replacement Reservoir (Clear no longer Blue)
37	P97-104	Faucet for Reservoir
38	W8010	Carbon Filters (6 Per Pack)
Note: Carbon Filters should be used for making drinking water only		

Aquastat Parts



Baffle Gasket
#KP32GA162



Baffle Nut
#KP05EC001



Baffle O-ring
#KP32GA163



Baffle without O-ring
#KP03WS010



Boiler Base Bottom
KP44WP005



Boiling Chamber Lid
(no gasket)
#P492-2A



Boiling Chamber New Style
Oval Connector
#P206-1H



Boiling Vessel
with Heater Only
#P186-1X



Button
#P81-174



Carbon Filters 6 per Pack
#W8010



Chassis Kit
#P35-197




Clip Thermostat
#P450-42

Aquastat Parts




**Condensing Coil
#P84-126**

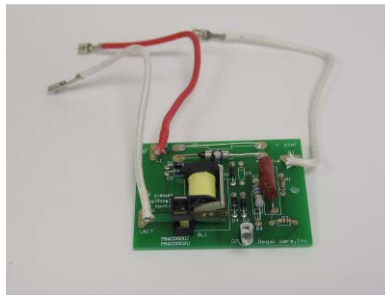

**Control Board
#KP64CO001U**



**Cover Top without
Vent Piece
#P72-8**

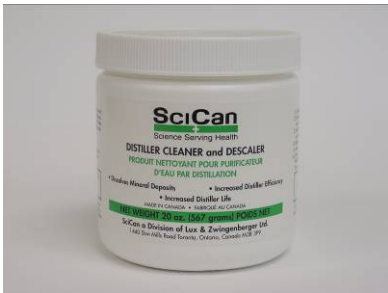

**Distiller Cleaner & Descaler
#D1-500200S**



**Distiller Wiring Kit
#P376-90K**



**Elbow
#P3-192**




**Fan Blade Clear
#KP76PL001**




**Faucet for Reservoir
#P97-104**



**Feet Gray
#P19-283**

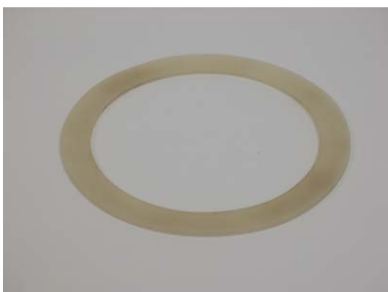

**Gasket for Boiling Chamber
Lid
#P167-72**

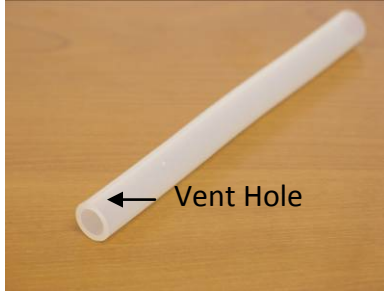

**Hose with Vent Hole
#P26-93D**



**Hose without Vent Hole
#P26-93**


Aquastat Parts



**Motor, 110V
#KP63MT019**



**Nut
#P340-81**

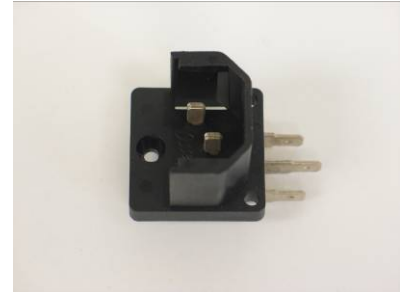

**Otter Connector Boiler
#P314-26**



**Otter Connector Chassis
#P13-78**



**Owners Manual
#L5446**



**Packaging
#J3366K**



**Power Cord
#P313-74**



**Power Socket Main Frame
#P118-14**



**Replacement Reservoir
(Clear no longer blue)
#W50211**



**Reservoir Cap
#P160-89**



**Retainer
#P118-96**



**Screw
#KP13MS570**

Aquastat Parts



Spacer
#KP12SN254



Spout
#KP03WS013



Spring
#P128-134



Strix Connector Boiler
#P338-26



Strix Connector Chassis
#P16-78



Thermostat Otter
#KP28TS091



Thermostat Strix
#P197-97




Thermostat Condensing Coil
#P185-97



Vent Piece for Top Cover
#P8-34


VistaBrite

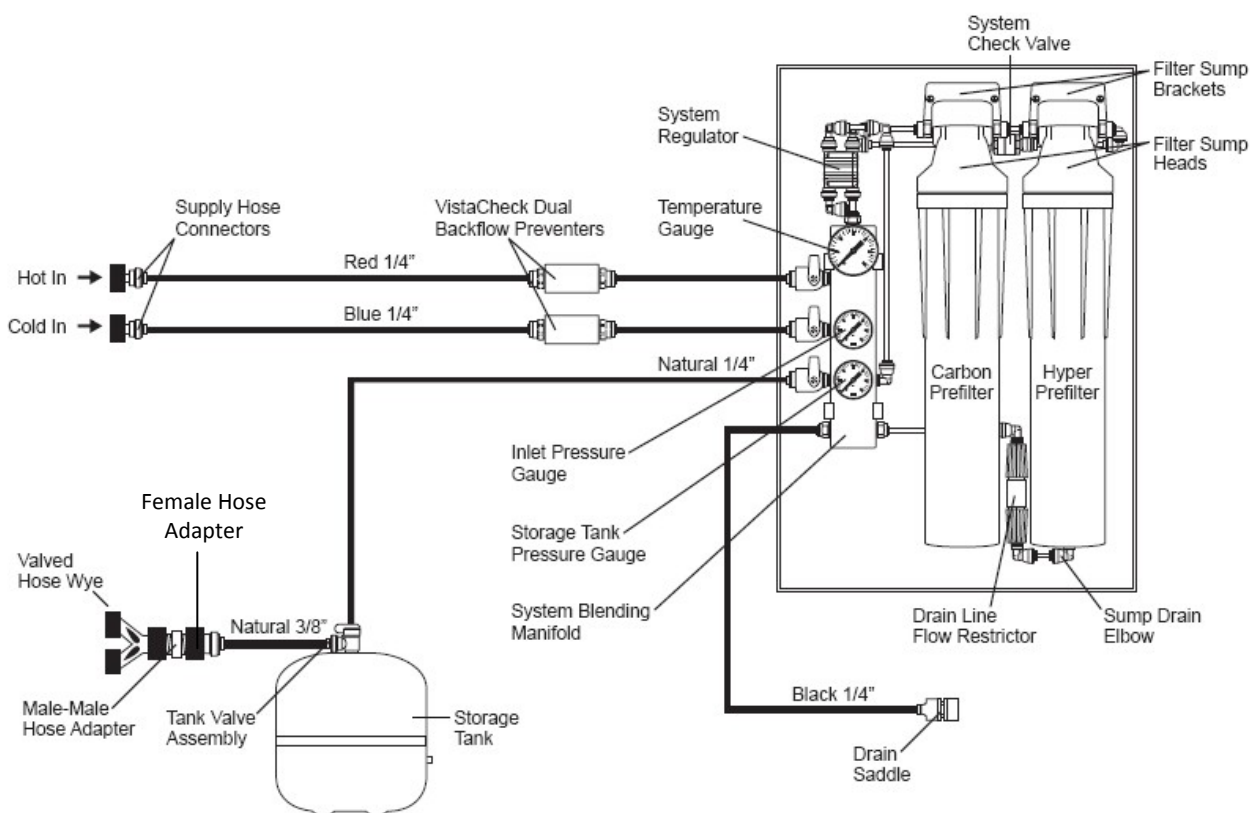

Name	Part #	Description
Supply Hose Connectors	S4231	Garden hose adaptor – Female hose x ¼" tube
Vista Check™ Backflow Preventer	VC250	Backflow check valve - In-line ¼" FNPT
System Check Valve	S6265P	In-line check valve – ¼" tube
System Regulator	S9130	Automatic shut-off valve - Hydraulic
Temperature Gauge	S6435	Temp gauge – ¼" MNPT back mount (0-250 F/C)
Filter Sump Brackets	S9170	Filter mounting bracket
Filter Sump Heads	S9171	Filter head
Inlet Pressure Gauge	S6431	Pressure gauge – Back mount ¼" MNPT (0-100 PSI)
Storage Tank Pressure Gauge	S6431	Pressure gauge – Back mount ¼" MNPT (0-100 PSI)
System Blending Manifold	S4310	System manifold ONLY (does not include fittings)
Carbon Pre-filter	R4151	Replacement carbon pre-filter (every 6 months)
Hyper Pre-filter	R4155	Replacement hyper filtration filter (approx 2 yrs)*
Drain Line Flow Restrictor	S9165	Waste water flow restrictor – ¼" tube (1500 ml/min)
Sump Drain Elbow	S6121	Male adaptor elbow – 1/8" NPT x ¼" tube
Drain Saddle	S9175	Drain saddle – ¼" tube
Valve Hose "Y"	S9180	Garden hose valve "Y"
Male-Male Hose Adaptor	S9181	Garden hose male to male adaptor
Female Hose Adaptor	S4230	Garden hose adaptor – Female hose x 3/8" tube
Storage Tank (Standard 7 GAL)	S9111	Pressure storage tank w/ stand – 7 gallon nominal
Storage Tank (Optional 14 GAL)	S9113	Pressure storage tank w/ stand – 14 gallon nominal
Tank Valve Assembly	*****	(see top-down view on following page)

* Hyper filtration filter should be changed every 2 years, or when TDS reduction is less than 70%



VistaBrite

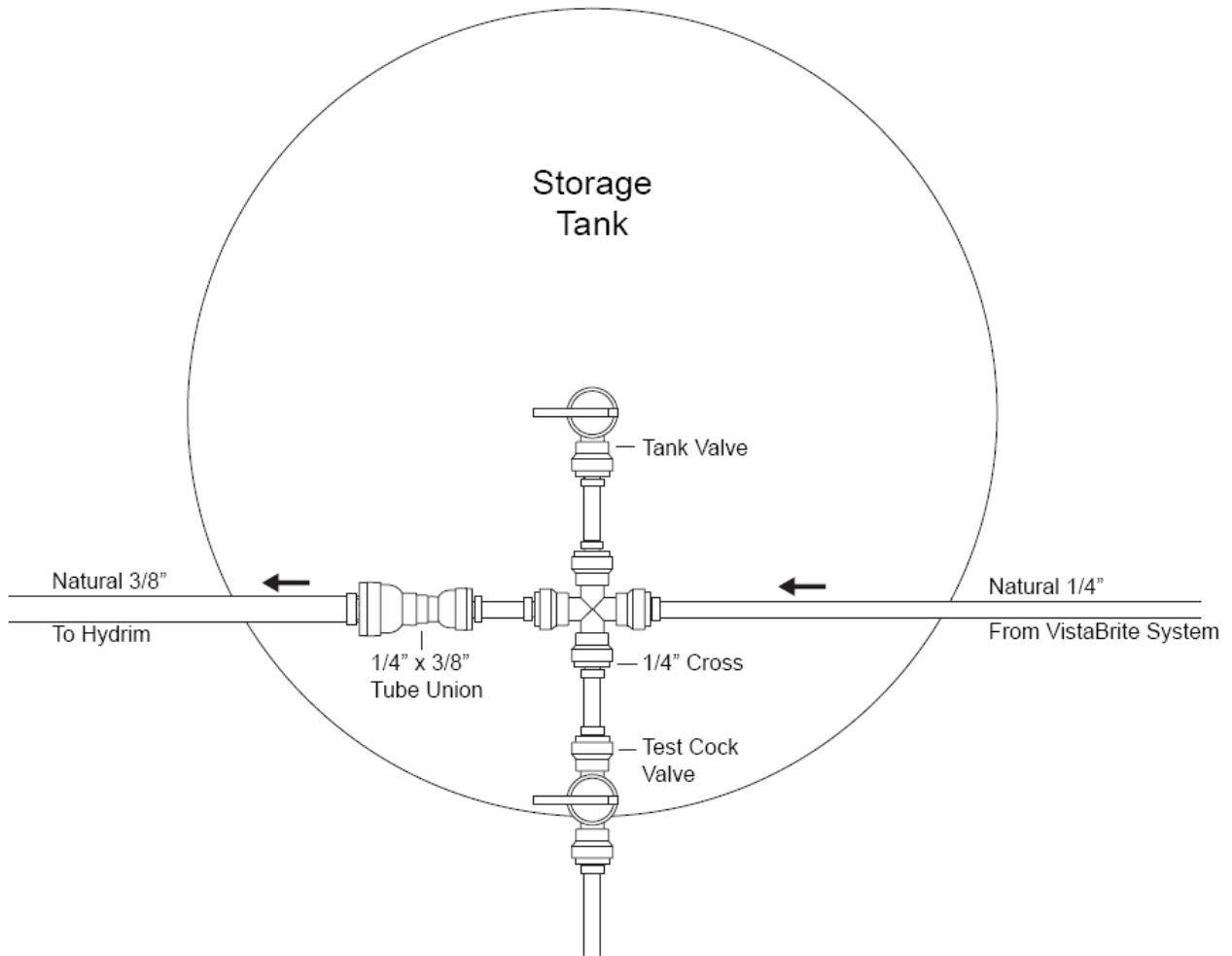

Storage Tank Tubing & Valve Assembly


Name	Part #	Description
Storage Tank (Standard 7 Gallon)	S9111	Pressure Storage Tank w/Stand (7 Gallon)
Storage Tank (Optional 14 Gallon)	S9113	Pressure Storage Tank w/Stand (14 Gallon)
Tank Valve	S6160P	Elbow Shut-Off Valve – 1/4" tube x 1/4" FNPT
1/4" x 3/8" Tube Union	S6310P	Union – 1/4" tube x 3/8" tube
1/4" Cross	S6175P	Cross – 1/4" tube
Test Cock Valve	S6155P	Valve – In-Line – 1/4" tube

VistaBrite

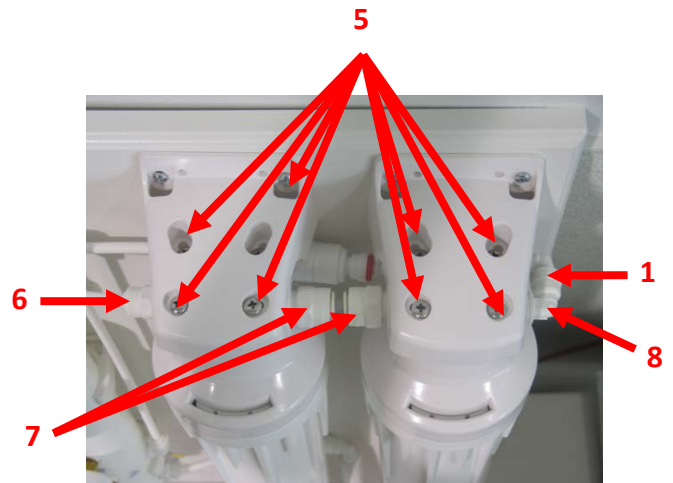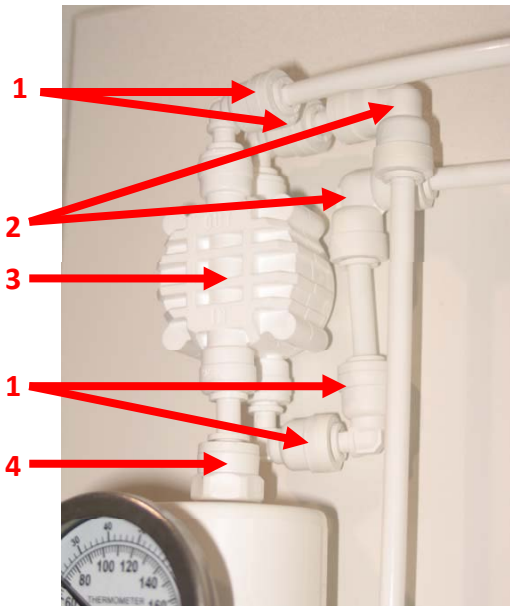

Item #	Part #	Description
1	S6112P	Swivel Plug-In Elbow – ¼" tube x ¼" stem
2	S6110P	Elbow Union – ¼" tube x ¼" tube
3	S9130	Automatic Shut-Off Valve – Hydraulic / (Regulator)
4	S6122	Male Adapter Straight – ¼" NPT x ¼" tube
5	S9172	Filter Head Self-Tapping Screws
6	S6123P	Male Adapter Straight – 3/8" NPT x ¼" tube
7	S6170P	Male Adapter Straight – 3/8" NPT x 3/8" tube
8	S6171P	Male Adapter Elbow – 3/8" NPT x ¼" tube
9	S6145P	Valve – Straight Male – ¼" MNPT x ¼" tube
10	S6570	Wall Mount "C-Clip" – 2"
11	S6120P	Male Adapter Elbow – ¼" NPT x ¼" tube


VistaCool Single V7501

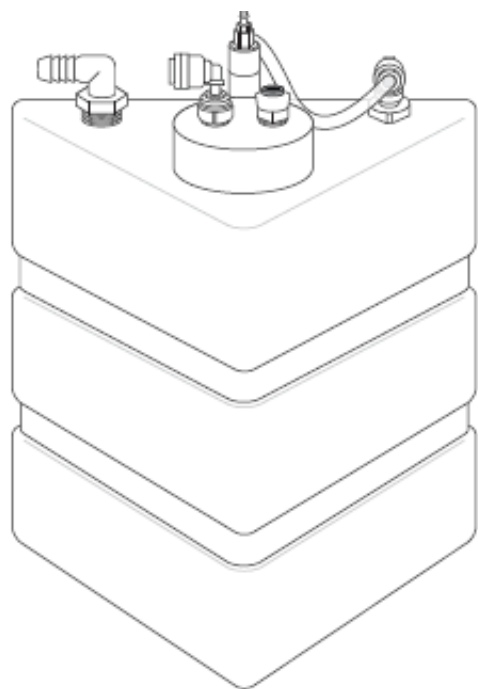

S6155P
Water Supply Valve



S7550
M3 Mono Plug

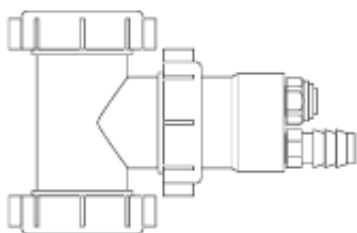

S7506
Drain Line Assembly




S7508
Inline Thermal
Sensor Assembly



VC250-A
VistaCheck Dual Check
Backflow Preventer



x4 **S7539**
Cabinet Grommet

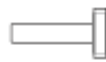

S6130P
Plug

Tubing



6' **TU751**
Cold Water Feed Line
1/4" OD x 1/8" ID
Blue Transparent Polyurethane



6' **TU753**
Condensate Drain Line
1/4" OD x 0.170" ID
Black Opaque LLDPE




6' **TU755**
Coolant Overflow Drain Line
1/4" OD x 1/8" ID
Clear PVC




9' **TU748**
High Temperature Feed Line
1/4" OD x 0.185" ID
Teflon®

VistaCool Double V7502

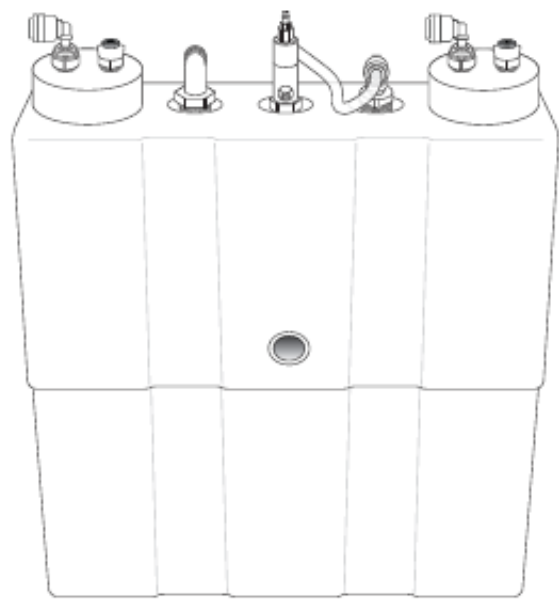

S6155P
Water Supply Valve

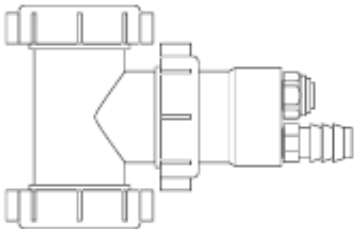

S7506
Drain Line Assembly



x2 S7550
M3 Mono Plug




x2 S7508
Inline Thermal
Sensor Assembly



VC250-A
VistaCheck Dual Check
Backflow Preventer



x8 S7539
Cabinet Grommet

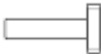

S6130P
Plug

Tubing




6' TU751
Cold Water Feed Line
1/4" OD x 1/8" ID
Blue Transparent Polyurethane



6' TU753
Condensate Drain Line
1/4" OD x 0.170" ID
x2 Black Opaque LLDPE



6' TU755
Coolant Overflow Drain Line
3/4" OD x 1/2" ID
Clear PVC

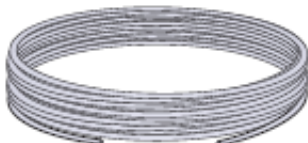

18' TU749
High Temperature Feed Line
1/4" OD x 0.185" ID
Teflon®